The joy of a dry night's sleep

Instructions for Use and Care

Please read the instructions before use

*These instructions apply to both the **DRI Sleeper[®] excel** alarm and the **DRI Excel[™]** alarm (USA market).*

*In the context of these instructions reference to the **DRI Sleeper[®] excel** also includes the **DRI Excel[™]**.*


Solving bedwetting with the **DRI Sleeper[®] excel** typically takes from a few weeks to a few months. For the best results work with your child. Teach them how the alarm works and practise what to do when the alarm is triggered. Bedwetting alarms teach your child to associate the sound of the alarm with a full bladder. If your child sleeps through the alarm it is important you wake them so they can learn this association and eventually respond to their full bladder before the alarm triggers.

Keep using the **DRI Sleeper[®] excel** EVERY NIGHT until there have been no wet beds for 14 consecutive nights.


DRI Sleeper[®] excel Alarm

The *DRI Sleeper[®] excel* uses a plug-in conductive plastic sensor to detect when the child wets the bed at night. The *DRI Sleeper[®] excel* comprises:

- A clip-on body worn alarm that can be easily attached to the shoulder of the night clothes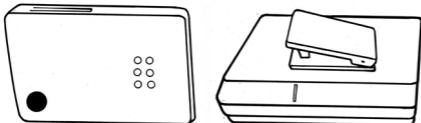


- One *Urosensor[™]*

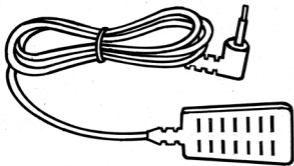- Four 1.5V alkaline button cell batteries (included in the alarm)

batteries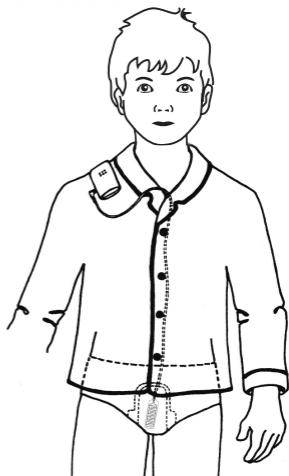

Cleaning the *DRI Sleeper® excel Alarm*

Alarm: Wipe clean with a damp cloth that has been soaked in mild detergent or skin disinfectant and wrung out. **NEVER** immerse the alarm in water or locate it where urine can enter the device.

Urosensor™: Rinse in liquid soap and water and dry well.


NOTE: do not use chlorine-based detergents or cleaners or put the **Urosensor™** in the washing machine or dryer as this will irreparably damage the sensor.

How to use your *DRI Sleeper® excel*

1. Plug the **Urosensor™** into the *DRI Sleeper® excel* alarm.
2. Test the alarm by placing something metal (stainless steel knife or stainless steel scissors) across the **Urosensor™** and the alarm will trigger.
3. Unplug the **Urosensor™** from the alarm box to disconnect the alarm after testing.
4. Clip the alarm to the child's night clothes – at the shoulder near the ear is best and avoids the child lying on it.
5. Place the **Urosensor™** inside a minipad (a women's small or regular sanitary pad) by making a pocket in the absorbent layers of the material. To do this, cut a 25mm (1 inch) slit into the absorbent layers with a sharp pair of scissors.
Alternatively, cut the top off the minipad. With the point of the scissors create a pocket for the **Urosensor™** to slide into.

NOTE: the top layer of a minipad is designed to stay dry and if the **Urosensor™** is placed just under the top layer it may not become wet and trigger the alarm when your child wets. Make sure the **Urosensor™** is inserted under some layers of absorbent material so that they will become wet and press onto the **Urosensor™**.

Avoid using the very thin type of minipad because they can cause unreliable triggering e.g. triggering in response to sweat or not triggering at all due to insufficient absorbent material.

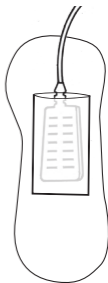6. Stick the minipad into the crotch of the underwear close to where the urine will escape.
7. Put the **Urosensor™** cord inside the night clothes and extend it up to the alarm. Plug the **Urosensor™** into the alarm.

CAUTION: DO NOT wrap the **Urosensor™** cord around your child's neck as this may lead to strangulation.

8. When the alarm triggers make sure your child gets out of bed promptly, unplugs the **Urosensor™** from the alarm to disconnect it and goes to the bathroom.
9. Remove the **Urosensor™** from the minipad and wash it in liquid soap or detergent and dry it well.

NOTE: Do not use chlorine-based detergents or soaps as these will irreparably damage the sensor.

10. Insert the **Urosensor™** in a new, dry minipad.
11. Plug the **Urosensor™** back into the alarm before getting back into bed.
12. Repeat steps 5-10 if the alarm triggers again.

NOTE: During the day, and when not in use, unplug the **Urosensor™** to save battery life and put it in a safe place to prevent damage.

13. Keep using the **DRI Sleeper® excel** every night until there have been no wet beds for 14 consecutive nights.

Other ways to position the **Urosensor™**

Using a paper handkerchief or some tissue

1. Wrap the **Urosensor™** in ½ a 2-3 ply paper handkerchief or tissue.
2. Put on a pair of close fitting underpants.
3. Insert the wrapped **Urosensor™** inside the underpants near where the urine will first escape.

Using double-gusset underpants

1. Cut a 25mm (1 inch) slit through one layer only of a pair of double-gusset underpants, down towards the crotch area. The cut can be made on the inside or the outside of the underwear.

2. Insert the *Urosensor*[™] inside the slit near where the urine will first escape.
NOTE: Wrapping the *Urosensor*[™] in a paper handkerchief or tissue before inserting it into the slit in the gusset can also help bedwetting to be detected more quickly.

Using two pairs of underpants

1. Put on two pairs of close fitting underpants.
2. Insert the *Urosensor*[™] in between both pairs of underpants above the crotch area, near where the urine will first escape.

NOTE: Wrapping the *Urosensor*[™] in a paper handkerchief or tissue before inserting it between the underpants can also help the bedwetting to be detected more quickly.

Using a disposable diaper or pull-ups

1. With a sharp pair of scissors, cut a 25mm (1 inch) slit in the absorbent layers on the inside of the pull-up diaper near the area where the urine will first escape.
2. With the point of the scissors, create a pocket for the *Urosensor*[™] to slide into.
3. Insert the *Urosensor*[™] inside the pocket.

NOTE: Make sure the *Urosensor*[™] does not slide down towards the back of the diaper or pull-ups away from the initial wetting area.

NOTE: As with the minipad, it is important the *Urosensor*[™] is inserted under some absorbent layers. However, disposable diapers tend to be highly absorbent so wrapping the *Urosensor*[™] in a piece of toilet paper before inserting it in the pocket can help the moisture reach the *Urosensor*[™] before it is absorbed by the diaper. Diapers that include an absorbent gel layer should be avoided as these will not result in sufficient moisture getting to the *Urosensor*[™] to enable it to trigger.

Sewing a pocket into the underwear

1. If you do not wish to use any of the above methods, then a piece of material can be sewn into the crotch of the underwear to make a pocket near where the urine will first escape and the *Urosensor*[™] can be inserted into the pocket. The pocket must fit the *Urosensor*[™] tightly to hold it in place and must be made of a material that allows the urine to pass through it onto the *Urosensor*[™].

‘Priming’: preparing your child to use the alarm

To help your child use the alarm it is recommended you practise what to do when the alarm triggers during the night. This is called ‘Priming’.

Priming is based on a strategy called Prospective Memory which is setting something up in the brain that has to be remembered in the future. In order to prime your child’s brain to wake to the sound of the alarm, it is a matter of practising what she or he needs to do when the alarm sounds.

Before your child settles down to go to sleep have him or her lie in bed pretending to be asleep. Plug in the *Urosensor*[™] and trigger the alarm by putting something metal across the *Urosensor*[™]. Ask your child to get up, unplug the alarm and go to the bathroom.

Practise this three or four times so that your child’s brain is more ready to respond when she or he hears the alarm.

At the outset you may also need to get up when the alarm sounds in the night to ensure your child follows this procedure. Also, depending on your child’s age, you may need to assist him or her prepare the *Urosensor*[™] for re-use.

We recommend that parents do this every night until the child is successfully responding to the feeling of a full bladder and waking to go to the bathroom before the alarm sounds. This may take a week or two, but in the end it will be worth it.

Troubleshooting

1. The *Urosensor*[™] does not trigger the alarm when tested prior to use. This means either:

a. The batteries in the alarm need replacing.

To replace the batteries:

- Remove the back of the alarm casing by separating the two raised areas on the side of the alarm. This is easy to do by putting any rigid item between them and twisting gently.
- Take care not to touch the internal circuitry.
- Insert 4 new LR44 button batteries in the direction indicated by the diagram inside the alarm.

b. The *Urosensor*[™] has worn out or has broken wires due to wear and tear.

NOTE: Just as the earphones of on an i-phone or i-pod need replacing the *Urosensor*[™] will also need to be replaced over time.

At any time you can test the functioning of the *Urosensor*[™] by plugging the *Urosensor*[™] into the alarm and slowly dipping the sensor into a glass of hot water starting with the first set of raised black detecting bars on the sensor. Note how many raised black detecting bars need to be inserted before the alarm triggers. If more than 6 pairs of bars need to be inserted it is time to replace the *Urosensor*[™].

2. The child unplugs the *Urosensor*[™] during the night.

In this case it can be helpful to put a rubber band tightly around the alarm with the *Urosensor*[™] cord under the rubber band.

3. The batteries are rusty.

This is not caused by faulty batteries. It is caused by water having entered inside the alarm casing e.g. by being put though the washing machine. If the batteries have got wet it is also likely the circuitry in the alarm will be damaged and the alarm will need to be replaced.

4. The alarm has suddenly lost its volume.

Sudden lowering of the volume will be due to interference with the buzzer.

This is not a warranty issue.

BATTERIES

Four LR44 button batteries are provided free of charge with the *DRI Sleeper*[®] *excel* alarm. Replacement batteries should be available from local electronic and photographic stores.

Battery life depends on usage. The batteries should last 6 months with nightly use provided the alarm is unplugged when not in use.

WARNING: Batteries should **NOT** be accessed by small children and should only be replaced by adults. If a battery is accidentally swallowed by a child please go **IMMEDIATELY** to your local hospital emergency centre. Do not allow your child to eat or drink anything until examined by a doctor.

Always dispose of batteries in an environmentally responsible and safe way.

MANUFACTURER

Made in New Zealand by Anzacare Limited

PO Box 400, 13 Mahara Place, Waikanae 5036, NEW ZEALAND

www.dri-sleeper.com

Do not dispose of the alarm as unsorted municipal waste.

It must be recycled.

WARRANTY

The manufacturer will replace any alarm which fails due to manufacturing fault within 6 months of purchase (proof of purchase date must be provided).

Contact info@dri-sleeper.com if you have a warranty claim.

This warranty does not cover batteries or misuse of or accidental damage to the alarm or *Urosensor*[™].

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Vervaardigd in Nieuw-Zeeland door Anzacare Limited