
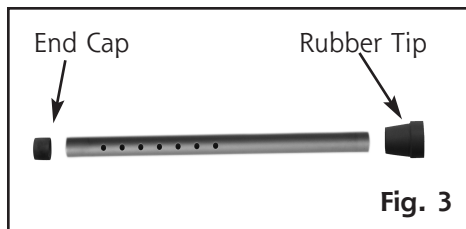
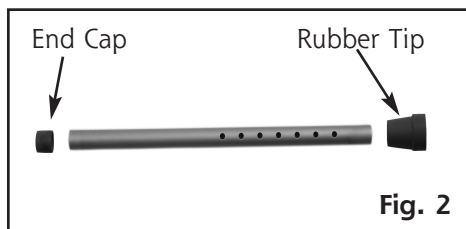


# Hi-Lo Cane

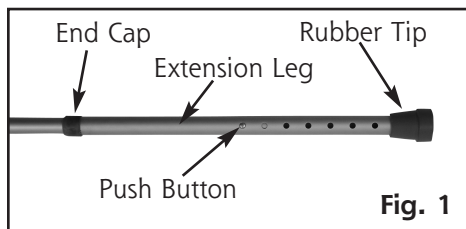

## Height Adjustment Instructions

- ▶ 1. To adjust the cane from 28-1/2" to 34-1/2", press the push button and slide the extension leg until the desired length is reached and the push button locks into place, Fig. 1.
- ▶ 2. To adjust the cane from 34-1/2" to 40-1/2" press the push button and slide the extension leg off completely.
- ▶ 3. Remove the end cap and rubber tip from the extension leg, Fig. 2.
- ▶ 4. Turn extension leg over and replace the end cap and rubber tip on the opposite ends of the extension leg, Fig. 3.
- ▶ 5. Slide the extension leg back onto the cane until the push button locks in place at the desired height.

**CAUTION:** For your safety, the push button must be completely through the hole in the extension leg before use.

**NOTE:** If any parts are missing, do not return to the place of purchase. Contact us at 877-505-6190.

Distributed by:  
 Duro-Med Industries  
 Div. of DMS Holdings Inc.  
 1931 Norman Drive South  
 Waukegan, IL 60085 USA  
 www.duromed.com

Made in Taiwan R.O.C.

Toll-Free Customer  
 Care Help Line  
 877-505-6190

#910-1302-0600 12/06  
 ©2006 Duro-Med Industries