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RTL2017CTW

Drive DeVilbiss Healthcare
99 Seaview Blvd.
Port Washington, NY 11050
1-877-224-0946
www.drivemedical.com

Elevation Wedge

COMFORT TOUCH™ Cushions by Drive™ are designed to help satisfy your therapeutic needs while immersing you in comfort with an ergonomic design, high-quality foam and luxurious look and feel.

Beneficial Uses

The Comfort Touch Elevation Wedge has a super soft velour cover and is ergonomically designed to provide comfort for those that seek relief from snoring or to ease breathing. At 10" in height, this bed wedge is also designed to help alleviate symptoms associated with acid reflux and gastroesophageal reflux disease (GERD).

How to Use

Unroll compressed cushion. Carefully remove product from plastic to start the decompression process; being careful if using scissors so as not to cut the cushion. Cushion will decompresses to full size in 1-2 hours.

Drive's Comfort Touch Elevation Wedge is intended to relieve snoring, ease breathing and help reduce symptoms associated with acid reflux and GERD. **Seek medical attention immediately if symptoms persist or condition worsens.**

Care and Cleaning Instructions

Cover: Remove the velour cover by unzipping and removing foam. Wash it on a cold gentle cycle with mild soap as needed. Do not bleach. Tumble dry on low heat or allow to air dry.

Memory Foam: If the memory foam has any odor or stains, surface clean it with a moist cloth and air it out until it's completely dry. No industrial solvents should be used at any time. Any lingering odor will dissipate over time. Never put foam in washer or dryer.

One Year Limited Warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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