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Instructions for Use – Please Read All Instructions Before Use.

Please consult a healthcare professional for the proper adjustment.

Opening the Rollator

1. Remove Rollator and other components from carton.
2. Unfold frame; spread front and back legs apart.
3. Push down locking mechanism to secure legs in place.
NOTE: Ensure the side crossbars (locking mechanism) lock down.

Folding the Rollator

1. If necessary, remove the accessories and lift the seat.
2. Pull up on the Folding Pull Strap that is attached to crossbar (locking mechanism) underneath the seat.
3. Push the two handgrips together.

Using the Hand Brake

Brakes should be used in one of the following methods:

- To Remain Stationary: Hold the hand brake up.
- To Continue Mobility: Release the brake. To unlock, pull up on the top portion of the brake handle.
- To Sit on Seat: Push down on the bottom portion of the brake handle to lock.
- To unlock, pull up on the top portion of the brake handle.
CAUTION: Always lock hand brakes when user is immobile; never sit on rest seat if hand brakes are not locked.

Adjusting the Height

1. Remove the adjustment knob and bolt by turning counterclockwise.
2. Consult with healthcare professional for proper adjustment.
3. Install the adjustment knob into one of the adjustment holes by turning clockwise.
4. Repeat STEPS 1 – 3 for the other side.

Make sure that the height adjustment is at the same height for both handles to ensure safety and proper use.

NOTE: After adjusting the height, the brakes of the rollator must be checked and, if necessary, adjusted.

Instructions for Use – Please Read All Instructions Before Use.

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LUMEX®

A Graham-Field Brand

- Improves mobility and allows for greater independence for the user on the go
- Lightweight design makes rollator easy to use, fold and store for convenient transport
- Padded seat and backrest provide additional comfort
- Ergonomic hand grips and hand brakes allow user to operate with ease
- Complete with basket for storage of personal items
- 6" wheels are ideal for indoor and outdoor use
- RJ4302 model has a lower seat position for shorter individuals
- 300 lb weight capacity (350 lb on RJ4318 Wide models)
- Available in various attractive colors
- Newly added Reflectors increase visibility

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© 2011, GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, Georgia 30360.
Hecho en China. www.grahamfield.com

Specifications subject to change without notice
Especificaciones son sujetas a cambio sin aviso

GF0600057RevC11

LUMEX®

Walkabout Lite, Hemi and Wide 4-Wheel Rollator

Walkabout Lite, Hemi and Ancho Rodador de 4 Ruedas
Items RJ4300, RJ4302 & RJ4318

User Instructions and Warranty Information

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Lifetime Limited Warranty

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products. We are dedicated to providing products that will give our customers years of dependable service.

GF Health Products, Inc. hereby warrants specific parts of the Lumex models listed on the cover of this hangtag for the lifetime of the product against defective workmanship and/or material defects when used for the purpose intended, under normal conditions, and provided the product receives proper care.

Parts covered under the lifetime warranty include: frame, welding, screws, and frame locking components. Specific items "excluded" from this warranty of any kind are: "ALL" other parts not specifically listed above. The warranty does not extend to non-durable components, such as tips, which are subject to normal wear and require periodic replacement. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to the manufacturer.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this Lumex product or this warranty, please contact an authorized GF Health Products, Inc. Distributor.

Attaching Basket, Pouch or Tote Bag

Basket

Install the basket underneath the seat. On some rollator models, the basket is designed to fit underneath the seat or in the front by positioning the hooks on the front seat support frame.

Pouch and Tote Bag

Hang the tote bag between the rollator frame tubes and secure with snap button flaps.

Maintenance

- For dust and dirt, wipe Rollator gently with damp, soft cloth.
- Inspect wheels periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the rollator.
- Verify operation of the brakes and have them adjusted if necessary by your GF Health Products, Inc. authorized distributor.
- Inspect Rollator weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrip or hand brakes are loose, **DO NOT** use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- For the best service from your rollator, always use replacement parts from GF Health Products, Inc.
- Periodically inspect the casters and caster stems for assembly tightness and verify that the wheels are free of hair or other debris that could interfere with free wheel operation.

Item Number	Weight Capacity
RJ4300s	300 lb
RJ4318s	350 lb
RJ4302s	300 lb
RJ4805s	300 lb
RJ4400s	500 lb
RJ4405s	500 lb
RJ4402s	500 lb
609201s	300 lb

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WARNING: Indicators a potential hazard situation or unsafe practice that, if not avoided, could result in death or serious personal injury.

CAUTION: Indicates a potential hazard or unsafe practice that, if not avoided, could result in personal injury or product/property damage.

NOTE: Provides application recommendations or other useful information to ensure that you receive the greatest benefit from this product.

WARNING

WARNING: DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: DO NOT use the rollator as a wheelchair or transport device. Rollators are not intended to be propelled while seated.

WARNING: If components are damaged or missing, contact your distributor immediately. DO NOT substitute parts.

WARNING: Before use, ensure that all assembled parts are secure and all hardware is tightened.

WARNING: Care should be taken to ensure that all hand and height adjustments are secure, and that wheels and moving objects are in good working order before using this rollator.

WARNING: When using the rollator in a stationary position, the hand brakes MUST be locked.

WARNING: ALWAYS observe the user weight limitation on the hangtag of this product and on the chart referenced at left.

WARNING: DO NOT attempt to reach objects if you have to move forward in the seat. Reaching for these objects will cause a change to the weight distribution of the rollator and may cause the rollator to tip, resulting in injury or damage.

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