


Medintim

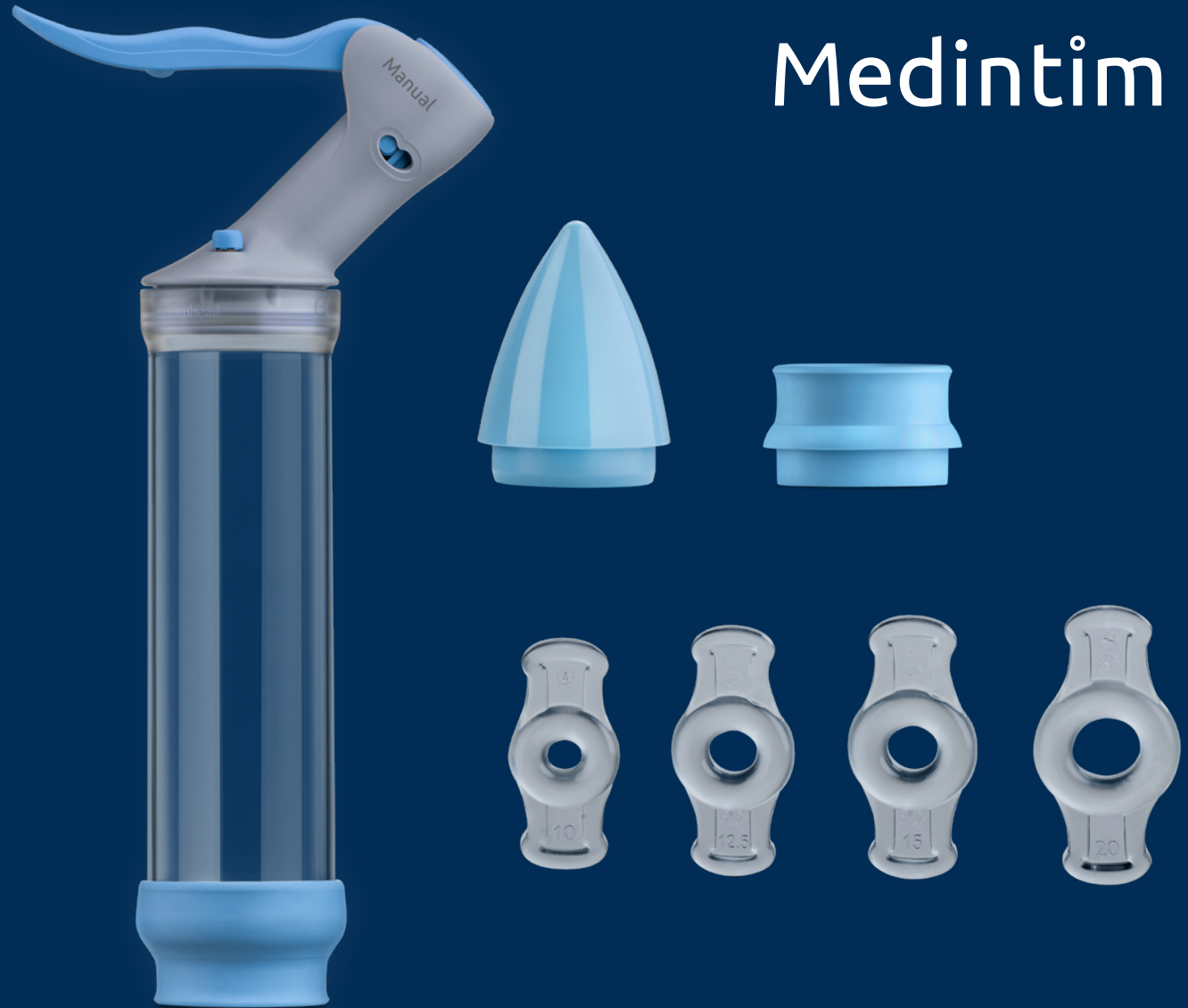

Manual

Erection System

Vacuum therapy for Erectile Dysfunction (ED)

Medintim

Place the pump head on the cylinder.

Manual
Erection System

Medintim

Place guide sleeve II.



Manual
Erection System

Medintim

Place guide sleeve I.

Manual
Erection System



Medintim

Place
the cone.

Manual
Erection System



Medintim

Slide on a
constriction ring
using the cone.

Manual
Erection System





Medintim

Alternatively, slide
on a constriction ring
using your hands.

Manual
Erection System

A person is shown from the side, applying a blue medical device to their upper arm. The device consists of a blue cap and a blue tube. The person's hand is visible, holding the device against their skin. The background is a solid blue color.

Medintim

Application on the body

A close-up photograph of a person's torso and arm. The person is holding a manual erection device, which consists of two clear cylindrical chambers connected by a central tube. The device is held against the person's arm. A hand is using a small spray bottle to apply lubricant to the blue cap of one of the chambers. The background is a warm, orange-toned gradient.

Medintim

Apply
lubricant.

Manual
Erection System



Medintim

**Introduce
the penis into
the cylinder.**

Manual
Erection System

Medintim

Hold the device
against your body
and pump.

Manual
Erection System



Medintim

Push the
constriction ring
onto the penis.

Manual
Erection System



Medintim

The constriction ring maintains the erection (max. 30 min.)

Manual
Erection System

A close-up photograph of a person's hands applying a dark red, cylindrical constriction ring to their penis. The person's torso and arms are visible, and the background is a warm, orange-toned gradient. The ring is being slid onto the base of the penis.

Medintim

**After application:
Slip off the constriction ring
and remove it.**

Manual
Erection System


A close-up photograph of a man's hands and lower torso. The man is holding a clear plastic applicator in his right hand and a small, dark, cylindrical object in his left hand. A blue callout box with a white border points from the text 'State after application.' to the man's penis, which is partially visible and appears to be in a state of erection. The background is a warm, orange-brown gradient.

Medintim

State after application.

Manual
Erection System

Medintim

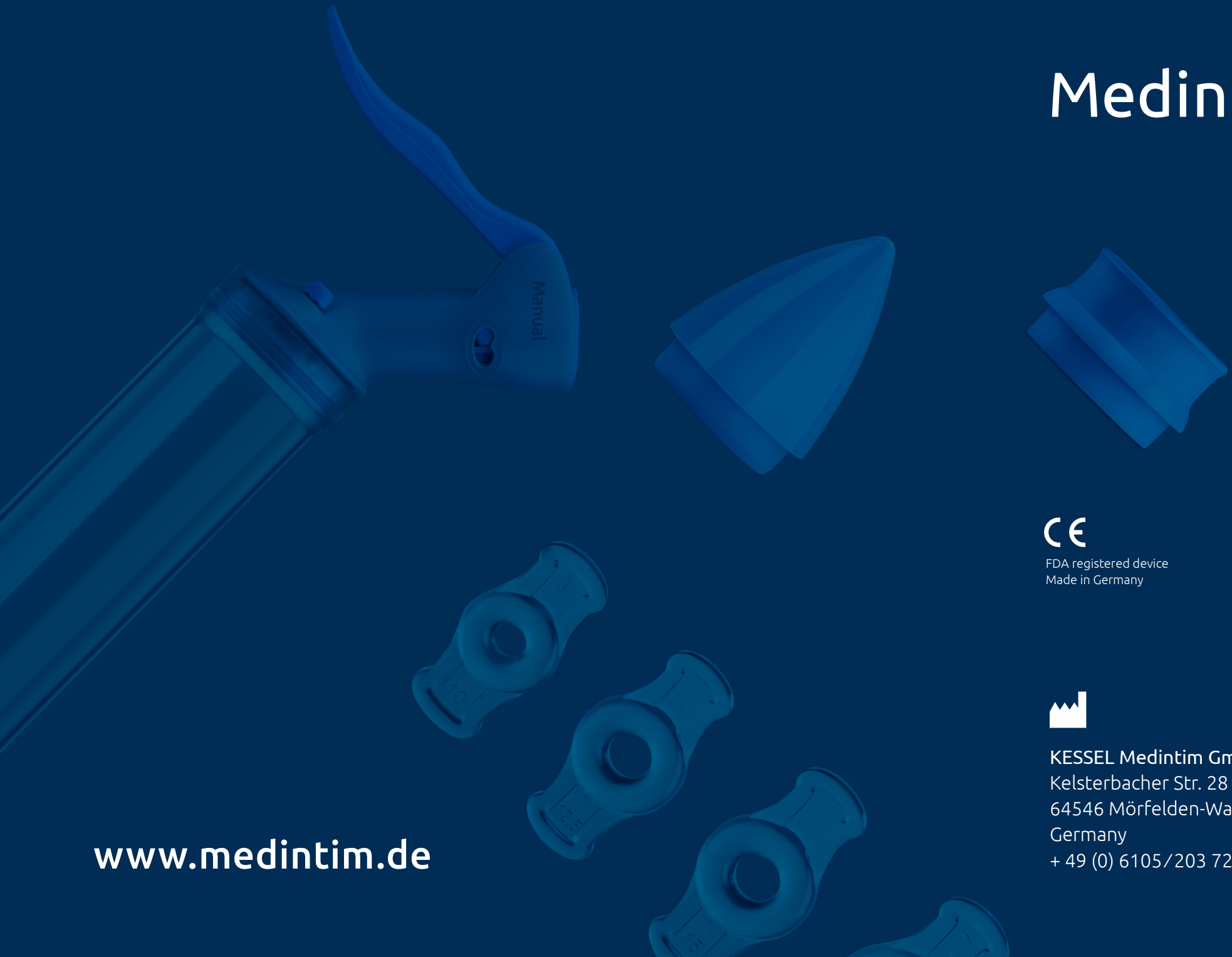

FDA registered device
Made in Germany



KESSEL Medintim GmbH
Kelsterbacher Str. 28
64546 Mörfelden-Walldorf
Germany
+ 49 (0) 6105/203 720

www.medintim.de