

## Collect Urine Sample

1


(A) Collect urine specimen.



(B) Recap the cup and place it on a flat surface. Peel off label to view results.




(C) Read drug strips at 5 minutes. Do not read the test results after 30 minutes.

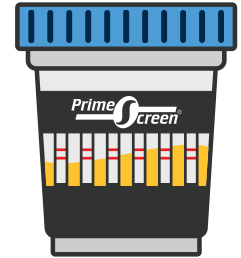


**IMPORTANT:**  
Make sure the cup is tightly capped.


## Reading Results

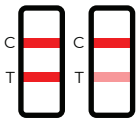

Negative

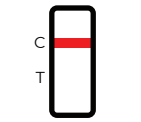

Preliminary Positive

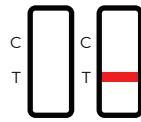

Invalid

### Sending the Sample for Confirmation

We recommend sending all preliminary positive result samples to the laboratory for confirmation testing. Mail the sample as soon as possible for accurate analysis. If test is negative, there is no need to proceed.

See below and reverse side for more details.

**Important:**

*The sample cannot be tested if it is more than 7 days old.*

*At least 90 MLs urine is recommended for lab testing.*



## Laboratory Confirmation Service

(A) To use our laboratory confirmation service, please scan QR Code, or visit us at

[primescreentest.com/products/laboratory-confirmation-service](https://primescreentest.com/products/laboratory-confirmation-service)

and create an account prior to your purchase. **Enter specimen ID # when creating account!**



2

(B) Next, return to the laboratory confirmation service page, insert your Specimen ID Number and email after the description prior to checkout.

(C) Once your order is completed, you will receive a prepaid shipping label in your email within 30 minutes.

**NOTE:** All orders after 6pm CST will be processed the next business day. Friday night, & weekend orders will be processed the following Monday.

# Send to our Laboratory


# 3

- (A)** Fill out the Laboratory Chain of Custody form.\* Next, date and peel the specimen ID label off from the bottom of the form and attach the label on top of the cup to seal it. Please make sure the cup is tightly capped.

\* *Select the drugs that tested preliminary positive to indicate which drug(s) need confirmation testing at the laboratory.*

- (B)** Once completed, please take off the first page of the transfer forms and place it inside the rear pouch of the plastic bio bag. **Keep the form under the first page for your records.**
- (C)** Place the labelled cup into the front pouch of the bio bag. Then, seal the bag.
- (D)** Place your sealed bio bag into the FedEx bag, and seal it. Attach the prepaid shipping label to the FedEx Bag and take your sample to a FedEx drop off location. Keep the tracking information for your records.

**If the urine sample is mailed the same day as collected, it can be kept at room temperature; otherwise the sample should be refrigerated or frozen.**

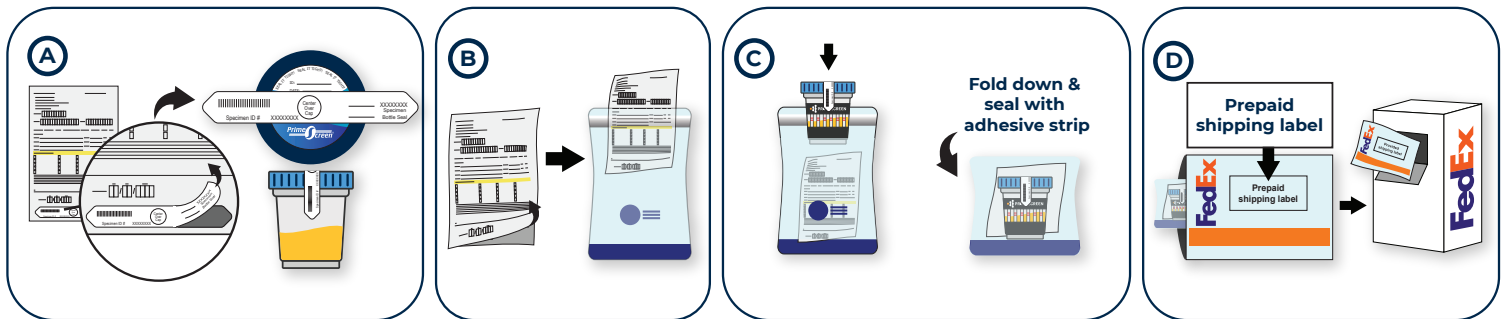

# View your results

- (A)** To view your results online, you can log in to your account portal on our website, or by scanning the QR code to the right, and clicking on the laboratory confirmation order.



- (B)** Results will be available within 5 business days after receipt at the lab. Once results are fully analyzed, you will receive an email with an order completion confirmation.

# 4

## Help From a Substance Abuse Professional

The Substance Abuse and Mental Health Services Administration (SAMHSA) National Helpline is a free, confidential, 24/7/365 treatment and information service (in English and Spanish) for individuals and families facing substance abuse and mental health disorders.

This service provides referrals to local treatment facilities, support groups and community-based organizations. Call 1-800-662-HELP (4357).