

STEP 1 - DIP

Submerge the test panel **until all the dyes rise to the surface**.
Do not submerge past the "MAX" line.



STEP 2 - WAIT


Place the test on a dry, non-absorbent surface.
Allow the test to sit for 5 minutes.



STEP 3 - READ

Critical to read the results precisely **after 5 minutes** for accuracy result.

Avoid reading after 10 minutes.

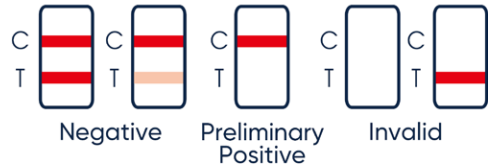

This test doesnt just tell you **POSITIVE** or **NEGATIVE**,

The Primescreen Multi-level THC test features a range of test levels. From 15ng/mL - the most sensitive on the market, to a broad 300ng/mL level, multi-level testing is **perfect** for tracking your sobriety progress.

