

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

This product does not belong to medical, please don't use for disease treatment!


Table of Contents:

- 1. Introduction to Manual
- 1.1 Purpose
- 1.2 Conventions
- 2. Medical Safety.....
- 2.1. Indications for Use.....
- 2.2. Contraindications
- 2.3. Precautions
- 2.4. Patient Safety.....
- 2.5. Potential Side Effects.....
- 3. System Overview.....
- 3.1. System Description.....
- 3.2. System Components
- 4. Package Listing.....
- 5. Installation.....
- 6. Treatment
- 6.1. Breast Lifting
- 6.2. Butt Lifting
- 6.3. Rhythm Massage Treatment
- 6.4. Scrub and cupping Treatment
- 7. Recommend Treatment Time
- 8. Attention




1. Introduction to Manual

1.1 Purpose

This instructions for use manual provides a description of the system components, its controls and displays, instructions for its operation, and other equipment information important to the user

-  **Warning:** Do Not operate this machine before reading this manual thoroughly. In addition to this manual,additional clinical training may be available by the company or your local distributor.
For more information on training available please contact your local representative.

1.2 Conventions

-  **Note:** Notes designate information of special interest.
-  **Caution:** Cautions alert the user to precautionary steps necessary to properly operate the system.Failure to observe these cautions may void the warranty
-  **Warning:** Warnings alert the user to information that is of the highest importance and vital to the safety of the patient and user

All procedures are broken down by numbered steps.Steps must be completed in the sequence they are presented

Bulleted lists indicate general information about a particular function or procedure.They do not imply a sequential procedure.

Control names are spelled as they are on the system,and they appear in **Bold text**

2. Medical Safety

2.1. Indications for Use

The breast enhancement lymphatic drainage system is indicated for use as a non-invasive

dermatological aesthetic treatment to:

- Breast butt care: Breast plumping & lifting, correct nipple & chest, recover breast elasticity face, chin, neck Care: Face thinning, removal nose blackhead, acne, eye circle, eye bag, blood marks, double chin & lifting canthus
- Shoulder, back, arm, waist, thigh, leg care: negative pressure metabolism, removal measles, press cavity, toxin excrete, tighten skin, improve skin elasticity, lymph toxin removal

2.2. Contraindications

The breast enhancement lymphatic drainage system is contraindicated for use in patients with:

- Open wounds or lesions on the face and/or neck
- Severe or cystic acne on the face and/or neck

2.3. Precautions

The breast enhancement lymphatic drainage system has not been evaluated for use over various materials. Therefore, treatment is not recommended directly over those areas with any of the following:

- Mechanical implants
- Dermal fillers
- Implanted electrical devices in the face and/or neck
- Metal stents in the face and/or neck area

Treatment energy is not recommended for use directly on an existing keloid.

breast enhancement lymphatic drainage system has not been evaluated for use in patients on an anticoagulant treatment plan

It is recommended that the following areas should be avoided during treatment:

- Thyroid gland, thyroid cartilage and trachea
- Major vessels

The breast enhancement lymphatic drainage system has not been evaluated for use in the following patient populations:

- Pregnant or breast feeding women
- Children
- Those with the following disease states
 - A hemorrhagic disorder or hemostatic dysfunction
 - An active systemic or local skin disease that may alter wound healing Herpes simplex
 - Autoimmune disease Diabetes
 - Bell's palsy

2.4. Patient Safety



Warning: breast enhancement lymphatic drainage system should not be used on a patient's eyes or in a location or technique where ultrasound energy can reach the eye



Warning: Use this system only if you are trained and qualified to do so.



Warning: If any problems occur during system operation, ask immediate action(s): lift the transducer off the patient's skin, press the STOP button to completely halt system operation.

2.5. Potential Side Effects

Side effects reported in the clinical evaluation of the breast enhancement lymphatic drainage system were mild and transient in nature. These were limited to:

- Erythema (redness): The treated area may exhibit erythema immediately following Treatment. This typically resolves within a few hours of treatment.
- Edema (swelling)rythema (redness): The treated area may exhibit erythema immediately following Treatment. This typically resolves within a few hours of treatment.
- Pain: Momentary discomfort may be experienced during the procedure while energy is being deposited. Post procedure discomfort or tenderness to the touch is also possible.
- Bruising: Mild bruising, which is caused by damage to soft tissue blood vessels, may occur occasionally and typically resolves within a few days of treatment.
- Nerve Effects:
 - Transient local muscle weakness may result after treatment due to inflammation of a motor nerve.
 - Transient numbness may result after treatment due to inflammation of a Sensory nerve.
 - Transient pain, paresthesia and/or tingling may be experienced No permanent injuries to facial nerves have been reported
- Scarring: The possibility for scar formation (which may respond to medical care) May exist if incorrect treatment technique is used

3. System Overview

3.1 System Description

Breast enhancement lymphatic drainage system can quickly improve size and shape of breast, without any aching and health threat, it does not involve silicon injection or pushing and filling bra, it would make you have natural and pretty breast according to physiological period.

3.2 System Components

3.2.1 Control Unit & accessories



A. Inhale

B. Function chose knob

C. Suction adjustment knob

4 Package Listing

main machine x 1
3 pairs of cups for breast
3 pairs of cups for cupping
6 Pairs gas cup for scrub
Metal Fingertip cups for rhythm massage x
3 Y type tubes x 2
Silicon tube x 1 Power
supply cord x 1

5. Installation

5.1 Install cotton filter



Fig 2.0


Install cotton filter

5.2 Connect the power cord

6. Treatment

6.1 Breast Lifting

Step 1. Massage breast with massage oil or cream for 2-3 minutes firstly

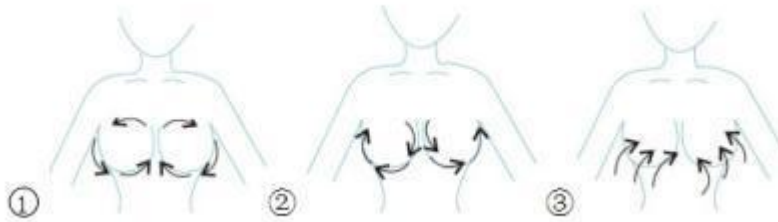

Step 2. Choose breast cup according to body condition of customer, connect two cups with Small Y type hose.

Step 3. Make customer half-sleep to stretch out breast, cover breast with plastic cloth, don't apply on people with sunken breast.

Step 4. Turn INTENSITY knob to adjust suction of cups and chose the BREAST LIFTING or TREATMENT mode

Step 5. Put both cups on breasts and make sure cup touch skin closely and no air leak. Turn INTENSITY button from left to right till the breast lifting, then adjust INTENSITY knob to get more suction until get the best comfortable feeling of customer.



Step 6. After 20 minutes, turn INTENSITY knob to left and the cups will loose, then apply lifting cream/oil on breasts and massage breasts for 2-3 minutes.

6.2 Butt Lifting

Step 1. Massage butt with massage oil or cream for 2-3 minutes firstly.

Step2. Choose butt cup according to body condition of customer,
connect two cups with Small Y type hose.

Step3. Make customer Lying on the bed with face down

Step4. Turn INTENSITY knob to adjust suction of cups, chose the
BREAST LIFTING or TREATMENT mode

Step5. Put cups on butts and make sure cup touch skin closely and no air leak.
Turn INTENSITY button from left to right till the butts lifting, then
adjust INTENSITY knob to get more suction until get the best
comfortable feeling of customer.

Step6. After 20 minutes, turn INTENSITY knob to left and the cups will loose,
then apply lifting cream/oil on butt and massage breasts for 2-3
minutes.

6.3 Rhythm Massage Treatment (Fingertip Moving)

6.3.1 For Shoulder, back, arm, waist, thigh, leg

Step1. Massage back with massage oil or cream for 2-3
minutes firstly.

Step2. Choose appropriate cup size, connect the cups with silicon
tube



- Step3. Turn INTENSITY knob to adjust suction of cup, chose BODY RHYTHM-1 mode
- Step4. Put ONE CUP on back and make sure cup touch skin closely and no air leak. Turn INTENSITY button from left to right till the skin lifting, then adjust INTENSITY knob to get more suction until get the best comfortable feeling of customer.
- Step5. Move cup slowly from neck to shoulder to back to waist to butt, back and forth for about 3-4 times, turn INTENSITY knob to left and the cup will loose, then apply lifting cream/oil on skin and massage arm for 2-3 minutes.



6.3.2 For face, chin, neck

- Step 1. Massage arm with massage oil or cream for 2-3 minutes firstly.
- Step 2. Choose appropriate cup size, connect the cup with silicon tube.
- Step3. Turn INTENSITY knob to adjust suction of cup, chose FACE RHTHM-1/FACE RHTHM-2 mode
- Step 4. Put a fingertip cup on arm and make sure cup touch skin closely and no air leak.turn INTENSITY button from left to right till the skin lifting, then adjust INTENSITY knob to get more suction until get the best comfortable feeling of customer.

Step 5. Move cup slowly from arm to underarm, back and forth for about 10-15minutes, turn INTENSITY knob to left and the cup will loose, then apply lifting cream/oil on skin and massage arm for 2-3 minutes.

6.4 Scrub and cupping Treatment

6.4.1 For Shoulder, back, arm, waist, thigh, leg

Step 1. Massage back with massage oil or cream for 2-3 minutes firstly.

Step 2. Choose appropriate cup size, connect the cups and big Y type hose

Step 3. Turn INTENSITY knob to adjust suction of cups, chose BODY RHYTHM-1 or BODY RHYTHM-1 mode

Step 4. Put both cups on back and make sure cup touch skin closely and no air leak. Turn INTENSITY button get the best comfortable feeling suction and then move cups on other treatment area after skin loose automatically, repeat operation until finish back treatment



Step 5. Apply lifting cream/oil on skin and massage back for 2-3 minutes after treatment

6.4.2 For Buttock Shape

- Step 1. Massage buttock with massage oil or cream for 2-3 minutes firstly.
- Step 2. Choose appropriate cup size, connect cup and suitable type hose
- Step 3. Turn INTENSITY knob to adjust suction of cups, chose body RHTHM-1/body RHTHM-2 mode
- Step 4. Pull up cup from the base to the thigh and pull from the sides of the hips to the hips, both for 3-5 times, and make sure cup touch skin closely and no air leak. Turn INTENSITY button get the best comfortable feeling suction. Then Select the right cup, secure the hips, and adjust the right amount of energy for 5-10 minutes.


6.4.3 For Abdomen

- Step 1. Massage Abdomen with massage oil or cream for 2-3 minutes firstly.
- Step 2. Choose appropriate cup size, connect cup and suitable type hose
- Step 3. Turn INTENSITY knob to adjust suction of cups, chose body RHTHM-1/body RHTHM-2 mode
- Step 4. Use cup push along six meridians from up to down to the groin for 3-5 times, and make sure cup touch skin closely and no air leak. Turn

INTENSITY button get the best comfortable feeling suction. Then Select the right cup use in the abdomen, such as acupuncture points as xiayuan, tianshu, daheng, guan yuan, waist side, suction and waiting for 5-10 minutes (depending on the customer).

6.4.4 For face, chin, neck

- Step 1. Massage face with massage oil or cream for 2-3 minutes firstly. Step 2. Choose appropriate cup size, connect both cups and big Y type hose
- Step 3. Turn INTENSITY knob to adjust suction of cups, chose FACE RHTHM-1/FACE RHTHM-2 mode
- Step 4. Put both cups on face and make sure cup touch skin closely and no air leak. Turn INTENSITY button get the best comfortable feeling suction and then move cups on other treatment area after skin loose automatically, repeat operation until finish back treatment

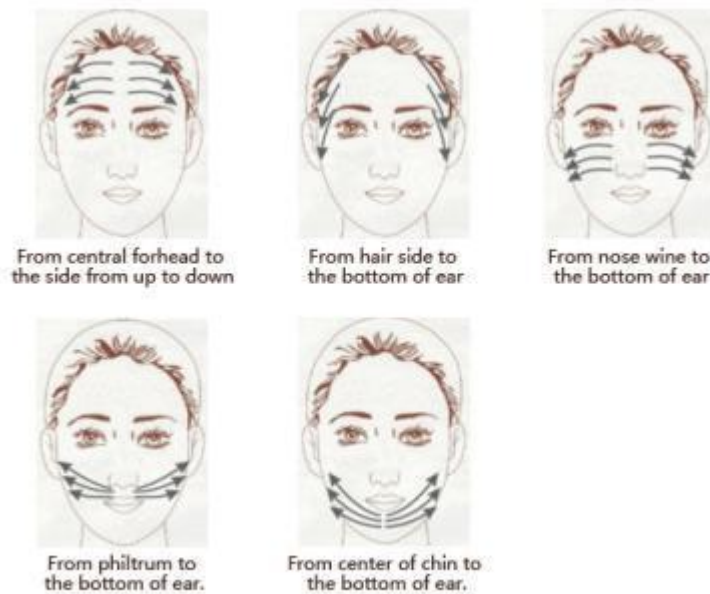

Step 5. Apply lifting cream/oil on skin after treatment

7. Recommend Treatment Time

- Scrub Treatment
4 treatments/course, one treatment/month

Note: shower/bath/swimming is forbid after treatment,12h interval will be okay

- Rhythm Massage Treatment
4 treatments/course, one treatment/week
- Breast & Butt Lifting
4 treatments/course, one treatment/week

8. Attention

1. Keep the machine level to avoid shaking and collision.
2. Turn off the power and pull out plug when washing the machine.
3. People with serious heart disease, diabetes mellitus and hepatitis are not suitable for treatment with the machine.
4. Women during menstruation and pregnancy are not suitable for treatment.
5. Scrub time should be controlled within 30 minutes with appropriate strength that customers feel comfortable.
6. Don't apply the machine half hour before or after meals, people with empty stomach, fatigue; insufficient sleeping and weak health are not suitable for treatment.
7. People with and blood productivity and spontaneous hemorrhage are not suitable for treatment, including hemophilia, purpuric disease and leukemia.
8. People with serious skin allergic symptoms or acariasis are not suitable for treatment.
9. People with malignant skin tumor or partial skin cancer, fracture, varicosity, partial skin elasticity loss
10. are not suitable for treatment. People in drunk, thirsty or hungry situation should pay special attention.
11. Wash filter regularly to ensure the machine effect and lifetime.
12. When pairing with essential oil, don't apply too much essential oil or cosmetics to avoid damage the machine.
13. After application of the machine, turn the air pressure to the lowest point.
14. Don't apply lymphatic detoxifying treatment when you are annoyed with lymphadenitis.
15. Patients with myocardial infarction, phlebitis depravation and intravenous tumor are not suitable for treatment.
16. Don't apply the treatment within three weeks after surgery for hyperthyroidism, pregnant women.
17. Patient with serious heart disease, diabetes mellitus and hepatitis are not suitable for scrub treatment.
18. Women during menstruation are not suitable for scrub treatment. Pregnant women can have light scrub treatment to promote blood circulation.
19. The scrub time shall not be too long, about half hour is ok with appropriate strength

that customers feel comfortable.

20. Don't apply the treatment half hour before or after meals, people with empty stomach, fatigue; insufficient sleeping and weak health are not suitable for scrub treatment.
21. After scrub treatment, have a glass of warm water after test for a while, and then have warm bath to enhance the effect.
22. You would feel a little bit aching one or two days after scrub treatment that is normal and would automatically disappear.