




**ENDOcube**

acc. to Pavel Krastev DDS, patented



PATENTED

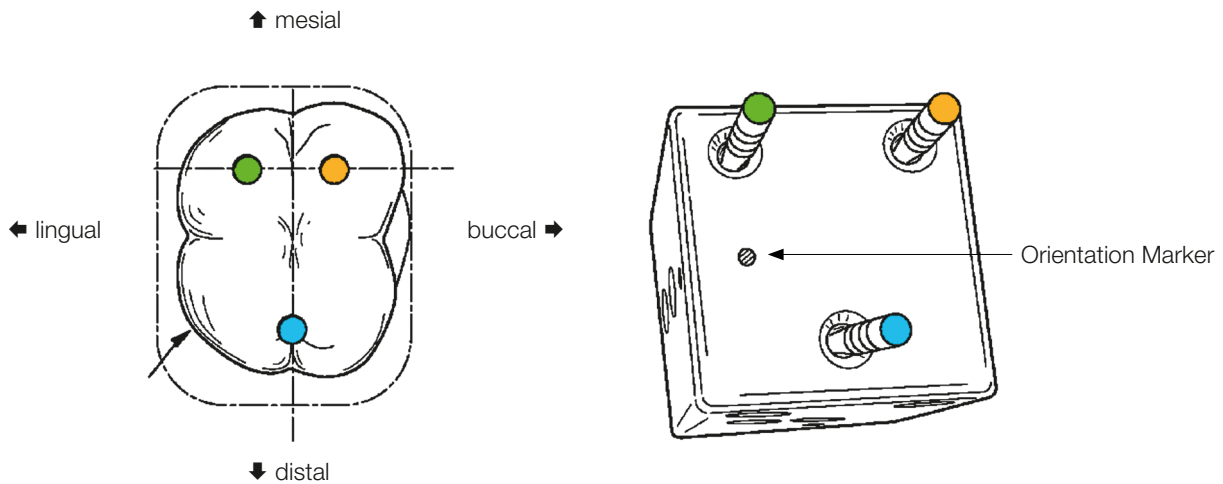



**ENDO CUBE**

acc. to Pavel Krastev DDS, patented

Never confuse your Master Gutta Percha Cones again during the obturation phase of root canal treatment.

The ENDO CUBE is designed to be used during the obturation phase of root canal treatment. It can also be used for measurement control since the ENDO CUBE features a ruler calibrated up to 37 mm.

ENDO CUBE features most commonly found canal configurations found in human teeth, to include the mid-mesial often found on lower molars.

ENDO CUBE is fully autoclavable for patient safety!



19.542.00

ENDO CUBE Master Cone Organizer, 43 x 43 x 43 mm, CERATEF coating



**Example of use**

**Step 1:** Following the instrumentation phase of root canal treatment, fit Master Gutta Percha Cones and verify with X-Ray.

**Step 2:** Depending on doctor position relative to which tooth is being endodontically treated, select what the Orientation Marker will represent to you.  
Example: B, L.

Note: In case of upper molars, the marker dot always indicates palatal!


**Step 3:** Transfer Gutta Percha Cones from tooth into ENDO CUBE exactly as they are positioned in tooth. Perform final canal irrigation and drying procedure.

**Step 4:** Coat each Master Cone with sealer and return each respective Gutta Percha Cone back into each respective canal.



Ruler 37 mm, calibrated

**Supplementary Instrument**

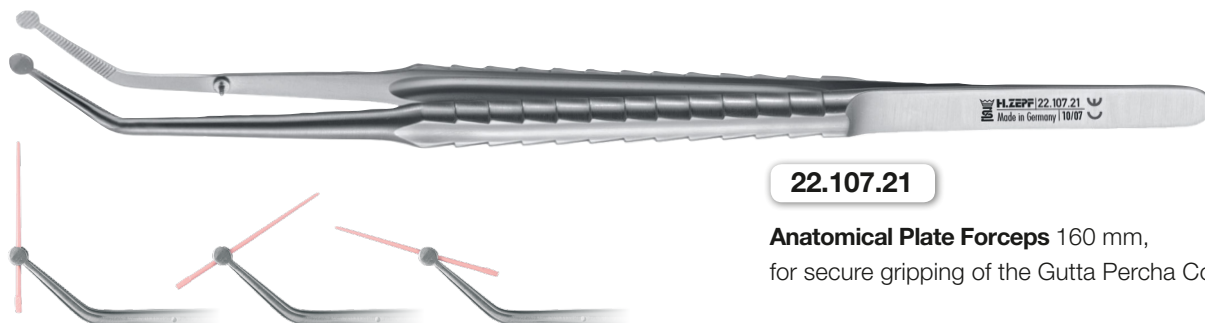

22.107.21

**Anatomical Plate Forceps** 160 mm, for secure gripping of the Gutta Percha Cones

AESTHETIC IS THE RESULT