

Corsinel®

Live as you like

StomaSafe Plus Ostomy Support Belt



Supports, stabilizes
and conceals ostomy
pouch

Reduces noise from
the pouch

Suitable for both 1-
and 2-piece systems

Seamless, microfiber
quality



Breathable



4-way stretch



Seamless



Latex-free

TYTEX®

How to apply

1

Pull on the StomaSafe – like underwear (the print should be located on the inside at the top).




2

Pull the ostomy pouch through the inner opening.



3

Place the bottom of the ostomy pouch in the small pocket in the bottom of the outer layer.

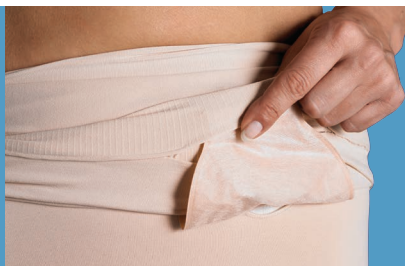

4

Fold down the outer layer.



The StomaSafe Plus Ostomy Support Belt conceals and supports the ostomy pouch, enabling the user to live an active life. It reduces content noise and can be used with both 1- and 2-piece systems.

Features:

- Provides support, stability and is a pouch cover - all-in-one unit
- Fixates the ostomy pouch, helping the user to feel secure
- Conceals and supports the ostomy pouch and enables the user to live an active life
- Reduces pouch and content noise
- The inner layer has a stoma pouch opening
- Can be used with both 1- and 2-piece systems
- Microfiber quality – soft to the skin
- Flexible fabric that is breathable, comfortable to wear and non-irritating to the skin
- No seams for sensitive skin
- Soft and stretchy material that is easy to pull on
- Latex-free material
- Tested for harmful substances according to Oeko-Tex standard 100


Indication:

StomaSafe Plus is specially designed for fixation and support of an ostomy pouch. Can be used for ileostomy, colostomy or urostomy. Recommended for daytime use, when more support is desired.

Precaution:

If too small or tight fitting, the product may affect the stoma drainage. If you have any questions regarding size, usage, indication, or if discomfort, or problems occur, please consult your doctor or your nurse.

Available colours:

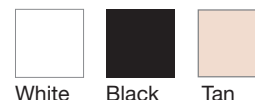

White Black Tan

Composition:


91% Polyamid
9% Elastane.

Care Handling:



Wash with similar colours and in a washing bag.

Measurement at the widest abdominal circumference

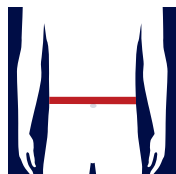

Recommended sizes

SIZE	CM	IN
S/M	80 - 110	34 - 44
M/L	105 - 125	42 - 50
L/XL	120 - 140	48 - 56

www.carefix.eu

CE TYTEX A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485



For further information please contact:

Dealer:

TYTEX®