



HOW TO USE THE OSTOFORM SEAL

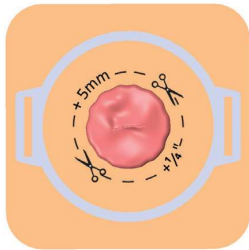


Step 1

Cut the pouch or wafer opening to stoma size + $\frac{1}{4}$ " (5mm)
e.g. for a 1" stoma, cut the pouch opening to 1 $\frac{1}{4}$ ".

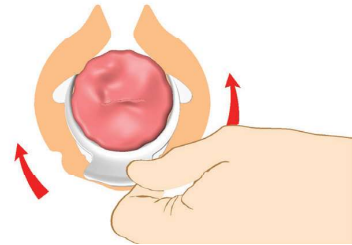


Step 2

Position the spout below the stoma
or rotate to align with the direction of output.

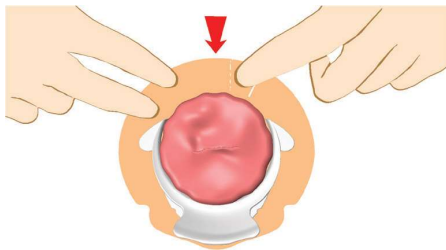


Step 3

Wrap around the stoma and overlap the tabs
to ensure a complete seal.

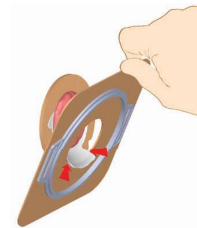


Step 4A

For secure 2-piece pouch attachment, position the baseplate
over the Seal first, ensuring that the spout passes through
the opening. Then, secure the pouch to the baseplate.

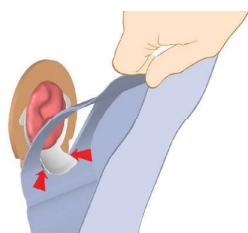

Step 4B

For secure 1-piece pouch attachment, ensure the spout is
fully inside the pouch opening. Then, pull the pouch up into
the slot underneath the spout.

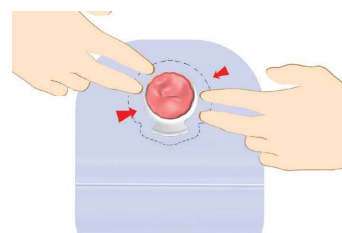

Step 5

Press the pouch adhesive against the Seal
and skin to achieve secure adhesion.