

carefix[®]

Creating functional comfort



LeggyFix

Leg bag holder



LeggyFix is specially designed to fixate a urine leg bag, without impeding appearance. The weight of a full leg bag is supported by the whole product. This way, the tension required to hold a bag in place is distributed evenly around the leg, rather than being concentrated in two narrow rings at the top and the bottom of the bag. This design not only reduces the risk of any constriction or pressure sores, but also prevents the bag from moving around and helps to dampen any undesired sounds, thus providing discretion.

Features:

- Easy to use
- Safe retention of leg bag for optimum support
- Tension is distributed evenly around the leg
- Provides discretion
- Helps to reduce unpleasant noises
- Reduces the risk of pressure sores
- Provides extra security during physical activity
- Very soft, stretchy and breathable material, that is comfortable to wear
- Fits left and right leg equally well
- Fits both thigh and calf
- With a slit at the bottom for urine bags with bottom tap opening
- Can hold up to 750 ml/25 oz (leg bags)
- Practical colour code for size indication
- Washable at 60°C/140°F
- Latex-free
- Oeko-Tex certified

Use:

For fixation of urinary leg bags. LeggyFix must be folded up, thereby providing a pocket for the urine bag. When LeggyFix is folded, the slit at the bottom will appear, and the valve of urine bags with bottom tap opening can be pulled through.

Composition:

63% Polyester, 37% Elastane

Washing instructions:



Recommended sizes:

Based on leg circumference.

SIZE RANGE	XS	S	M	L	XL
Leggyfix	CM: 25 - 33 IN: 10 - 13	CM: 33 - 40 IN: 13 - 15½	CM: 40 - 50 IN: 15½ - 19½	CM: 50 - 60 IN: 19½ - 23½	CM: 60 - 70 IN: 23½ - 27½

Colour code on the product indicates size

Packaging:




10 pcs. per bag


Instruction for use:



1 Pull the colour-coded end of the sleeve over the leg with seams on the outside.

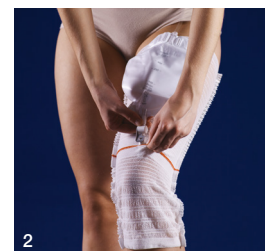

2 Pull the valve of the urine bag through the hole placed at the red line of the LeggyFix.

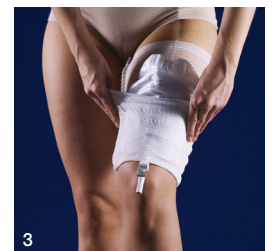

3 Pull up the lower part of the sleeve over the leg bag.



4 LeggyFix is in place.

For further information please contact:

Dealer: