

ACRYL-STAINS DESCRIPTION:

AcrylStains is an innovative new color system for permanent dental and ocular appliance staining. Great for creating a variety of effects on teeth, eyes and any other acrylic based projects. AcrylStains are very user friendly, as the wide variety of premixed colors make blending easier and more natural. The extended work time and durability lasts longer than any similar product, making AcrylStains the top painting system on the market today.

ACRYL-STAINS INSTRUCTIONS:

1. Use in a well-ventilated area and wear proper safety gear. Rubber gloves, and safety glasses. A 3M organic vapor respirator is a good thing as well if ventilation is an issue.
2. Never use Acryl-stains directly in the mouth, Acryl-stains are made for dentures and theatrical teeth. To stain actual teeth in someone's mouth use "PEGWORKS" Tooth stains from PPI as they are formulated to match Acryl-stains.
3. Acryl-stains are designed to be used in layers and or thin washes, if lifting of color occurs between coats allow more time between coats to dry.
4. If Acryl-stains start to thicken you can thin them down with the included thinner.
5. Acryl-stains come in a variety of colors but you can mix them to create any additional color you may need.
6. Work from you light colors to dark as its easier to add color rather than take away.
7. Multi glaze sealer can be used when you are finished painting, you can add 1 to 3 coats depending on the level of "gloss" you want to create the more layers the higher the shine.
8. When you are finished painting allow enough time to dry before putting teeth into the mouth and be sure to wash the appliance before putting into the mouth. Approximately ½-1hour should be

adequate a good rule of thumb is once the smell has dissipated the teeth are dry.

9. Be sure to securely close the Acryl-stain bottle to keep them from drying out when you are finished using them.

ACRYL-STAINS Q AND A:

Q, why use Acryl-stains?

A, acryl-stains are great for creating a variety of effects on teeth and any other acrylic based projects and come in a variety of premixed colors. Acryl-stains are also very user friendly in that they mix well together to make any number colors you may require, this and the extended work times make acryl-stains the top painting system on the market today.

Q, Can I paint acryl-stains directly in the mouth?

A, No, Acryl-stains are solvent based, and not for cosmetic use, Acryl-stains are not safe to use directly in the mouth. Once you have painted your dentures with Acryl-stains and they have had sufficient time to dry then they are safe for the mouth.

Q, what can I use in the mouth then?

A, PPI makes a series of cosmetic tooth stains called "PEGWORKS" that you can use in conjunction Acryl-stains that match in color.

Q, can I use acryl-stains with other dental stain products like minute stains?

A, Yes! Acryl-stains are compatible with most color systems available today.

Q Hey! My acryl-stains are getting thick.

A, Acryl-stains can begin to thicken over time use a few drops of the included thinner to extend the life of your Acryl-stain system as needed.