

Wheelchair

Front Caster Bearing

Removal and

Installation

Instructions



Tools You Need



Hammer



Allen Keys



Adjustable Wrench



Pin Punch
(Included in Front Caster
Bearing Took Kit)



Bearing Removal Tool
(Included in Front Caster
Bearing Took Kit)



Bearing Insert Tool
(Included in Front Caster
Bearing Took Kit)



Grease
(Included in Front Caster
Bearing Replacement Kit)



Lint Free an Cloth or
Acetone Wipes
(Included in Front Caster
Bearing Replacement Kit)

Instructions

1. Remove wheelchair
caster



2. Insert bearing
removal tool into
bearing and tighten



3. Using pin punch and
hammer, remove
bearing from other
side



4. Repeat steps 2 & 3
for each additional
bearing

5. Clean and lightly
grease new
bearings, axle and
spacer



6. Use a hammer and
bearing install tool
to install bearings
gently



We Want to Hear from you:

<http://www.wheelchairbearings.com>

If you have any question, comments or feedback we would love to hear it. In addition, if you would like any of our tools adapted or have ideas for other tools, please contact us, we would love to help.

**Your Source For wheelchair tools,
accessories and more!**

Proudly made in Texas 