

Wheelchair

Rear Wheel Bearing

Removal and

Installation

Instructions



Tools You Need



Hammer



Adjustable Wrench



Bearing Removal Tool
(Included in Rear Wheel
Bearing Tool Kit)



Grease
(Included in Rear Wheel
Bearing Replacement Kit)



Bearing Press Tool
(Included in Rear Wheel
Bearing Tool Kit)



Lint Free an Cloth or
Acetone Wipes
(Included in Rear Wheel
Bearing Replacement Kit)



Pin Punch
(Included in Rear Wheel
Bearing Tool Kit)

Instructions

1. Remove wheelchair rear wheel using quick release axle



2. Insert bearing removal tool into bearing and tighten



3. Using pin punch and hammer, remove bearing from other side



4. Repeat steps 2 & 3 for each additional bearing

5. Clean and lightly grease new bearings, axle and spacer



6. Use a hammer and bearing install tool to install bearings gently



We Want to Hear from you:

<http://www.wheelchairbearings.com>

If you have any question, comments or feedback we would love to hear it. In addition, if you would like any of our tools adapted or have ideas for other tools, please contact us, we would love to help.

**Your Source For wheelchair tools,
accessories and more!**

Proudly made in Texas 