PART OF MILLMEAD OPTICAL GROUP

## **KN95 FACE MASK INSTRUCTIONS**

## **PUTTING THE MASK ON**



Clean hands with soap and water or hand sanitiser before touching the mask.



Check mask for any defects. Throw away if defective. Remove mask from plastic wrapper.



Ensure mask exterior faces outwards.



Place mask on your face with the bendable edge on your nose.



Put one loop around each ear.



Mould or pinch edge to the shape of your nose.



Make sure the mask covers your nose and mouth so that the bottom edge is under your chin.



Clean hands with soap and water or hand sanitiser after touching the mask.

## **REMOVING THE MASK**



Clean hands with soap and water or hand sanitiser before touching the mask.



Carefully remove the mask using the ear loops, taking care not to touch the front of the mask.



Dispose of mask after use.



Clean hands with soap and water or hand sanitiser after touching the mask