


How to apply Cushion Grip®

Denture Adhesive

53331662

1

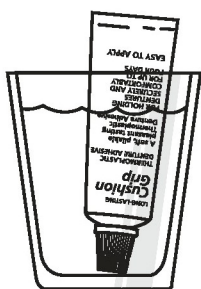

Warm the capped tube in hot tap water (not boiling) for 5 minutes.

This makes Cushion Grip easier to apply.



2

Use the key

Attach enclosed slotted key to end of warm Cushion Grip tube. Turn key carefully squeezing out a ball of Cushion Grip $\frac{3}{4}$ inch in diameter.

3

Apply to lower denture first



It's important to first seat your upper denture in your mouth without Cushion Grip on it. The lower plate must be thoroughly clean and dry. On lower plate start the Cushion Grip ball at one end of the gum

groove. Spread ball with fingers until entire groove is covered with a thin, even layer of Cushion Grip. Press layer firmly to plate. Let set for 5 minutes. Then put in lower denture and bite firmly.

4

...then apply to upper denture



Plate must be thoroughly dry and clean. With fingers, spread Cushion Grip ball from one end of gum groove to the other in a thin even layer. Be sure to cover

entire back edge of upper plate with a thin layer of Cushion Grip as shown. Press layer firmly to plate. Wait 5 minutes. When inserting upper denture, press against front gum. Then, with thumb push up firmly to roof of mouth. Complete the seal by biting down firmly.