

How to remove **Cushion Grip**[®]

Denture Adhesive

Because Cushion Grip was made to be longer lasting than pastes or powders, it doesn't wash or rinse off plates with water. Before making each new application, the old one should be removed. It will be easier to remove if you follow these directions.

1 Soak first



Soak plates in hot (not boiling) water for 10 minutes to soften the old Cushion Grip application.



2 Peel away

Peel away the warm Cushion Grip from dentures with fingers or a wood manicure stick. A new application gives better results on a clean denture surface.

Why One Application of Cushion Grip Denture Adhesive Holds Dentures Securely—Comfortably Up to 4 Days

Cushion Grip, a unique pliable, thermoplastic denture adhesive is different from anything you may ever have used. One application creates a secure comfort seal that lasts up to 4 days. Economical because it eliminates need to frequently apply powders, pastes and creams.

Cushion Grip is not dissolved by hot or cold liquids. It stays on despite frequent cleansing and soaking. And it recreates a secure comfort seal each time you reinsert your dentures. That's why it's applied in a different, more modern way.

IMPORTANT: Ill-fitting, broken or irritating dentures may impair your health. Consult your dentist at least every six months for a periodic examination.

Helpful Hints and Information

- Replace Cushion Grip on your dentures at least once a week.
- Daily use of a dental cleaner should not loosen Cushion Grip.
- If you don't get enough Cushion Grip on your plates the first try, dry plate and apply more over the first layer.