

# NANO DROPPER COMPATIBILITY LIST

**Please note this is NOT a comprehensive list**

If you do not see your exact bottle on this list please use the **Compatibility Guide** on pg. 5 to test your bottle.

If you can confirm compatibility or incompatibility of any bottles not on this list, please send a picture of the bottle (or email the drug name, concentration, and manufacturer) to [support@nanodropper.com](mailto:support@nanodropper.com).

*Alphabetized by brand name*

## COMPATIBLE with Nanodropper (brand names A-L)

Brand name	Generic name	Manufacturer
Alaway	Ketotifen fumarate	Bausch + Lomb
Alphagan P	Brimonidine tartrate	Allergan
Artificial Tears Solution	Polyvinyl alcohol	Rugby
Azopt	Brinzolamide	Novartis/Alcon
Bepreve	Bepotastine besilate	Bausch + Lomb
Betimol	Timolol maleate	Akorn
Brimonidine tartrate	Brimonidine tartrate	Bausch + Lomb
		Sandoz
Combigan	Brimonidine tartrate/Timolol maleate	Allergan
Cosopt	Dorzolamide HCl/Timolol maleate	Akorn
Cyclomydril	Cyclopentolate HCl/Phenylephrine HCl	Alcon
Cyclopentolate HCl	Cyclopentolate HCl	Akorn
		Sandoz
Dorzolamide HCl	Dorzolamide HCl	Valeant/Bausch
Ketorolac tromethamine	Ketorolac tromethamine	Micro Labs Ltd.
Latanoprost	Latanoprost	Bausch + Lomb
		Sandoz
Lotemax SM	Loteprednol etabonate	Bausch + Lomb
Lotemax Gel	Loteprednol etabonate	Bausch + Lomb
Lotemax Suspension	Loteprednol etabonate	Bausch + Lomb
Loteprednol Etabonate	Loteprednol etabonate	Oceanside Pharmaceuticals
Lumify	Brimonidine tartrate	Bausch + Lomb
Lumigan	Bimatoprost	Allergan

## COMPATIBLE with Nanodropper (brand names M-R)

Brand name	Generic name	Manufacturer
Muro 128	Sodium chloride	Bausch + Lomb
Mydriacyl	Tropicamide	Alcon
Naphcon A	Naphazoline HCl/Pheniramine maleate	Alcon
Neomycin/Polymyxin B/Dexamethasone	Neomycin/Polymyxin B/Dexamethasone	Bausch + Lomb
Olopatadine HCl	Olopatadine HCl	Apotex
Opti-Free PureMoist Rewetting Drops		Alcon/Novartis
Paremyd	Hydroxyamphetamine hydrobromide/Tropicamide	Akorn
Phenylephrine	Phenylephrine	Akorn
		Paragon
Pred Forte	Prednisolone acetate	Allergan
Pred-Moxi-Brom	Prednisolone/Moxifloxacin/Bromfenac	Imprimis Rx
Pred-Moxi-Nepaf	Prednisolone/Moxifloxacin/Nepafenac	Imprimis Rx
Prednisolone acetate	Prednisolone acetate	Bausch + Lomb
		Pacific Pharma
		Sandoz
Prednisolone acetate PF	Prednisolone acetate PF	Imprimis Rx
Prolensa	Bromfenac	Bausch + Lomb
Proparacaine HCl	Proparacaine HCl	Akorn
		Alcon
		Bausch + Lomb
		Sandoz
Refresh Digital		Allergan
Refresh Plus		Allergan
Refresh Optive Gel Drops		Allergan
Refresh Relieva		Allergan
Refresh Tears		Allergan
Rhopressa	Netarsudil	Aerie Pharmaceuticals
Rocklatan	Netarsudil/Latanoprost	Aerie Pharmaceuticals

## COMPATIBLE with Nanodropper (brand names S-Z)

Brand name	Generic name	Manufacturer
Simbrinza	Brinzolamide/Brimonidine	Novartis/Alcon
Sodium Chloride Hypertonicity Solution	Sodium chloride	Rugby
Systane		Alcon
Systane Complete		Alcon
Systane Ultra		Alcon
Systane Zaditor	Ketotifen fumarate	Alcon
Timolol maleate	Timolol maleate	Akorn
		Sandoz
Tropicamide	Tropicamide	Akorn
		Bausch + Lomb
		Sandoz
Vuity	Pilocarpine HCl	Allergan
Vyzulta	Latanoprostene bunod	Bausch + Lomb

## NOT COMPATIBLE with Nanodropper (brand names A-C)

Brand name	Generic name	Manufacturer
Acuvail	Ketorolac tromethamine PF	Allergan
Auroprost	Latanoprost	Thermodoors
Azarga	Brinzolamide/Timolol	Alcon
Bimatoprost	Bimatoprost	Alembic
		Somerset
Blink Gel Tears Lubricating Eye Drops		Johnson & Johnson
Brimonidine tartrate	Brimonidine tartrate	Somerset
		Akorn
BromSite	Bromfenac	Sun Pharma
Can-C		Can-C
Cosopt PF	Dorzolamide HCl/Timolol maleate PF	Akorn
Cyclopentolate HCl/Tropicamide/Phenylephrine HCl	Cyclopentolate HCl/Tropicamide/Phenylephrine HCl	Leiters

## NOT COMPATIBLE with Nanodropper (brand names D-Z)

Brand name	Generic name	Manufacturer
DORTIMOPT	Dorzolamide HCl/Timolol maleate	Mylan (NZ)
Dorzolamide HCl	Dorzolamide HCl	Akorn
		Micro Labs Ltd.
Dorzolamide HCl PF	Dorzolamide PF	Imprimis Rx
Dorzolamide HCl/Timolol maleate	Dorzolamide HCl/Timolol maleate	Akorn
		GSMS
		Hi-Tech
		Somerset
Dorzolamide HCl/Timolol maleate PF	Dorzolamide HCl/Timolol maleate PF	Aurobindo Pharma
Fluress	Benoxinate/Fluorescein	Akorn
Latanoprost	Latanoprost	Greenstone
		Somerset
Natacyn	Natamycin	Alcon/Novartis
Pataday	Olopatadine HCl	Alcon
Pilocarpine HCl	Pilocarpine HCl	Somerset
Refresh Plus PF		Allergan
Regener-Eyes Lite		Regener-Eyes
Regener-Eyes Professional Strength		Regener-Eyes
Restasis	Cyclosporine	Allergan
Restasis Multidose	Cyclosporine	Allergan
TheraTears Dry Eye Therapy		Thera Tears
Timolol maleate	Timolol maleate	Pacific/Greenstone
		Rising Pharm
		Sandoz
TPROST	Travoprost	Gerresheimer
Travatan Z	Travoprost	Alcon
Travoprost	Travoprost	Par Pharmaceutical
		Sandoz
Tropicamide	Tropicamide	Somerset
Tropi-Phen	Tropicamide/Phenylephrine HCl	Pine Pharmaceuticals
Tropicamide/Phenylephrine HCl	Tropicamide/Phenylephrine HCl	Leiters
Xalatan	Latanoprost	Pharmacia/Pfizer
Xelpros	Latanoprost	Sun Pharma
Xiidra	Lifitegrast	Novartis
Zirgan	Ganciclovir	Valeant/Bausch

# NANODROPPER COMPATIBILITY GUIDE

Follow these instructions to make sure your bottles are compatible with Nanodropper.

## Option 1: Print (recommended)

Note: If you cannot print, see Option 2 on pg. 6

1. Download and print this page on 8.5"x11" paper, no added margins  
(Note: only print pg. 5 of this document to save paper)
2. Cut or fold the blue line at the bottom of this page
3. Carefully cut along the black dotted lines of the sizing cut-outs
4. Take the cap off your bottle and place in sizing cut-out  
**(SEE EXAMPLE ON PG. 6)**

### !! Cap must be off

- ✓ If your bottle (including the thread grooves) fits exactly in the left cut-out, you are compatible!
- ✗ If your bottle is smaller than the left cut-out and/or fits within the right cut-out, you are not compatible with our current Nanodropper
- ? Send any compatibility questions to [support@nanodropper.com](mailto:support@nanodropper.com)

  
**COMPATIBLE**



**IMPERIAL SIZE:**  
9/16 in (~15 mm)



**NOT COMPATIBLE**



There should  
be no gap



**THIS SIZE  
COMING SOON**



**METRIC SIZE:**  
13 or 14 mm



We are working hard to make the **METRIC** size adaptor to fit this bottle!


**CUT OR FOLD  
THIS LINE**



# NANODROPPER COMPATIBILITY GUIDE

If you would like assistance determining compatibility, please take a picture like the example below and send to [support@nanodropper.com](mailto:support@nanodropper.com)

## EXAMPLE OF HOW TO USE COMPATIBILITY GUIDE

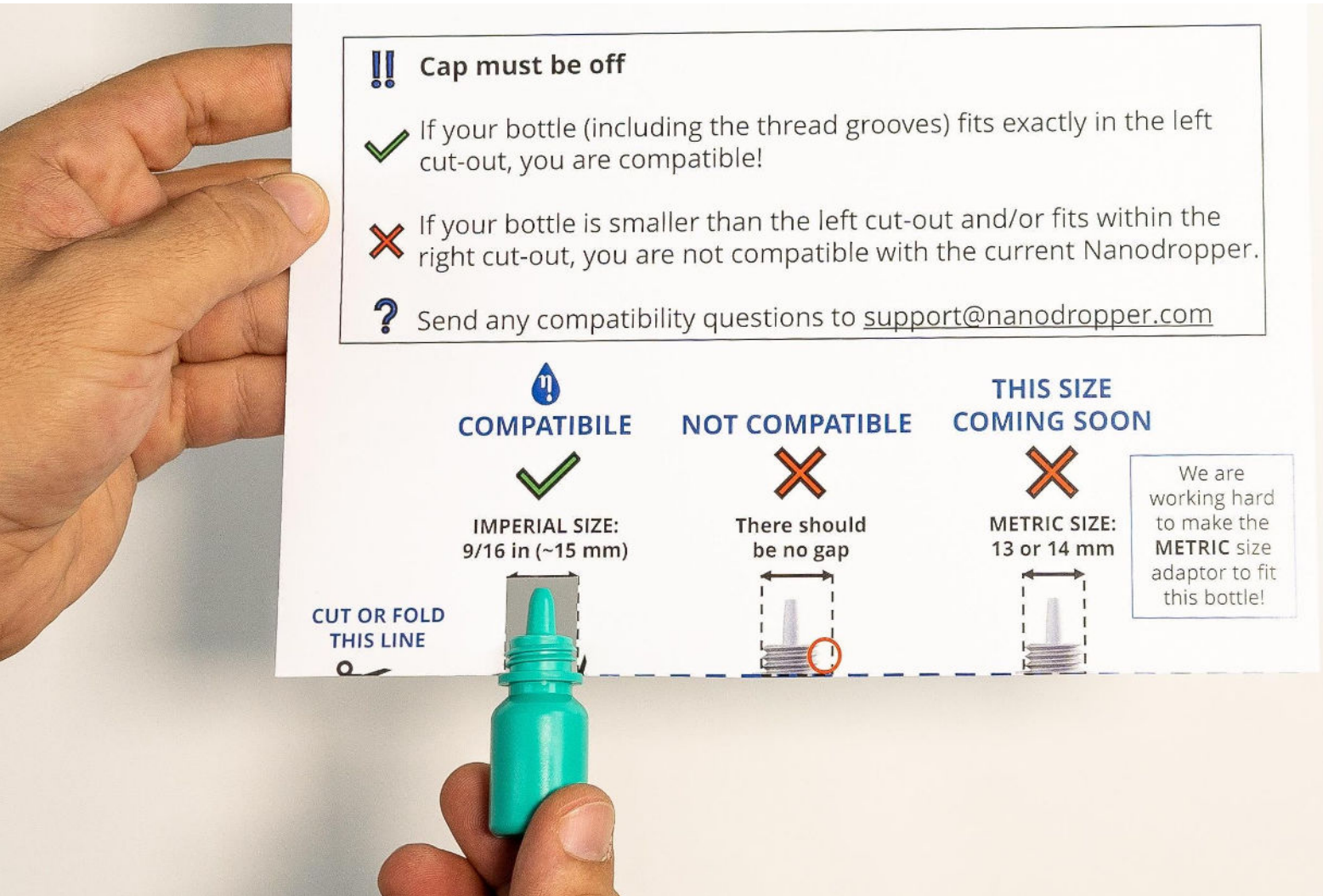

### Option 2: Use phone or computer screen (only if you cannot print)

1. Zoom in until the scale bar below is exactly 1 inch (hold a ruler to your screen)



2. Hold your bottle up to the screen to use compatibility guides on pg. 5