



# Instructions for Use:

## Greeny<sup>®</sup> Compostable Underpad

In this guide, you'll discover how to use the Greeny Compostable Underpad, developed as the compostable, breathable alternative to traditional, single use underpads

PUBLISHED APRIL 2024






## Greeny® Compostable Underpad

- Introduction
- Instructions for Use
- Life Cycle of Compostable Products

The Greeny has been developed to help alleviate the burden of traditional, single use underpads on our environment.

While we recognise that it will take much more than this to achieve a sustainable healthcare system, we figure a highly absorbent, certified compostable alternative is a great place to start.

Made of cornstarch-based bioplastic with a 5-ply paper fluff pulp top layer sourced from responsibly managed forests, Greenys are 100% compostable and compliant with AS 4736 compatibility standards.<sup>1,2</sup>

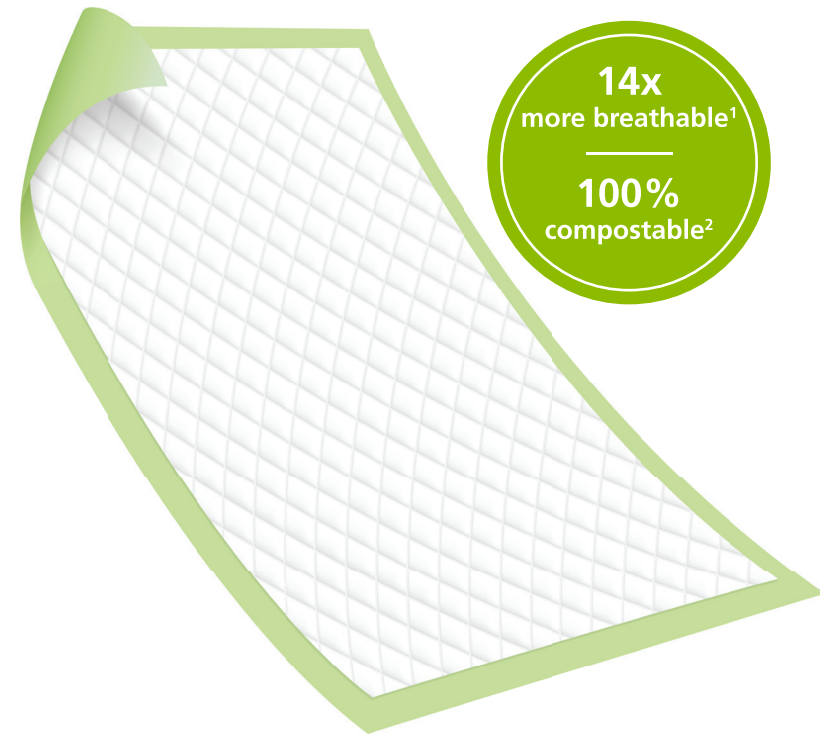


**PLEASE NOTE: Greeny is not a patient handling device and should not be used to move or reposition a patient. Always follow these directions for use.**


## INSTRUCTIONS FOR USE



1 Perform hand hygiene and apply necessary Personal Protective Equipment (PPE) as per facility guidelines.

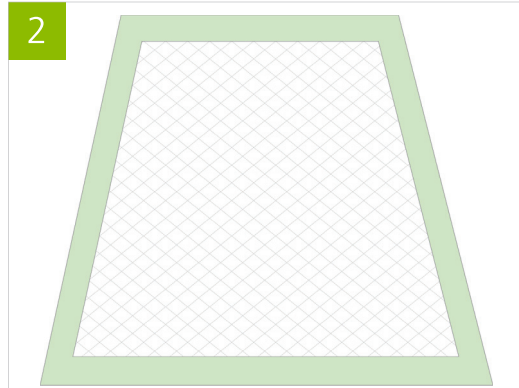


2 Unfold a new Greeny Compostable Underpad.

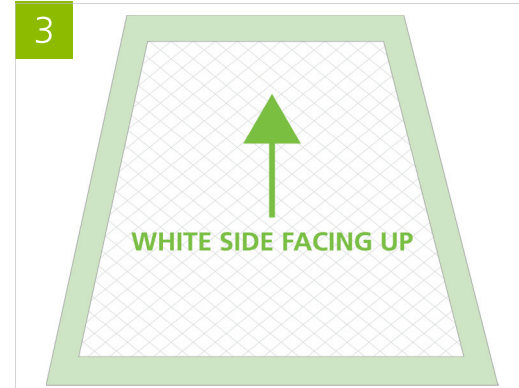

3 Place the pad in a position with the white side facing up and green side down, against the surface to be protected.



4 If inserting under an immobile patient, refer to facility patient handling guidelines for suitable insertion procedure, such as the 'log rolling' technique commonly used to change patient linen.

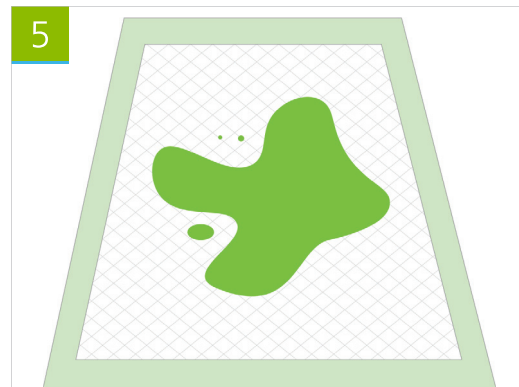

5 Replace if soiled as per facility guidelines.

Instructions for Use:  
Greeny®  
Compostable  
Underpad

Instructions for Use

## HOW TO DISPOSE OF A USED GREENY



Unsoiled



Soiled or  
unsoiled




If other  
streams  
unavailable

For clean, green disposal, discard via **green waste** or **incineration** waste streams.

For further information about Greeny disposal, please refer to the Greeny Disposal Guide and Wall Poster.



## LIFE CYCLE OF COMPOSTABLE PRODUCTS

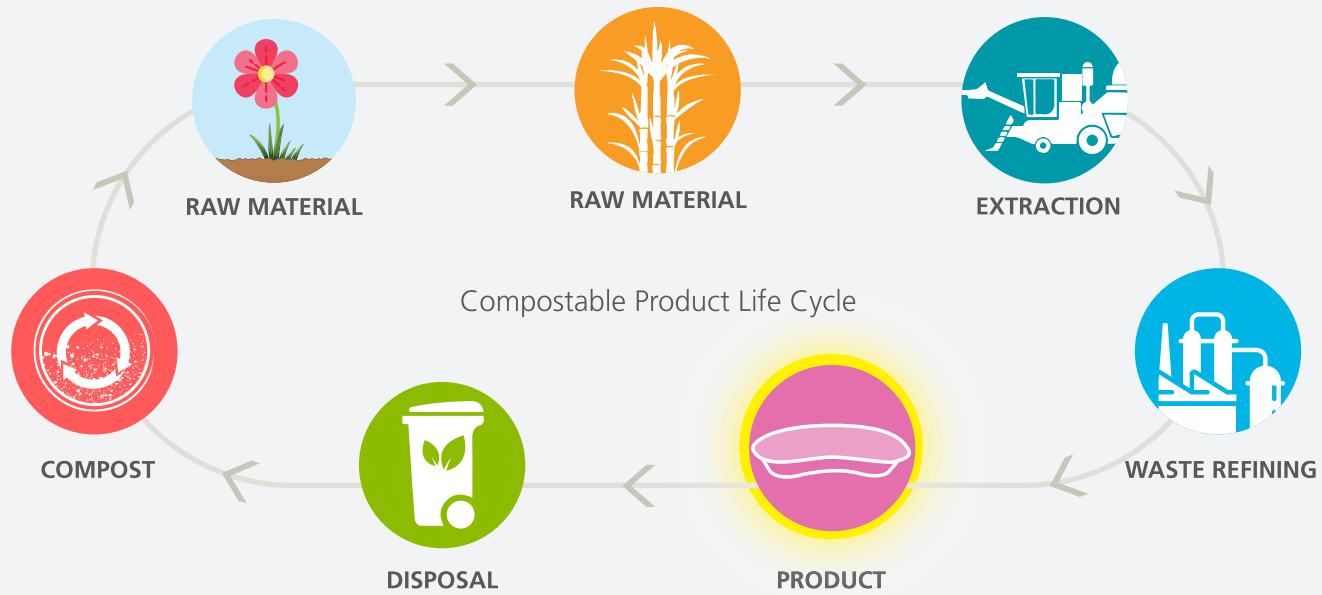

Instructions for Use:  
Greeny®  
Compostable  
Underpad

**Life Cycle of  
Compostable  
Products**

1. Test data and/or certification on file.
2. Intertek AS 4736 Compostability Report. Completed December 16, 2022.

**Haines**<sup>®</sup>  
MEDICAL AUSTRALIA

*Greeny*

The compostable, breathable alternative to  
traditional, single use underpads





Haines Medical Australia  
T (08) 8294 5999  
[www.hainesmedical.com.au](http://www.hainesmedical.com.au)