

humanⁿ | nitric oxide indicator strips

Nitric Oxide Indicator Strips are a simple way to see if you are getting enough essential dietary nitrate from your diet, or are Nitric Oxide deficient.

Non-invasive, the results are instant and easy to read. Which means evaluating N-O status or monitoring progress is a snap. Each tube contains 25 Indicator Strips.



1 MINUTE N-O INDICATOR TESTING OVERVIEW Scan QR Code To Watch Now



USING THE NITRIC OXIDE INDICATOR STRIPS

Place a drop of saliva on the indicator strip first thing in the morning, before eating and drinking anything. Then compare the indicator strip with the accompanying color chart on our N-O Indicator Strip packaging.

Results are practically instantaneous, and you can tell in a glance, if you're getting sufficient Nitric Oxide-activating nutrients through your diet. If you're not, adding Neo40 Professional, the most concentrated N-O solution, to your daily routine can help boost Nitric Oxide production.

NITRIC OXIDE RESTORATION SUGGESTED

IDEAL ONGOING LEVEL

DEPLETED

LOW

OPTIMAL

This color indicates a spike of nitric oxide may occur immediately following consumption of nitric oxide-producing products or foods.



Since 2015, **Neo40 Professional** has been a leading *physician-recommended* Nitric Oxide boosting product.