MAGNET RELAX™ MASSAGE MASK

USER MANUAL

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MAGNET RELAX™ MASSAGE MASK

USER MANUAL

BETTER THAN A MASSAGE,
A UNIQUE EXPERIENCE

magnetinnov
MEDICAL INNOVATION

TABLE OF CONTENTS

FR

EN

Product Specifications.....	13
Preparation.....	14
Feature configuration.....	15
Usage Manual.....	16
Precautions and Warnings.....	17-19
Troubleshooting and technical parameters.....	20
Transport, storage, and maintenance.....	21
Warning, contact us.....	22

Product Specifications

FR
EN

- This product is a high-technology multifunctional device that provides relaxing massages in the area around the eye and temples.
- The Magnet-Innov mask adopts an infrared heat generator at the cutting edge of technology that activates cells around the eyes, stimulates cellular molecules through the use of vibrations, reinforces molecular combinations, and improves tissue regeneration capabilities.
- The product uses gentle air pressure to protect and maintain eye health
- The air pressure gives your eyes a total massage. This process can reduce eye fatigue, improve ciliary muscle function, and prevent optical axis deterioration.
- The Magnet-Innov mask is also equipped with an integrated three-dimensional magnetic field. The constant movement of the magnetic field ensures an effective massage of the face's main parts and muscles around the eyes, by generating micro-flow in the blood vessels and activating blood circulation around the eyes.


Preparation

FR

EN


Battery Installation

1. Open the battery compartment lid in the back of the remote.
2. Insert 4 AA batteries according to the directions in the compartment.
3. Close the battery jack compartment lid.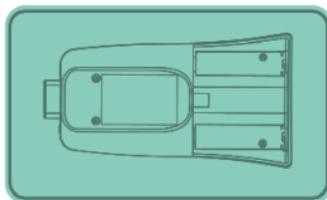

Mask Installation


1. Connect the adaptor provided into the DC 6V outlet.
2. Then plug in the cable into an outlet.

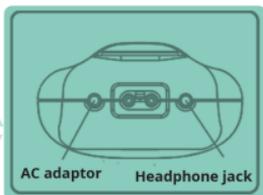Plug in headphones if you want.

Position the mask opposite your eyes.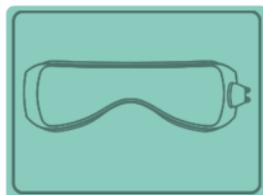


Position the mask opposite your eyes.

Magnetinnov
MEDICAL INNOVATION


Feature Configurations

FR
EN


Magnet-innov® eye massage mask features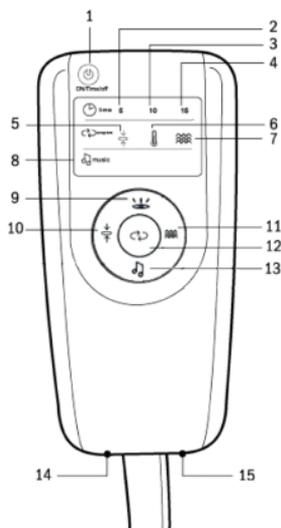

1. Start/Stop timer
2. 5 minute timer
3. 10 minute timer
4. 15 minute timer
5. Air pressure mode light
6. Infrared mode light
7. Multi-frequency vibration mode light
8. Music light
9. Infrared mode selection button
10. Air pressure mode selection button
11. Multi-frequency vibration mode selection button
12. Massage mode selection button
13. Music turn on/off
14. Power supply adapter jack
15. Headphone Jack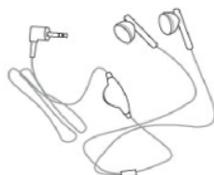

Headphones

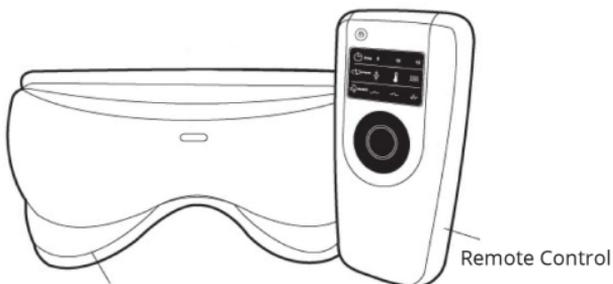
Fabric cover

Remote Control

AC Adapter

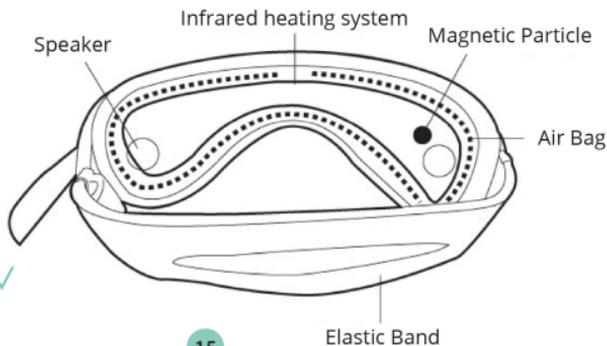Speaker

Infrared heating system

Magnetic Particle

Air Bag

Elastic Band

On/off/time button

Air Infra Temps Music Vibration

On/off/time button

Press the "↻" to choose between three automatic massage modes.

Attention:

The first automatic massage mode will be automatically activated when the device is turned on which consists of:

- + Air compression level 1 mode
- + Low temperature infrared mode
- + 15 min time
- + Relaxing music

-On/off button

Press once to turn on/press for a long period of time to turn off

-Change the time

Press the on/off/time button once

Music selection

Relaxing music

Background music

Nature sounds

Press briefly to change music

press for a longer period of time to stop the music

Air compression mode

Circular (the symbol will flicker once)

Power level 1

Power level 2

Power level 3

Infrared mode

Low temperature

High temperature

Choose duration

5min

10min

15min

Choose period

Vibrates twice

Vibrates ten times

Precautions

FR

EN

- Please read the instructions of this manual carefully in order to ensure correct and safe use of the product.
- Do not use the mask as a substitute for an eye operation or to relieve eye injuries, glaucoma, cataracts, retinal detachment, and other serious eye diseases, etc.
- Those with high and low blood pressure should consult their doctor before using the product.
- People with mental or intellectual disabilities should not use the mask.
- Do not use diluents, gasoline, chlorine, or alkaline or acidic cleaning products.
- Stop using the mask if you suffer from an eye disease.
- To clean the device, wipe it down with a cotton ball soaked in rubbing alcohol. Do not submerge or wash the device.
- Do not rebuild, disassemble, or repair the device.

Attention

- Put on the device before turning it on in order to avoid damaging the inflatable cushion.
- Do not wear contact lenses during product use.
- Do not use the product in an environment with high temperatures or an environment that is too humid.
- Do not use this product while driving or walking.

- Children under 12 must have adult supervision while using the device.
- We recommend that children do not use the mask for more than 20 minutes.
- Remove batteries while storing the device in order to avoid any leaks.
- Keep your eyes closed while you use the mask. After the massage, gently massage the eye region to remove red marks caused by the massage device.
- Do not throw the device on the ground or submerge it in water.

Warning

This device can be used by children starting from the age of 8 years old and by people with reduced physical, sensorial or mental abilities, solely under supervision and by following the instructions on the usage of the device, under the condition of a clear awareness of the following warnings.

Read the user manual very carefully prior to using the product. The company is not responsible for the consequences of an improper usage of the device, or from a non-respect of the instructions outlined in this user manual.

1. Do not use while driving.
2. Do not use in environments where the temperature or humidity levels are excessively high.
3. To be used carefully by people who manifest disease symptoms.
4. Do not use the device on the areas of the body that have been recently operated or have had surgery performed on them on previous occasions.
5. Place the device on your eyes at first and then switch it on, in order to avoid damage to the inflatable bag due to overpressure.

- 6.** The usage of the mask is not recommended for individuals that have suffered from a stroke, heart disease, hypertension or hypoglycemia.
- 7.** Do not consume any alcohol prior to using the mask.
- 8.** Remove contact lenses, hair ties and any other accessories from the head and the face in order to avoid any risk of injury.
- 9.** Use the electrical adaptor which conforms to the norms of your country and pay attention to electrical safety (5V/1A, not provided).
- 10.** The product is a home health device. It cannot act as a replacement to a medical treatment by any means.
- 11.** You must close your eyes during the massage.
- 12.** If you are feeling unwell during the usage, stop the session immediately and consult a healthcare professional.
- 13.** Do not use the device if you had surgery performed on your eyes or if you are suffering from any eye diseases. Do not use if your eyes are swollen.
- 14.** The device is equipped with a hot compress function during which the temperature can get reach 42°C. People who are sensible to temperature variations need to be particularly cautious when using the device.
- 15.** If the massage area has got any wounds, redness, inflammation of other physical abnormalities, you must stop the usage of the device immediately.
- 16.** We recommend a daily 10-minute session. The mask would switch off automatically after 10 minutes.
- 17.** Minors, handicapped people and people of third age must use this product under constant supervision.
- 18.** For cleaning, dry the mask with a soft cloth with alcohol and a cleaning product. Do not wet the device.

Troubleshooting and technical problems

FR

In case of problems during use, follow the procedure below.

EN

Read the instructions below to find the problem or determine if the device is defective.

If the device doesn't work properly, cut off the power supply and contact customer service.

Problem	Possible cause of the problem	Solution
Device does not turn on	Batteries are unloaded	Change the batteries.
	Batteries are improperly placed	Check if the batteries are in the right direction.
		Check if the battery is properly placed in its compartment.
	Improperly plugged in cable	Make sure that the cable is plugged in properly.
Damaged cable	Check the cable connections. If the cables are damaged, contact customer service.	
The light is too dim.	Batteries are unloaded	Change the batteries.
The air pressure is too low.	Batteries are unloaded	Change the batteries.
		Make sure the batteries are installed and fastened in their holders.

Technical parameters :

Ideal temperature : 5°C-40°C

Relative humidity: <80%

Atmospheric pressure : 860hpa-1060hpa

Magnetic field strength: 50mT-250mT

Fomentation : 30°C-42°C

Input power : DC6V 1A DC6V 1A

Power : 6W

Net Weight: 485g

Transport, storage, and maintenance

FR

Power supply verification

Check the power supply and turn on the device if the voltage exceeds the nominal range of the device.

Maintenance

If the device must be cleaned, use a cotton ball with rubbing alcohol. Put the device in a safe place after use. Do not put heavy objects on the device. Turn the device off before cleaning. Avoid humidity.

Contact a qualified technician if there is a problem. Do not dismantle the device yourself.

Transport and storage

Ideal temperature : 5°C-40°C

Relative humidity : --.80% Keep in a well-ventilated area away from direct sunlight and corrosive gases.

Atmospheric pressure : 860hpa-1060hpa

Transport : The product has a simple shock-absorbing design and can be transported by air, train, truck/car, or boat. Avoid snow, rain, and shocks, and always keep the top side facing upwards.

EN

Avertissement:

This symbol on a device or its packaging means: do not throw away electronic devices in unapproved municipal landfills: use separate collection facilities. Contact your local authority to obtain information on available collection systems in your area. If electronic devices are thrown away in landfills, dangerous substances can seep into underground water and end up in the food chain which is dangerous to your health and wellbeing.

Contact Us

If you have any questions, feel free to contact us, and we'll do the best we can to address your questions or concerns.

contact@magnet-innov.com

Magnet-Innov

[magnet_innov](#)

magnetinnov
MEDICAL INNOVATION

Thank you for purchasing the Magnet Relax™ massage mask

Discover now our best products

Magnet-Perfect™
The Smart
Cervical Massager

Magnet-Zen™
The Shiatsu
Massage Pillow

Magnet-Relax 2.0
The Smart Eye
Massager

 Magnet-Innov

 magnet_innov

magnetinnov
MEDICAL INNOVATION
www.magnet-innov.com

 Scan me
for a gift