



**PROTEKT[®] DIGITAL
PATIENT LIFT SCALE**

MODEL: 30300

**For use with Patient Lift Models:
33500/33600/33400P**



OWNER'S MANUAL

**270 Washington Street
Mount Vernon, NY 10553
855-237-7622
www.proactivemedical.com**

Thank you for choosing the Proakt® Digital Lift Scale!

To better serve you, please record the following information:

Supplier Name: _____

Telephone: _____

Serial #: _____

Date of Purchase : _____

INSPECT YOUR MERCHANDISE

Upon receipt of your patient lift scale, verify that all merchandise is complete and free from any shipping damage. Refuse delivery if the packaging appears to be badly damaged. If the merchandise is received damaged or is missing components, contact the shipping company immediately and file a claim.

For further assistance, contact your local dealer or Proactive Medical Products at the following:

Proactive Medical Products
270 Washington Street
Mount Vernon, NY 10553
Tel: 855-237-7622
www.proactivemedical.com

INTRODUCTION

Important Safety Information

- ⚠ WARNING: Important! Read and understand these instructions before assembling or using the Protekt® Digital Lift Scale. If you do not understand any part of these warnings or instructions, contact a healthcare professional for direction in the use of this product. If the Digital Lift Scale is not properly assembled, personal injury and damage to the Lift Scale could result.**
- ⚠ WARNING: DO NOT use this product without proper instruction from a healthcare professional.**
- ⚠ WARNING: If components are damaged or missing, contact your Proactive Medical distributor immediately. DO NOT use substitute parts. Use only Proactive Medical replacement parts. Non-Proactive replacement parts could cause personal injury and damage to the Lift Scale.**

⚠ WARNING: This product has a maximum weight capacity of 600 lb (300 kg). If patient weight exceeds maximum weight capacity, the scale will designate overload by displaying “oL-”. DO NOT exceed the maximum weight capacity of the lift being used for measurement.


⚠ WARNING: California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

⚠ WARNING: Proactive Medical Products assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

Intended Use








The intended use of the **Protekt® Digital Lift Scale** is the measurement of patient weight.

SCALE INSTALLATION ON PATIENT LIFT


1. Use the following instructions to attach the Lift Scale to the appropriate Patient Lift with the included mounting hardware. Do not use any other mounting hardware than that provided.
2. Keep the scale in a cool, dry place. If operation temperature is over 104°F (40°C), the tolerance will not be accurate within 0.2 lb (0.1 kg).
Recommended storage temperature is 32°F ~158°F (0°C ~ 70°C).
3. When the low battery indicator () appears in the lower right corner of the display, replace the battery (see **MAINTENANCE** section).
4. If the scale is not used for an extended period of time, remove the battery.
5. The scale maximum weight capacity is 600 lb. (300 kg). If patient weight exceeds maximum weight capacity, the scale will designate overload by displaying “OL-” .

Protekt® Digital Lift (30300) SCALE COMPONENTS

Ensure all components shown below are included in the Scale package.

Item	Qty	Picture	Description
1	1		Scale
2	1		Bushing 1220
3	1		Bushing 1020
4	1		Universal bracket
5	2		Pin
6	2		Key ring
7	2		Washer

SCALE INSTALLATION ON PROACTIVE PATIENT LIFTS: 33500 / 33600 / 33400P

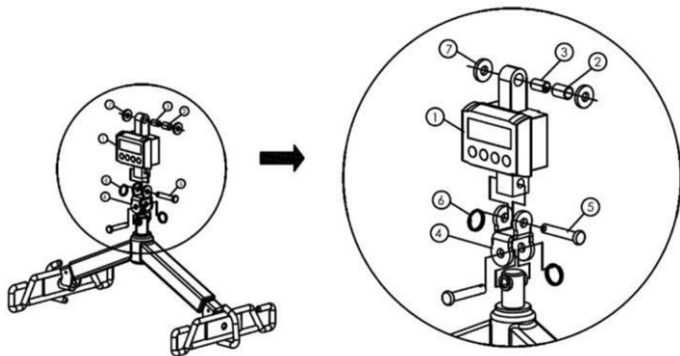

Install 30300 Scale on Proactive Patient Lifts
33500/33600/33400P as shown above.


**⚠ WARNING: Ensure scale is securely mounted
as shown above before proceeding.**

LIFT SCALE OPERATION INSTRUCTIONS

Keyboard Illustration



Display Illustration

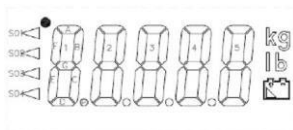

Instructions for Use

1. Place sling to be used onto spreader bar without patient/resident/user.
2. Press the scale faceplate On/Zero button to zero-out the sling weight (this weight will remain zeroed out until the Off button is pressed).
3. Press the **Lb.** button to select mode you prefer.
4. Carefully lift the patient/resident/user in the sling, performing the procedure so that all of the weight is supported by the lift. Stabilize the movement of the sling after lifting.
5. Weight will display on the scale. Press the Operate/Hold button to hold the weight displayed. To reset the scale to zero, press the Operate/Hold button again.
6. Lower patient/resident/user safely.
7. If you are weighing another individual using the same sling, perform the lift and their weight will display. Press Operate/ Hold to maintain their weight on the scale display.
8. Press the Off button when the weighing procedure is complete.
9. The scale will automatically power off five minutes after measurement has been completed.

LIFT SCALE


MAINTENANCE Cleaning

As needed, wipe scale gently, using mild soap or a mild disinfectant.



NOTICE: DO NOT wash under water pressure or steam clean. DO NOT use solvents, abrasive cleaners, or scouring pads to clean scale.

Battery Replacement

When the low battery indicator () appears in the display's right corner, replace the battery:

1. Open the scale back battery compartment.
2. Remove battery and replace with a 9V alkaline battery. Ensure battery polarity is correct.
3. Close the scale back battery compartment.

If the scale is not used for an extended period of time, remove the battery.

LIFT SCALE SPECIFICATIONS

Model Number	30300	
Capacity	600 lb (300 kg)	
Resolution	1/3,000 ±1d	
Division	0.2 lb (0.1 kg)	
Power Source	9V alkaline battery x 1	
Temperature	Operating	32°F ~ 104°F (0°C ~ 40°C)
	Storage	32°F ~ 158°F (0°C ~ 70°C)
Dimensions (W x H x D)	3.54 x 2.83 x 2.17 in. (90 x 72 x 55 mm)	
Product Weight	24 oz ± .35 oz / 1.5 lb ± .02 lb (675 g ±10 g)	
LCD Display	2.60 x 1.28 in. (66.0 x 32.4 mm)	
Display Description	Five digit indication	
	Lb./Kg selectable unit mode	
	Low power alarm battery symbol	
	S1 "Zero" indication	
	S4 "Hold" indication	
	(S2 and S3 are not used)	

LIMITED WARRANTY

The Propekt® Digital Patient Lift Scale you purchased is guaranteed to be free from defects in material and workmanship under normal use and service. The warranty period for this product is 1 Year from the date of purchase by the original purchaser. This warranty shall be voided upon transfer of ownership of this Product. Proactive Medical agrees to repair or replace this product, at our discretion and at no charge, within the warranty period provided and that Proactive Medical consents the unit is defective. The repaired or replacement unit shall be warranted for a period equal to the balance of the defective unit's warranty period.

NOTE:

To ensure the best service to our customers, Proactive Medical requires the following information to be included with the returned unit:

1. Model Name/Number and Serial Number on the packaging.
2. Proof of purchase, i.e. a copy of the original invoice.
3. A Return Authorization number (RA#) obtained by calling Authorized dealer/distributor prior to the return of the product. The RA# must be clearly indicated on the outside of the packaging.
4. A detailed description of the problem and its symptoms on a note.

This warranty shall not apply to any product which has been repaired or altered in any way so as, in our judgment, to affect its functionality and durability, nor to any product subject to abuse, misuse, negligence or accident, improper maintenance, improper installation, nor to any product used with other parts, components and/or accessories with quality and/or specifications not compatible with this product. This warranty does not cover products that have been impaired by occurrences considered Acts of God over which authorized dealer has no control. This warranty shall also be voided if any required periodic maintenance, if applicable, has not been properly performed on this product.

This warranty and the aforementioned remedies presented are exclusive and in lieu of all other express or implied warranties. No other representations or claims shall be binding or obligate authorized dealer in any way. Any warranty applicable to this product is limited to the aforementioned period indicated. In no event shall authorized dealer be liable for any special, incidental, or consequential damages; loss of revenue; or cost of replacement goods; resulting from the use or malfunction of this product to the associated equipment on which and with which it is used. This warranty gives specific legal rights and you may be entitled to other rights that vary from state to state.