


## 5 C O O T E R

## **Installing the Triaxe Cruze Basket**

1) The following items are included in the Triaxe Basket Installation kit...

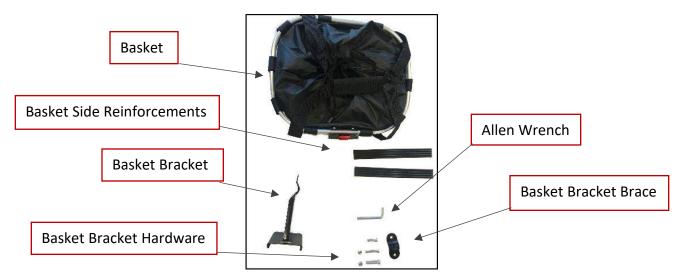


2) Attach the Basket Bracket and the Basket Bracket Brace to the Handlebar Tube using the hardware and included Allen Wrench as shown below.



3) Make sure the other end of the Basket Bracket is positioned as shown here.



4) Slide the Basket onto the basket bracket. The red lever will lock the basket into place.

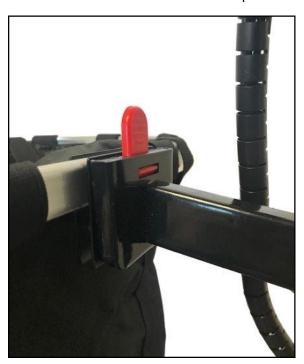

5) Slide the Basket Side Reinforcements into the side pockets of the basket.



Contact your local TRIAXE dealer or the Enhance Mobility Service Department with any questions.

