

**APPLICATION DATASHEET**  
**Assistica® ELASTIC MEDICAL BELT**  
**FOR UMBILICAL HERNIA**  
**(for Children)**

**Model:**

- AA 501-D

**Purpose:**

The umbilical hernia belt is intended for conservative treatment of umbilical hernia for children aged up to 6, for prevention of abasement and impaction of hernial sac.

**Medical Indications:**

Conservative treatment of umbilical and paraumbilical hernias and epigastric hernia, prevention of relapses after herniotomy, prolapse and incarceration of hernial sac and other complications. .

**Structure of raw material:**

LATEX FREE: Polyurethane foam – 80%, Polyamide – 10%, Cotton – 10%.

Bandage: PVC – 100%.

**Technique of application:**

**Attention!** This belt should only be used under medical supervision.

Hernia belts can be used at hospitals, in the process of outpatient care and at home, but only after being trained in the technique of umbilical hernia taxis. The belt should be worn from 2 to 24 hours a day, unless prescribed otherwise by the doctor.

The product is made from soft, ecologically clean linen, the internal surface of which is cotton and the external surface napped polyamide, easily fastened with the help of the Velcro fastening.

The product consists of two parts: the front wider part (a hernial bandage is sewn in from the inner side) and a narrow belt with the velcro fastening. It is recommended to put the bandage on in a lying down position, following the reduction of the hernial protrusion. The semi-rigid bandage is placed on the umbilical area, then the belt is affixed with the help of the velcro fastening on the front part of the product. It is compulsory to ensure that the belt fits tightly to the body, while at the same time not disturbing the blood supply of the soft tissues. If the navel has not yet healed, it is recommended to use a cotton (gauze) pad.

**Sanitary-and-hygienic processing:**

The umbilical hernia belt should be hand-washed in a soap solution, at a temperature of +35°C to +40°C, not using substances containing bleach and by taking out the bandage. Do not apply chemical cleaners. It is recommended to carefully wring without twisting and spread flat for drying. Ironing is prohibited.

**Storage:**

The product is to be stored in a clean, dry and ventilated place at a temperature of +3°C to +25°C and relative air humidity 60% to 65%.

**Contraindications:**

individual incompatibility with components of raw materials of the product, constricted and irreducible hernias, intertrigos.

**Packing:**

1 piece.

**Available sizes:** One size.

Waistline circumference: 36.0-52.0 cm

The size of the belt is universal and can be used even for infants.

**Best before** – 5 years.

**Your remarks and offers direct to the address of seller:**

*SELL WELL SIA*

*Ropazu iela 21-1, Riga, LV-1039, Latvia*

*E-mail: [sales@assistica.com](mailto:sales@assistica.com)*

*Made in European Union*

*Manufacturer is certified to ISO 9001*

