

APPLICATION DATASHEET

Assistica® MEDICAL ELASTIC COMPRESSION SLEEVE

Models:

- AA 403-0T, AA 403-1T, AA 403-2T

Purpose:

Medical elastic compression sleeve is recommended for prophylaxis and treatment of a lymph outflow disorder shortly after breast excision and at a later period, as well as after traumas, fractures, deep burns, effects of lymphangitis and thrombophlebitis of arms, and underarm radiotherapy.

Description:

The sleeve is woven of medical elastic stockinet in accordance with seamless technology. The product length is from the wrist to the underarm. The product's interweaving secures the distributed pressure (decreasing from hand or forearm towards shoulder) on soft tissues of arm that leads to increase of the venous blood flow rate, filtration reduction and the increased reabsorption of extracellular fluid, and secures the physiologic lymph flow. This prevents the appearance of oedemas and reinforces the previous treatment result.

The sleeve with a shoulder can be fixed on the body with the help of a special elastic band or on a bra shoulder-strap with the help of the *velcro* fastening.

Composition of raw materials: polyamide – 80%, lycra – 20%.

Methods of application:

The compression class and the product type are determined by a doctor. The product size is selected strictly according to the individual anatomic measurements of the arm.

The compressive sleeve should be worn daily. Put it on in the morning, as soon as you wake up. First, turn the sleeve inside out up to the part which is put on the wrist. Put the product on. Evenly, without twisting, straighten the product at your arm. Make sure that the sleeve is evenly stretched along the entire arm and that it neither forms folds, nor causes excessive compression.

Contraindications:

Individual incompatibility with components of raw materials of the product, dermatitis of various aetiologies, eczemas (primarily, weeping ones), erysipelalous inflammation (in an acute stage), severe forms of cardiopulmonary decompensation, and chronic arterial insufficiency.

Sanitary-and-hygienic processing:

The compressive products are hand-washed in a soap solution, at the temperature of +40°C, not using substances containing bleachers. Do not apply cleaning with chemicals. Careful wringing without twisting and spreading for drying is recommended. Ironing is prohibited.

Storage:

The product is to be stored in a clean, dry and ventilated place at a temperature of +3°C to +25°C and relative air humidity 60% to 65%.

Packing:

1 piece.

GUARANTEE:

The manufacturer guarantees the most important elements for success in compression therapy:

- The adjusted distribution of therapeutic pressure;
- Exact compliance of the product to the determined compression class;
- Compliance to the size selected strictly according to the individual anatomic measures. The scheme of measurements and the table of sizes are provided in online item description.

The guaranteed therapeutic compression is retained for 6 months under conditions of daily wear and with the observation of the product usage rules provided in the Instructions for Use.

The guarantee does not cover the mechanical damage of the products.

The manufacturer's warranty covers **only retention of the curative effect**, i.e. retention of compressive properties of the product.

- We recommend taking jewellery off prior to putting on the compressive products. Using rubber gloves is advised.
- We do not recommend drawing or cutting off technological threads on the inner side of the product, as well as threads and stitches which appear during careless wear.
- The compressive products are resistant to the impact of creams and ointments, but accumulation of remnants of these substances, dust and epidermis leads to fraying of threads. We recommend daily manual washing in warm water with chlorine-free soft detergents.

The products are replaced only after their examination by the company specialists or following and independent expert examination.

Best before – 5 years from the date of purchase.

Your remarks and offers direct to the address of seller:

SELL WELL SIA

Ropazu iela 21-1, Riga, LV-1039, Latvia

E-mail: sales@assistica.com

Made in European Union

Manufacturer is certified to ISO 9001

