



**THUASNE**

## Dorso Rigid 35

Immobilizing Hyper-Extension Brace



**TOWNSEND**

THUASNE USA

### Patient Instruction Guide

Dorso Rigid 35<sup>®</sup> is a rigid brace designed to support the spine in a hyper-extended position. Its adjustment capabilities promote functional rehabilitation with an excellent level of comfort.

Use in accordance with the recommendations of the health professional that prescribed or dispensed the product.

**⚠** The adjustments made by the health professional (height, width, angle of hinges, strap tension) must never be modified by the patient.

- 1) Open the fastening lever and unhook the strap from the strap Adjustment Screw (Fig. 1A)
- 2) Position Dorso Rigid 35<sup>®</sup> over the chest then pass the strap behind the back and attach it to the strap adjustment screw
- 3) Position Dorso rigid 35<sup>®</sup> so that the Sternal Pad (Fig. 1B) centered over the chest and the Pelvic Strap (Fig. 1C) does not cause discomfort when sitting
- 4) Tighten the Dorso Rigid 35<sup>®</sup> by using the Fastening Lever (Fig. 2A)

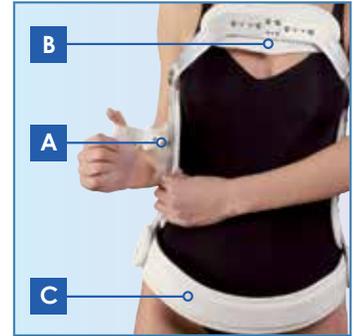


Figure 1

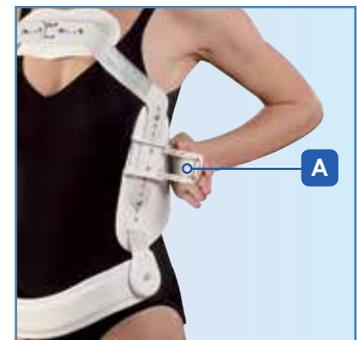


Figure 2

### Water Resistant materials

The frame and fastening system are made of water resistant materials. The padded parts of the corset are made of water-repellent materials, enabling patients to wear Dorso Rigid 35<sup>®</sup> safely in the shower or during functional rehabilitation sessions in the swimming pool.

### Washing instructions:

- Hand wash and air dry only
- Do not use harsh detergents, fabric softeners or aggressive products containing chlorine or bleach
- Dry away from heat sources

In the event there are any issues related to use of Dorso Rigid 35<sup>®</sup> consult the medical provider who prescribed or dispensed Dorso Rigid 35<sup>®</sup>.

Manufactured by:



THUASNE SAS 118-120 rue Marius AUFAN  
92300 Levallois Perret France  
Commandes/Orders (St-Etienne) :  
Tél. 04 77 81 40 42 (France)  
Tel. +33 (0)4 77 81 40 01/02 (Export)

Distributed by:



4615 Shepard Street Bakersfield, CA 93313  
Telephone 800.432.3466 - Fax 800.798.2722

[www.ThuasneUSA.com](http://www.ThuasneUSA.com)



1000 Calle Cordillera San Clemente, CA 93313  
Telephone 855.784-6600 - Fax 855.784.6601