

## GETTING STARTED



### CHARGE THE LED DEVICE

PLACE THE BLACK LED DEVICE INTO ITS CHARGING PAD, PLUG THE DEVICE INTO AN OUTLET USING THE PROVIDED USB-C CORD. THE DEVICE WILL 'GLOW' WHILST CHARGING. IT TAKES 3.5 HOURS OF CHARGING TIME FOR ABOUT 6 TIMES USAGE.



### ASSEMBLE THE DEVICE

ENSURE THE RUBBER MOUTHPIECE IS SECURELY ATTACHED TO THE LED DEVICE.



### BRUSH TEETH BEFORE USE

BRUSH YOUR TEETH FOR 2 MINUTES BEFORE STARTING THE WHITENING TREATMENT. THIS REMOVES ANY PLAQUE AND BACTERIA, AND GIVES THE WHITENING PRODUCTS A CLEAN SURFACE TO WORK ON.

## APPLYING THE GEL



### OPENING THE PLATINUM PEN

TWIST THE BOTTOM OF THE WHITENING PEN TO ENGAGE THE GEL INTO THE BRUSH. IF THIS IS THE FIRST TIME YOU ARE USING THE PEN, THIS WILL TAKE A GOOD FEW TWISTS (UP TO 20 SECS).



### APPLY TO TEETH

USING THE BRUSH END OF THE WHITENING PEN, BRUSH A LAYER OF WHITENING GEL ONTO THE FRONT SURFACES OF EACH TOOTH. THE BRUSH IS GREAT FOR REACHING IN BETWEEN TEETH ALSO.



ENSURE THE GEL DOES NOT TOUCH YOUR GUMS, CHEEKS OR TONGUE AS THIS MAY CAUSE IRRITATION. AVOID PLACING TOO MUCH GEL ON THE TEETH TO PREVENT IT FROM PUSHING UP ONTO THE GUMS WHEN THE MOUTHPIECE IS PLACED IN THE MOUTH.

## STARTING THE TREATMENT



### FIT MOUTHPIECE

PLACE THE MOUTHPIECE WITH THE LED DEVICE ATTACHED INTO YOUR MOUTH. ENSURE THE MOUTHPIECE IS SEATED COMFORTABLY IN THE MOUTH AND YOUR LIPS ARE ON THE OUTSIDE OF THE LIGHT.



### POWER ON LED DEVICE

PRESS AND HOLD THE BUTTON FOR 1 SEC ON THE LED DEVICE TO POWER ON.



### WHITEN FOR 25 MINUTES

THE BLK LIGHT 2.0 HAS AN IN-BUILT 25 MINUTE TIMER. WHITEN UNTIL THE LED DEVICE AUTOMATICALLY POWERS OFF.



### FINISHING UP

ONCE THE BLUE-LIGHT HAS SWITCHED OFF ON THE LED DEVICE, RINSE YOUR MOUTH THOROUGHLY WITH WATER. YOU MAY USE A COLOURLESS MOUTHWASH TO FRESHEN AFTER WHITENING, BUT AVOID BRUSHING YOUR TEETH IMMEDIATELY AFTER TO ALLOW THE TEETH TO REMINERALISE.

## MAINTENANCE OF THE KIT



### CLEANING THE MOUTHPIECE

AFTER EACH USE, DETACH THE RUBBER MOUTHPIECE FROM THE LED DEVICE AND RINSE WITH COLD WATER. LEAVE TO DRY BEFORE REASSEMBLING.



### STORAGE

IT IS IMPORTANT YOU STORE THE PLATINUM KIT IN A COOL, DARK, AREA AWAY FROM DIRECT SUNLIGHT. THE ACTIVE INGREDIENTS IN OUR PRODUCTS ARE MOST STABLE WHEN KEPT IN COOLER TEMPERATURES. IF YOU WANT TO, YOU CAN EVEN STORE YOUR WHITENING PENS IN THE FRIDGE FOR BEST RESULTS.

THE PLATINUM KIT  
THE WHITENING CO.

## WHAT'S INCLUDED:



### 2 X PLATINUM PENS

FOR 30 WHITENING TREATMENTS.



### 1 X BLK LIGHT 2.0

WITH A 25 MINUTE BUILT IN TIMER AND DETACHABLE RUBBER MOUTHPIECE.



### 1 X CHARGING PAD

THE NEW WAY TO CHARGE YOUR BLK LIGHT. PLACE THE LIGHT INSIDE AND CHARGE YOUR DEVICE WIRELESSLY.



### 1 X USB-C CORD

TO POWER YOUR CHARGING PAD.

MANUFACTURED BY  
THE WHITENING CO.  
4 PONSONBY ROAD,  
GREY LYNN,  
AUCKLAND 1011

WWW.WHITENINGCO.NZ  
@THEWHITENINGCO

## THE BLK LIGHT 2.0



### THE RE-ENGINEERED BLACK LIGHT

CRAFTED FROM YEARS OF RIGOROUS TESTING AND TRIALS WITH OUR PREVIOUS DEVICES, WE HAVE ARRIVED AT THE PINNACLE OF WHITENING TECHNOLOGY. THE BLK LIGHT 2.0 ENCOMPASSES EVERYTHING YOU COULD WANT IN A POCKET-SIZED WHITENING DEVICE.

## ACTIVE INGREDIENTS



THE 6.9% PENS ARE MADE UP OF EXACTLY THAT - THEY CONTAIN 6.9% HYDROGEN PEROXIDE AS THE ACTIVE INGREDIENT.

OUR FORMULA IS DESIGNED TO OXIDISE INTRINSIC AND EXTRINSIC TOOTH STAINS WITHOUT DAMAGING THE TOOTH STRUCTURE OR CAUSING TOOTH SENSITIVITY. HYDROGEN PEROXIDE IS THE MOST COMMON, SAFE AND EFFECTIVE TEETH WHITENING INGREDIENT AND IS TRIED AND TRUSTED BY DENTISTS AND TEETH WHITENERS WORLDWIDE.

## WHEN TO USE:



TAILOR THE PLATINUM KIT TO MEET YOUR TEETH WHITENING NEEDS. WE RECOMMEND USING THE KIT AT NIGHT TIME AFTER YOUR ORAL CARE ROUTINE. THE BEAUTY OF A HOME WHITENING KIT MEANS YOU CAN WHITEN AS MUCH OR AS LITTLE AS YOU LIKE. YOU WILL SEE GRADUAL RESULTS WITH EVERY USE HOWEVER THE BEST WHITENING RESULTS COME FROM CONSISTENT USE.



FIRST-TIME TEETH WHITENERS: YAY! CONGRATULATIONS ON STARTING YOUR TEETH WHITENING JOURNEY. FOR YOU, WE WOULD RECOMMEND USING THE HOME WHITENING KIT ONCE EVERY 1-2 DAYS FOR 7-14 DAYS TO REACH YOUR INITIAL DESIRED SHADE.



MAINTENANCE: FOR THOSE WHO ARE WANTING TO MAINTAIN THEIR ALREADY WHITE TEETH, WE RECOMMEND USING THE KIT 1-2 TIMES A WEEK.



TEETH WHITENING WORKS BEST WHEN USED ALONGSIDE A GOOD ORAL HYGIENE ROUTINE. FOR BEST ORAL HEALTH, IT IS ENCOURAGED TO BRUSH TWICE DAILY FOR 2 MINUTES, FLOSS ONCE A DAY AND VISIT YOUR DENTIST/HYGIENIST EVERY 6-12 MONTHS.