GoMini™ Electric Breast Pump

Single / Double

Instruction Manual

1.0 Usage


1.1 Breast Pump parts	1
1.2 Precautions for use	2-3
1.3 How to assemble	4-6
1.4 How to use	7-8
1.5 How to disassemble	9
1.6 How to store expressed milk	10
1.7 How to wash and sterilize	11-12
1.8 About the bottle and nipple	12
1.9 Troubleshooting	13

2.0 Product specifications	14
---	-----------


EN

1.1 Breast Pump parts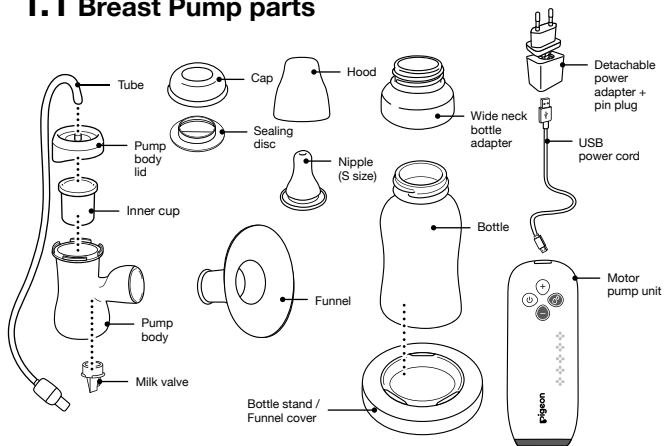

Quick start guide

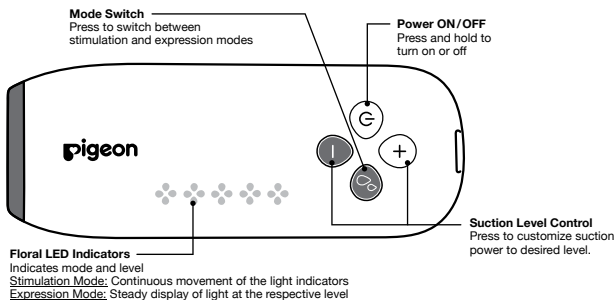Switch on the pump by pressing the ON/OFF button. By default, it will be in the stimulation mode. After 2 minutes of continuous pumping in the stimulation mode, it will automatically switch to level 3 of the expression mode. You may switch to the expression mode manually even before the 2 minute mark according to your preference. The suction level can be adjusted to your level of comfort.

1.2 Precautions for use

IMPORTANT SAFEGUARDS When using electrical products, especially when children are present, basic safety precautions should always be maintained, including the following:
PLEASE READ THIS BEFORE USING THE PUMP

1. THIS APPLIANCE IS NOT INTENDED FOR USE BY PERSONS (INCLUDING CHILDREN) WITH REDUCED PHYSICAL, SENSORY OR MENTAL CAPABILITIES, OR LACK OF EXPERIENCE AND KNOWLEDGE, UNLESS THEY HAVE BEEN GIVEN SUPERVISION OR INSTRUCTION CONCERNING USE OF THE APPLIANCE BY A PERSON RESPONSIBLE FOR THEIR SAFETY. CHILDREN SHOULD BE SUPERVISED TO ENSURE THAT THEY DO NOT PLAY WITH THE APPLIANCE. 2. THIS APPLIANCE CAN BE USED BY CHILDREN AGED FROM 8 YEARS AND ABOVE AND PERSONS WITH REDUCED PHYSICAL, SENSORY OR MENTAL CAPABILITIES OR LACK OF EXPERIENCE AND KNOWLEDGE IF THEY HAVE BEEN GIVEN SUPERVISION OR INSTRUCTION CONCERNING USE OF THE APPLIANCE IN A SAFE WAY AND UNDERSTAND THE HAZARDS INVOLVED. CHILDREN SHALL NOT PLAY WITH THE APPLIANCE. CLEANING AND USER MAINTENANCE SHALL NOT BE MADE BY CHILDREN WITHOUT SUPERVISION. 3. THE APPLIANCE IS ONLY TO BE USED WITH THE POWER SUPPLY UNIT PROVIDED WITH THE APPLIANCE.

The following symbols indicate incorrect handling that can result in injuries or damage.

 Warning	Indicates incorrect handling that can cause major injury or death.	 Caution	Indicates incorrect handling that can cause injury or property damage.
 Instruction	Indicates incorrect handling that can cause major injury or death.	 Prohibited	Indicates incorrect handling that can cause injury or property damage.

Danger: To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not use while bathing.
- Do not place or store this product where it can fall or be pulled into a tub or sink.
- Do not place or drop pump into water or other liquid.

 **Warning: To reduce the risk of burns, electrocution, fire, or injury to persons:**

-  Close supervision is necessary when this product is used near children or invalids.
-  This product should never be left unattended when plugged in.
-  Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.

-  Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
-  Wipe thoroughly, if there is dust on the AC adapter attachment plug. Failure to do so may cause fire.
-  Do not disassemble or modify the AC adapter and motor. Doing so may cause fire as a result of electric shock or contamination.
-  Do not insert or remove the AC adapter attachment plug with wet hands. Also do not handle batteries with wet hands. Doing so may cause electric shock.
-  Be careful not to damage the AC adapter. (Do not scratch, modify, place near a heater, bend excessively, twist, pull, place heavy objects on, or bundle it.) Doing so may cause fire, electric shock, or failure.
-  Never use an AC adapter other than the one provided. Doing so may cause fire, electric shock, or product failure.
-  Keep the cord away from heated surfaces.
-  Never use while sleeping or drowsy.
-  Never drop or insert any object into any opening or tubing.
-  Do not use this product if it is broken or any parts are missing.
-  Do not allow small children or pets to play with the pump unit, AC adaptor, etc.
-  Do not use outdoors with the AC adapter or operate where aerosol spray products are being used or where oxygen is being administered.

Caution

-  This breast pump is intended for use by a single user only.
-  Do not use excessively or for continued suction. Doing so may cause nipple pain.
-  Stop using the breast pump if your nipples are sore. Continued use may cause your symptoms to worsen.
-  Do not wipe with thinners, benzine, or alcohol. Doing so will cause deformities and damage.
-  Do not use detergents that contain abrasives. Doing so will cause scratches to the product.
-  Do not leave it near an open flame. Doing so will cause deformities and damage.
-  Do not expose the pump to severe shocks such as dropping or throwing. Doing so may cause malfunction or damage.
-  Do not express more than 160 ml of liquid into the bottle. The expressed breastmilk may overflow.
-  Prior to using the pump, check to make sure there are no abnormalities such as scratches, breaks, cracks, or deformities.
-  Clean your hands before expressing.
-  Wash and sterilize before using.

1.3 How to assemble the product

- Please assemble the parts when they are completely dry after washing and sterilizing

- 1 Attach the milk valve to the pump body.


- 2 Insert the inner cup into the pump body.


- ◆ **Tip:** Please make sure that the inner cup is not turned inside out.

Note
Please make sure that there is no moisture or dust on the inner cup before use. Moisture or dust on the inside of the inner cup may cause malfunction.


	Correct Direction	Inside Out
View from diagonally above	<p>Thin line in the middle</p>	<p>Gap</p>
Cross-sectional diagram		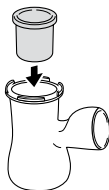


- 3 Align the Δ symbol on the pump body lid with the ∇ symbol on the pump body, insert it, and turn it clockwise until it stops.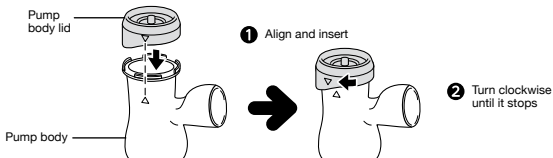

- 4 Attach the pump body to the bottle.

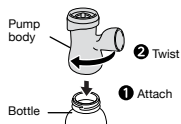
- 5 Attach the funnel to the pump body.

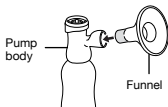
- 6 Press the bottle into the bottle stand.


◆ **Tip:** Using the pump with the bottle stand keeps it from toppling easily.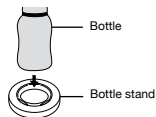

- 7 Insert one end of the tube into the pump body.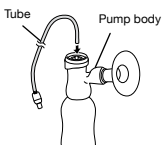

- 8 Insert the tube with connector into the motor.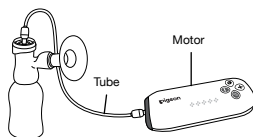


- 9 Plug the USB power cable to the power adapter and the micro-USB cable to the motor. Secure the pin plug to the power adapter before plugging into the socket.

◆ Tip: Other than using the power adapter to supply power, you can also connect the USB power cable to a portable power source (with output specification of 5V, 2A).

Note

The power adapter sometimes heats up during use. It is not a malfunction. Breast pump package comes in UK 3-pin plug for sale in Singapore.



Assembly completed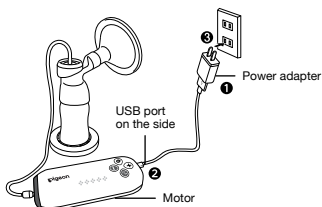


1.4 How to use

Fitting the funnel properly

- Centre the funnel over your nipple/papilla.
- Press the funnel firmly against your breast so that air does not escape from the sides.
- You can create a good seal by using your other hand to support your breast from below.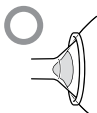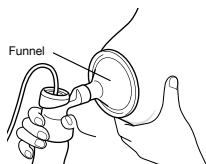

The nipple position is not centred.

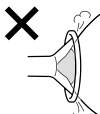Air escapes



Expression sequence

Stimulation mode

Gentle stimulation rhythm

Expression mode

Select the preferred suction pressure (5 levels)

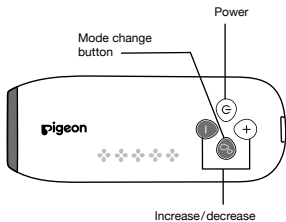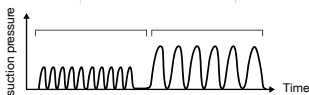Tip

- The estimated duration for expressing each breast is 10 minutes.
- The volume of breast milk expressed varies with the condition of the breast and the individual's condition.
- Please observe your condition while expressing.

Note

- Please observe your condition while you adjust the suction pressure to something that is right for you. Suddenly increasing the suction pressure may cause nipple pain.
- Do not over-express or express for a prolonged period. Doing so may cause nipple pain.
- The clicking sound emitted during operation is the release of pressure from the milk valve and is normal.

Stimulation mode

Gently stimulate your breast using a slow rhythm.

Breast milk starts expressing

Expression mode

Select the preferred suction pressure.

Stimulation mode – 1 Level

Long press the power button to enter the Stimulation mode. When breast milk starts expressing, press the  button to switch to the Expression mode. It is recommended that the Stimulation mode be used for not more than 2 minutes.

Expression mode – 5 Levels

After entering the Expression mode, press the   button to select the preferred suction pressure (5 levels). The suction pressure increases for each level. It is recommended to start expressing using lower levels of suction, then as your body adapts, gradually set the suction pressure to what is right for you.

Please wash after use.
(Refer to page 11-12)

How to operate the motor**1. Power**

- Power On: Long press 
- Power Off: Long press 

* Power cuts off automatically 30 minutes after the power is switched on.

2. Stimulation mode

- When the power is switched on, the pump automatically enters the stimulation mode.

3. Expression mode

- Press  to enter the Expression mode. (After entering this mode, the pump automatically switches to suction pressure Level 3.)
- Switching modes: Press  again to return to the Stimulation mode.
- Suction pressure: Press   to adjust the suction pressure.



Indicator lamp

- Stimulation mode: blinks from the first floral indicator to the fifth floral indicator.
- Expression mode: lights up from the first to third floral indicator, then additional floral indicator light up or go off as you press  .


1.5 How to disassemble

Please disassemble the parts that were pre-assembled at the time of purchase.

Pump body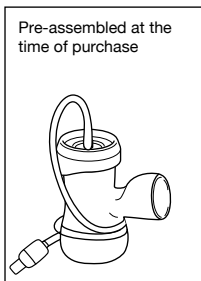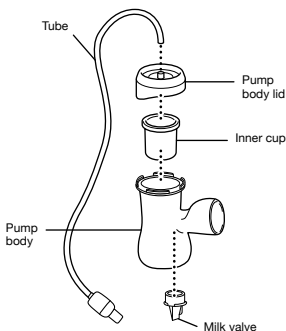

- 1 Hold the end of the tube and pull it out, then twist and remove the pump body lid.
- 2 Remove the inner cup.
- 3 Remove the milk valve from the pump body.

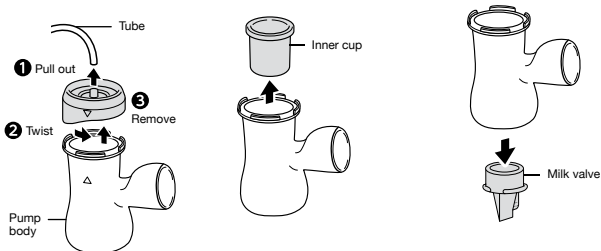1.6 How to store expressed breastmilk

Shelf life of breast milk

Please feed the expressed breast milk to your baby as soon as possible. Do not store the breast milk for longer than the duration indicated below.

	Refrigerator ($\leq 4^{\circ}\text{C}$)	Freezer (about -18°C)
Fresh breast milk	24 hours	3 months
Thawed breast milk (thawed in the refrigerator or submerged in running water)	24 hours	Do not refreeze

✳ Do not refreeze or refrigerate breast milk after heating

How to thaw frozen breast milk

Please use one of the following methods:

- Immerse in warm water (approximately 40°C).
- Place in the refrigerator.

✳ Remove from the freezer and place it in the refrigerator.

Do not leave in the refrigerator for more than 24 hours.

- Place in running water.

Heating to a suitable temperature for feeding

Immerse in warm water (approximately 40°C).

✳ It is normal for stored breast milk to become separated, and does not affect its nutrients. Please shake it gently to mix well before feeding.

✳ It is recommended that you bring along frozen breast milk (in an unthawed state) when you go out for a brief duration.

Note

- ⊗ Do not boil or use boiling water or a microwave to thaw or heat chilled or frozen breast milk. Doing so will destroy the nutrients in breast milk.
- ⊗ Do not thaw breast milk at room temperature.
- ⊗ Do not refreeze breast milk that has thawed.
- ⊗ Do not mix newly-expressed breast milk with the breast milk that has been stored.
- ⊗ The scale on the bottle is for reference only. If you need an accurate measurement, use a measuring cup or a measuring spoon.
- ⊗ Please note that the temperature may vary with the frequency of opening and closing the refrigerator door, as well as position in the refrigerator.

1.7 How to wash and sterilize

Washing

Precautions

- ※ Please disassemble all parts before washing and sterilizing.
- ※ The motor, power adapter, USB power cord, and tube cannot be washed and sterilized. Doing so may cause a malfunction. Please wipe with a dry cloth.


- Be sure to wash and sterilize the product before the first use.
- Please wash the product promptly after use. It is recommended to use the "PIGEON Liquid Cleanser".
- Be sure to sterilize each part after washing.
- The breast pump contains many small parts. Please be careful not to lose the small parts during washing.
- Use a soft brush for washing.
- Parts made of silicone rubber may be damaged or scratched if they are handled roughly. Please handle with care.
- If the pump body or tube become dirty, apply a diluted neutral detergent to a cloth and wipe clean.

Sterilizing

Boiling sterilization

- Boil the thoroughly-washed parts in a pot of boiling water for 5 minutes.
- Immerse the parts to be sterilized completely in water. Do not allow them to come into contact with the sides of the pot. Doing so will cause deformation or damage.
- Do not boil excessively. Doing so can shorten the lifespan of the parts.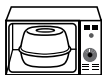

Microwave sterilization

- Please use containers designed specifically for microwave sterilizing.
- Do not place parts directly in the microwave for sterilization.
- Please follow the operating instructions for your microwave.
- Ensure the container is filled with water during heating.

Steam sterilization

- It is recommended to use a PIGEON steam sterilizer for steam sterilization.
- When using an electric steam sterilizer, please follow its operating instructions.

Other sterilization methods

- Do not use dry heat, UV or a dishwasher/dryer to sterilize the parts. Doing so may cause deformation or damage.

※ Parts are hot immediately after sterilizing. Please use caution to avoid injury.

Risk of scalding

Please assemble the parts.
(Refer to pages 4-6)

1.8 About the bottle and nipple

How to assemble nipple and cap

Place the nipple inside the cap

Attach the cap and nipple to the bottle


Fasten securely


Cover the nipple with the hood

1.9 Troubleshooting

The funnel may not produce enough suction pressure immediately after being pressed against the breast. Please continue to press it against the breast, then check the following: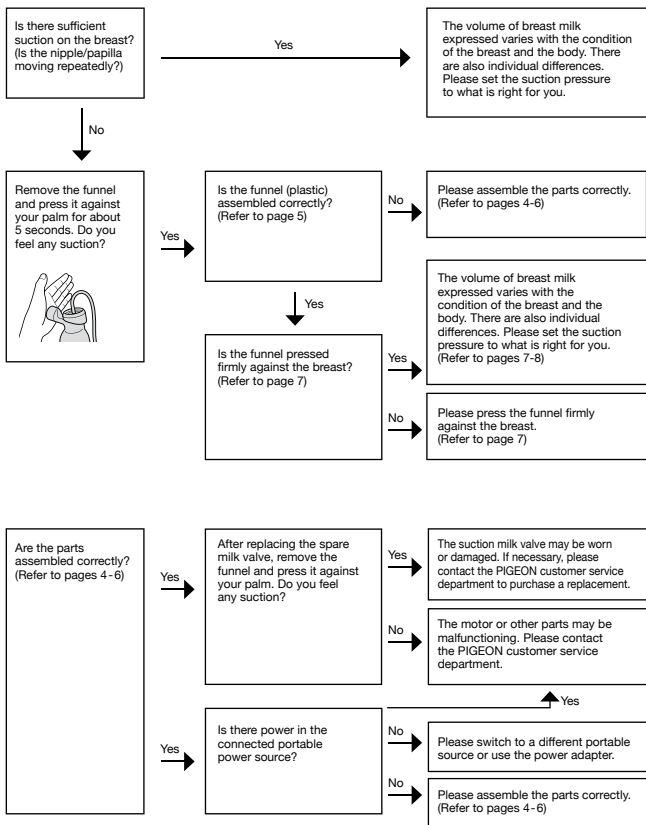

2.0 Product specifications

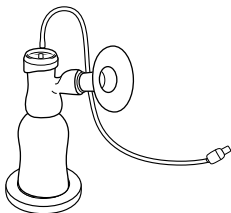■ Motor

Power	5W (power adapter or external portable power source)
-------	---

■ Power adapter

Input	AC100-240V 50/60Hz 0.2A Max
Output	DC5V1A

■ External portable power source specifications

Output current	1A
Output voltage	5V

Stimulation mode suction pressure/-kPa (Double pump)	10
--	----

Stimulation mode frequency (times/min)	117
--	-----

Expression mode suction pressure/-kPa (Double pump)	15 ~ 34
---	---------

Expression mode frequency (times/min)	32 ~ 85
---------------------------------------	---------

Under License

PIGEON

PIGEON CORPORATION

4-4, Nihonbashi Hisamatsu-cho, Chuo-ku,
Tokyo 103-8480 JAPAN www.pigeon.com

Assembled in Indonesia

Assembled by: **PT. Pigeon Indonesia**

Jl. Raya Serang Km.68, Kawasan Industri Modern
Cikande Kav. 4, Serang - Banten 42186, Indonesia

Visit our breastfeeding support
website for new mother and father!

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