# Breastmilk Storage Bags

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#### CAUTION تحذير PERHATIAN

- Do not thaw by immersing in boiling water, or by heating in the microwave or over an open flame. The nutritional content of the breastmilk may be damaged.
- Do not let breastmilk thaw at room temperature.
- Breastmilk storage bags are not reusable. Discard after each use.
- Keep out of the reach of children.
- In rare cases, an odor may be present due to the fact that the bags have been sterilized with gamma rays.
- To avoid freezer burn, be sure to compress all the air out of the bag before sealing.
- Be careful when handling bags. If a frozen bag is scratched, this can weaken the plastic and cause it to leak upon thawing.

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#### How to use

##### Before using

Clean hands thoroughly by washing with soap before using breastmilk storage bags, pumping breastmilk or feeding.

##### How to store breastmilk

- Tear away the safety seal at the top of a bag and open the bag by gently pulling the zipper seal apart. Do not blow into the bag.
- Carefully pour breastmilk into the bag. Do not fill the bag past the 180 ml line, as this may cause it to expand and leak in the freezer.
- Squeezing the bag with both hands above the level to which breastmilk has been filled, slowly compress the air out of the bag.
- Once all the air has been squeezed out, close the bag tightly by sliding the zipper shut while using your fingers to hold the end of the zipper down.
- After marking the top of the bag with the date and capacity, promptly place it in the freezer or refrigerator.

##### Storage period

The following are merely guidelines and do not provide any assurance of quality.

	Refrigerator (4°C or below)	Freezer (around -18°C)
Freshly pumped breastmilk	24 hours	3 months (recommended) 6 months (usable)
Thawed breastmilk (Thawed in refrigerator or under running water)	24 hours*	Do not refreeze.

\* If thawed by immersing in warm water, do not place back in the refrigerator.

##### How to thaw frozen breastmilk

Thaw breastmilk using one of the following methods.

- Immerse in warm water at around 40°C
- Place in the refrigerator. \*For up to 24 hours after taken out of the freezer
- Place under running water

##### How to warm to a suitable temperature for breastfeeding

Immerse in warm water at around 40°C.

- Breastmilk may separate during storage. This is normal and does not affect its nutritional content. Swirl gently before feeding.
- When going out, it is recommended that you transport the breastmilk in a frozen state (without thawing it).

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