

Shipping and posting information:

Because of overseas shipping requirements, we do not receive samples on Weekends or Bank Holidays Please ensure your samples are received by us no later than Friday Midday.

Due to the nature of this specimen and its volume, you are not allowed to send this sample through the Royal Mail postal system. Please use the UPS service offered or another of your choice.

For health and safety reasons, you must use all the packaging enclosed and seal each bag correctly before returning your samples. Regenerus Laboratories reserves the right to dispose of any sample received which does not meet this criteria.

Returning your samples in the UK via UPS

Enclosed you will find a return shipping label for our UPS courier service. Should you choose to use this service then you will be charged accordingly, details of which are separately enclosed.

Within the UK, UPS offer 3000 convenient drop off locations, this service means no waiting for couriers to arrive, we suggest you visit the UPS website at www.ups.com/dropoff. Alternatively the UPS home /office collection service normally offer a 4 hour collection window. Please see the enclosed shipping information sheet.

Track the status of your shipment, copy the tracking number on your shipping label and track its progress using the UPS website.

Regenerus Laboratories do not keep records of your tracking number.

CMI02

Comprehensive Stool Analysis + Parasitologyx 1

Before You Start:

Please read all of the directions including shipping before commencing collection. If you are taking antifungal or antibiotic medications, please finish the course of medication, and then wait three days before starting this collection. Please refrain from taking digestive enzymes, antacids, and aspirin for two days prior to and during the specimen collection, unless otherwise instructed by your physician. *Never discontinue prescription medications without first consulting your physician.*

Refrain from taking probiotic supplements for two weeks prior to specimen collection unless instructed otherwise by your physician

Verify Kit Contents:

- 1 Test requisition form
- 1 Collection container
- 1 YELLOW capped vial
- 1 BLACK capped vial
- 1 Disposable glove
- 1 Zip-lock bag with absorbent material
- 1 Blue Reply Envelope
- 1 White capped vial, 1 Polar Gel Pack

If you are missing components, please call Regenerus Laboratories Limited, customer service department for assistance.

Telephone : 0203 750 0870
Email: info@regeneruslabs.com
www.regeneruslabs.com

Aero 14 | Redhill Aerodrome | Kings Mill Lane | RH1 5YP
Regenerus Laboratories Limited: Inc in England and Wales: 08077110

Caution: Avoid contact of the skin and eyes with the fluid in the vials. For eye contact, flush with water thoroughly for 15 minutes. For skin contact, wash thoroughly with soap and water. For accidental ingestion, contact your physician immediately.


Collection Instructions:




Before commencing, write the patient's name, the date of collection, and patient's date of birth on all specimen vials. (You do not have to provide an ID#. The lab will assign one upon arrival). **The test cannot be performed without the patient information on the vials.**



1) Collect your stool specimen into the collection container. *DO NOT contaminate the specimen with either urine or water from the toilet.*

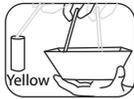


2) Unscrew the cap on the YELLOW-capped vial and, using the attached spoon, transport stool specimen into the vial. Take multiple portions from different areas of the collection container. Fill the vial to the fill line (approximately half full). **DO NOT OVERFILL.** Screw the cap on *tightly*.

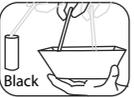

3) Unscrew the cap on the BLACK -capped vial and, using the attached spoon, transport stool specimen into the vial. Take multiple portions from different areas of the collection container. Fill the vial to the fill line (approximately half full). **DO NOT OVERFILL.** Screw the cap on *tightly*.

3) Unscrew the cap on the **white cap** vial and, using the attached spoon, transport stool specimen into the vial. Take multiple portions from different areas of the collection container. Fill the vial to the fill line (approximately half full). **DO NOT OVERFILL.** Screw the cap on *tightly*.

Place the white capped vial into the clear bag with the GEL freezer bag.
Seal and place in freezer for 6 hours

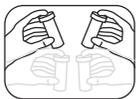

4) Shake both the YELLOW and BLACK -capped vials vigorously for approximately 30 seconds to mix the stool specimens with the preservative in the vials.

5) Place the both the BLACK and yellow-capped vials in the zip-lock bag, then place the bag into the box and wait completion of the test

When white capped sample has frozen for 6 hours or overnight, take all samples and place them in the return box with paperwork.
Place into blue reply bag , seal and ship without delay.



7) Fill out the test requisition form completely and sign it. **The test cannot be performed without a properly filled out requisition.** Place the form in the return box, The specimens are now ready for shipment.

To obtain the best results, Doctor's Data recommends shipping specimens as soon as possible. If you cannot ship the specimens the same day you finish the collection, such as over a weekend or holiday, place the cardboard box containing the black and yellow-capped vials into a refrigerator until you are ready to ship the test collection kit.

Keep the white cap frozen sample in the freezer during this time.

Consult your physician if you have any questions during the test collection process.