

Shipping and posting information:

Because of overseas shipping requirements, we do not receive samples on Weekends or Bank Holidays Please ensure your samples are received by us no later than Friday Middy.

Due to the nature of this specimen and its volume, you are not allowed to send this sample through the Royal Mail postal system. Please use the UPS service offered or another of your choice.

For health and safety reasons, you must use all the packaging enclosed and seal each bag correctly before returning your samples. Regenerus Laboratories reserves the right to dispose of any sample received which does not meet this criteria.

Returning your samples in the UK via UPS

Enclosed you will find a return shipping label for our UPS courier service. Should you choose to use this service then you will be charged accordingly, details of which are separately enclosed.

Within the UK, UPS offer 3000 convenient drop off locations, this service means no waiting for couriers to arrive, we suggest you visit the UPS website at www.ups.com/dropoff. Alternatively the UPS home /office collection service normally offer a 4 hour collection window. Please see the enclosed shipping information sheet.

Track the status of your shipment, copy the tracking number on your shipping label and track its progress using the UPS website.

Regenerus Laboratories do not keep records of your tracking number.

Telephone : +44 (0) 203 750 0870
Email: info@regeneruslabs.co.uk
www.regeneruslabs.com

Aero 14, Redhill Aerodrome, Kings Mill Lane, Redhill, RH1 5YP
Regenerus Laboratories Limited : Inc in England & Wales: 08077110



Regenerus Laboratories Limited

CMI 11-12-13

Parasitology x1 or 2 or 3

Before You Start:

Please read all of the directions inc. shipping to familiarise yourself with the collection procedures. If you are taking antifungal or antibiotic medications, please finish the course of medication, and then wait three days before starting this collection. Please refrain from taking digestive enzymes, antacids, and aspirin for two days prior to and during the specimen collection, unless otherwise instructed by your physician. *Never discontinue prescription medications without first consulting your physician.*

This test kit will contain 1, 2 or 3 duplicate sets of collection kits, depending on the number of days your physician or practitioner has requested samples to be collected for. If this is more than 1 kit, please repeat process on consecutive each day until completed.

Verify Each Kits Contents:

- 1 Test requisition form
- 1 Collection tray
- 1, 2 or 3 Black-capped vials
- 1 Disposable glove
- 1 Zip-lock bag with absorbent material
- 1 Blue Return Bag
- 1 Shipping Label
- Original Shipping Box (Keep for ret

If you are missing kit components, please call Regenerus Laboratories Limited Customer Service department for assistance.

Caution: Avoid contact of the skin and eyes with the fluid in the vials. For eye contact, flush with water thoroughly for 15 minutes. For skin contact, wash thoroughly with soap and water. For accidental ingestion, contact your physician immediately.

Collection Instructions for “Each Day”:



1) Collect your first stool specimen into the collection tray. *DO NOT contaminate the specimen with either urine or water from the toilet.*



2) Unscrew the cap on the black -capped vial and, using the attached spoon, transport stool specimen into the vial. Take multiple portions from different areas of the collection container. Fill the vial to the fill line (approximately half full). *DO NOT OVERFILL.* Screw the cap on *tightly*.

3) Shake the capped vial vigorously for approximately 30 seconds to mix the stool specimen with the preservative in the vial.



4) Write the patient's name, the date of collection, and patient's date of birth on the specimen vial. ***The test cannot be performed without the patient information on the vial.***

5) Place the black capped vials in the zip-lock bag, then place the bag into the return box.

(Repeat this process daily until all kits supplied have been completed).



6) Fill out the test requisition form completely and sign it. ***The test cannot be performed without a properly filled out requisition.*** Place the form in the return box The specimen is now ready for shipment.

Place box into return bag, seal and apply postage if appropriate

To obtain the best results, Doctor's Data recommends shipping specimens as soon as possible. If you cannot ship the specimen the same day you finish the collection such as over a weekend or holiday, place the Reply Envelope into a refrigerator until you are ready to ship the test collection kit.

Consult your physician if you have any questions during the test collection process.