


# ELBOW BRACE

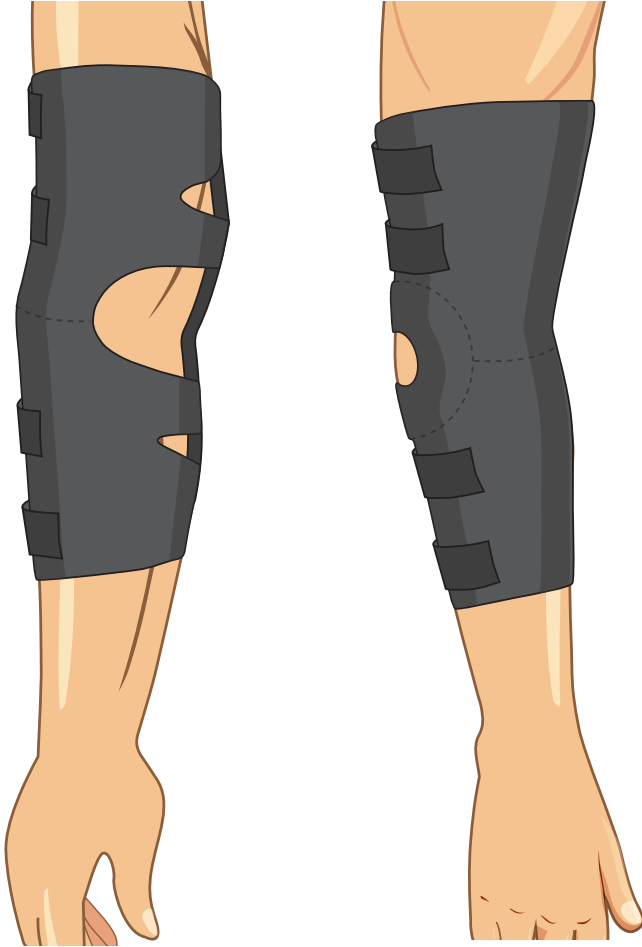

*Support, Compression and Protection for  
Bursitis, Sleeping at Night and Elbow Pain*

# APPLICATION INSTRUCTIONS

Thoroughly read the following information prior to application.  
Product function requires proper application.

.....

1. Prior to application, loosen all straps.  
Initially, brace is best applied with assistance.
2. Slide brace onto affected arm. With arm slightly bent, position the circular opening over the elbow. Secure middle straps as shown.
3. Secure upper biceps and lower forearm straps as shown.
  - Do not over tighten.

**Note:**

4. To facilitate re-application, when removing the brace, loosen but do not detach all straps. Then slide the brace off.
- .....

## INDICATIONS

Double layered fabrics around the elbow help to cushion and reduce pressure for elbow injuries. Helps to support a weak elbow with full range of arm motion.

.....

## CARE INSTRUCTIONS

Prior to cleaning, remove plastic stay from pocket. Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring. Reshape, dry flat. Wipe plastic stay clean with mild disinfectant. Replace plastic stay into pocket with textured surface facing outward, secure sleeve closure.


.....

**WARNING:** Incidents of pain, swelling, changes in sensation or unusual reactions should be immediately reported to your medical professional.


.....

Manufacturer makes no guarantees that injury will be prevented using this product.


1

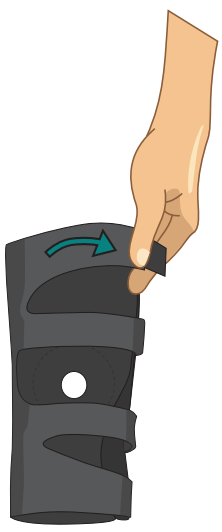


2

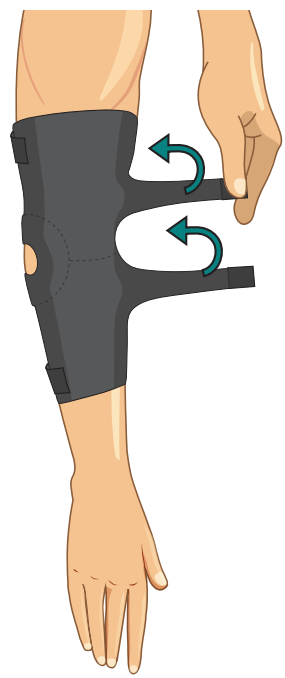

3

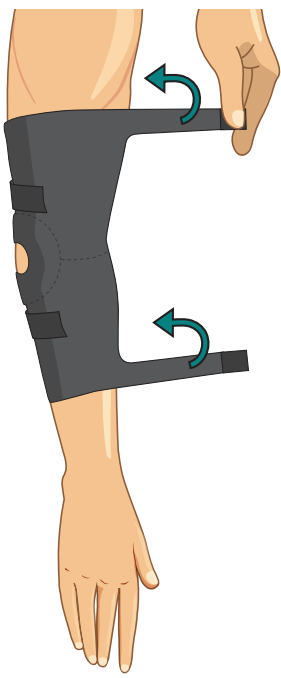

4

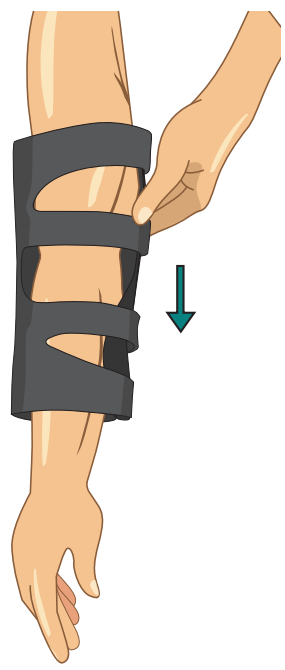

ID# \*281839\*



H/CB

Overstock  
Ortho 

[overstockortho.com](http://overstockortho.com)

