

Treatment Preparation

AquaBoost Back Facial



PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME & PRODUCT AMOUNT
PSN130 OR PSN131	Serum 1: Cleanse & Exfoliate Serum 2: Deep Cleanse
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves

CONTRAINDICATIONS
<ul style="list-style-type: none"> • Open lesions on the skin. • Pregnant & nursing women should consult with a physician. • Accutane or other skin-thinning medications or topical gels/creams. <p><i>**Always follow your company guidelines on who should and should not receive a treatment**</i></p>

SERVICE TIPS
<ul style="list-style-type: none"> • Guests may find their sides to be sensitive and prefer not to have their sides included in the treatment. • The AquaBoost Back Facial is suitable for the entire back however the 60-minute protocol includes the Mid Back, Upper Back, & Shoulders only in regards to product quantity. • It is appropriate to perform the back facial by spot treating versus the entire back if necessary.

HEMOCARE RECOMMENDATIONS
<p>It is recommended guests go home with or replenish the following to maintain results</p> <ul style="list-style-type: none"> • Purifying and/or Exfoliating Cleanser • Spot treatment for acne challenges

POST TREATMENT INSTRUCTIONS
<ul style="list-style-type: none"> • Wear Sunscreen Daily • Resume your normal skin care next day • Avoid exfoliation for the next 7 days. • Schedule next visit.

Quick Reference

AquaBoost Back Facial



STEP	PRODUCT NAME	INSTRUCTIONS
CLEANSE	Select Cleanser of Your Choice	
EXFOLIATE	Serum 1 Cleanse & Exfoliate	<ul style="list-style-type: none"> • Hold the skin taut with one hand while gliding the treatment wand with the other. This will help avoid skipping and rapid release of the wand.
EXTRACTIONS	Serum 2 Deep Cleanse	<ul style="list-style-type: none"> • Serum 2 is optional • If Serum 2 is not appropriate for your guest, perform manual extractions only at this time.
HYDRATE	Serum 3 Soothe & Hydrate	<ul style="list-style-type: none"> • Press any serum into the skin that is not immediately absorbed to avoid dripping.
CRYOWAND	Select Gel Mask or Heavy Gel Serum	<ul style="list-style-type: none"> • Alert the guest they will experience a cold feeling from the CryoWand. • Move in circular motions upward and outward. • Keep the CryoWand moving at all times.
MOISTURIZE	Select Moisturizer of your choice	<ul style="list-style-type: none"> • Apply additional moisturizer if needed.