

# Treatment Preparation

## AquaBoost Facial



### PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME & PRODUCT AMOUNT
PSN130 OR PSN131	Serum 1: Cleanse & Exfoliate Serum 2: Deep Cleanse
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves

#### CONTRAINDICATIONS

- Open lesions on the skin.
- Active Rosacea flare-up.
- Pregnant & nursing women should consult with a physician.
- Accutane or other skin-thinning medications or topical.
- Injections within the last 7 days.

*\*\*Always follow your company guidelines on who should and should not receive a treatment\*\**

#### SERVICE TIPS

- The AquaBoost Facial Enhancement is intended to be added to an existing facial for added exfoliation, decongestion, and hydration.
- It is recommended that the AquaBoost Enhancement replace the exfoliation in your current protocol.
- The enhancement begins at the exfoliation step of your current protocol.
- It is strongly recommended to wear gloves during the service.

#### HEMOCARE RECOMMENDATIONS

It is recommended guests go home with or replenish the following to maintain results

- Cleanser
- Purifying and/or exfoliating product
- Moisturizer

#### POST TREATMENT INSTRUCTIONS

- Wear Sunscreen Daily
- Resume your normal skin care next day
- Avoid exfoliation for the next 7 days.

# Details Protocol AquaBoost Facial



STEPS	PRODUCT NAME	INSTRUCTIONS
SKIN ANALYSIS	Select Cleanser of Your Choice	Analyze the skin and select serum <ul style="list-style-type: none"> <li>• 1 - Dry, Normal, Combination Skin Serum</li> <li>• 2 - Oily, Congested, Acneic / Acne Prone Skin</li> </ul>
SERUM 1 or SERUM 2	Cleanse & Exfoliate or Deep Cleanse	Perform 1 pass with Serum 1 or 2: <ul style="list-style-type: none"> <li>• Beginning at the chin, move in upward and outward motions.</li> <li>• Hold the skin taut with one hand while gliding with the other to ensure a smooth glide without skipping or rapid release.</li> <li>• Follow the grid provided in the Training Manual for added guidance.</li> <li>• Spend additional time as needed in areas of increased congestion.</li> </ul> *Approximately 3 fl. oz of serum will be used.
SERUM 3	Soothe & Hydrate	Perform 1 pass with Serum 3: <ul style="list-style-type: none"> <li>• Press excess serum into the skin throughout the service.</li> </ul> *Approximately 2-3 fl. oz will be used.
CRYOWAND	Select Gel Mask or Heavy Gel Serum	Perform 1 pass with CryoWand <ul style="list-style-type: none"> <li>• Begin at the chin, move in circular motions upward and outward.</li> <li>• Keep the CryoWand moving at all times.</li> </ul>