

Treatment Preparation

AquaBoost Hand Rejuvenation



PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME & PRODUCT AMOUNT
PSN130	Serum 1: Cleanse & Exfoliate
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves

CONTRAINDICATIONS

- Open lesions on the skin.
- Active Rosacea flare-up.
- Pregnant & nursing women should consult with a physician.
- Accutane or other skin-thinning medications or topical.

Always follow your company guidelines on who should and should not receive a treatment

SERVICE TIPS

- It is strongly recommended to wear gloves during the service.

HEMOCARE RECOMMENDATIONS

It is recommended guests go home with or replenish the following to maintain results

- Hand Cream

POST TREATMENT INSTRUCTIONS

- Wear Sunscreen Daily
- Resume your normal skin care next day
- Avoid exfoliation for the next 7 days.

Detailed Protocol

AquaBoost Hand Rejuvenation



STEPS	PRODUCT NAME	INSTRUCTIONS
CLEANSE – LEFT HAND		<ul style="list-style-type: none"> Cleanse the left hand with a warm towel.
SERUM 1	Cleanse & Exfoliate	<ul style="list-style-type: none"> Perform 1 pass with Serum 1 Hold the skin taut with one hand while gliding with the other to help ensure a smooth glide without skipping or rapid release. <p>Top of Hand:</p> <ul style="list-style-type: none"> Glide from the wrist down to the Metacarpal Bone Joints, working from the inside of the hand to the outside of the hand. Take caution over the Metacarpal Bones to not add too much pressure with the treatment wand. Glide down the top of each finger. Spend additional time as needed in areas of greater keratinization. <p>Palm of Hand (optional)</p> <ul style="list-style-type: none"> Repeat top of hand pattern. Can be performed on the entire palm of hand or used to treat calluses only. <p><i>*Approximately 2 fl. oz of serum will be used per hand.*</i></p>
SERUM 3	Soothe & Hydrate	<p>Perform 1 pass with Serum 3:</p> <ul style="list-style-type: none"> Follow the same glide patterns as you did for Serum 1. Press any excess serum into the hand as you proceed. <p><i>*Approximately 2 fl. oz will be used per hand.*</i></p>
CRYOWAND	Gel Mask or Serum of your choice	<ul style="list-style-type: none"> Turn on the CryoWand. Apply a thin layer of mask to the top of the hand. Alert the guest they will experience a cold feeling from the CryoWand. Use circular motions across the hand. Remove excess product with a dry towel.
MOISTURIZE	Moisturizer of your choice	<ul style="list-style-type: none"> Using a product of your choice, apply moisturizer to the hand and wrist. Perform brief hand massage. Place hand under draping when finished.
REPEAT ON RIGHT HAND		<ul style="list-style-type: none"> Repeat all steps on the right hand.