

Treatment Preparation

AquaBoost Facial



PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME
PSN130	Serum 1: Cleanse & Exfoliate
PSN131	Serum 2: Deep Cleanse
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves
PSN126	Prosana Refining Cleanser
PSN127 OR PSN123	Prosana Refining Mist Prosana Hydration Mist
MASK OF YOUR CHOICE	Suggestions: <ul style="list-style-type: none"> • Hyaluronic Acid Sheet Mask (HM4606) • 4-in-1 Full Face Crystal Collagen Gel Mask (HM4551) • Vitamin C Crystal Collagen Gel Mask (HM4569) • Neck Lift Crystal Collagen Gel Mask (HM4564)
PSN116	Prosana Nail & Cuticle Oil
PSN129	Calming Balm

CONTRAINDICATIONS
<ul style="list-style-type: none"> • Open lesions on the skin. • Active Rosacea flare-up. • Pregnant & nursing women should consult with a physician. • Accutane or other skin-thinning medications or topical. • Injections within the last 7 days. <p><i>**Always follow your company guidelines on who should and should not receive a treatment**</i></p>

SERVICE TIPS
<ul style="list-style-type: none"> • It is strongly recommended to wear gloves during the equipment use portion of this protocol. • Adjust your suction and flow settings according to your guests needs.

POST TREATMENT INSTRUCTIONS
<ul style="list-style-type: none"> • Wear Sunscreen Daily • Resume your normal skin care next day • Avoid exfoliation for the next 7 days.

HEMOCARE RECOMMENDATIONS
<p>It is recommended guests go home with or replenish the following to maintain results</p> <ul style="list-style-type: none"> • Cleanser • Purifying and/or exfoliating product • Moisturizer

Details Protocol

AquaBoost Signature Facial



STEPS	PRODUCT NAME	INSTRUCTIONS
CLEANSE	Prosana Refining Cleanser	<ul style="list-style-type: none"> Cleanse the skin using 1-2 pumps of the Prosana Refining Cleanser Once the skin has been cleansed thoroughly, remove the cleanser with a hot towel
TONE	Prosana Refining Mist OR Hydrating Mist	<p>Cover the eyes with a cotton round or esthetic wipe and apply:</p> <p>Refining Mist:</p> <ul style="list-style-type: none"> For skin in need of added exfoliation or dull, lackluster skin. Apply with a cotton round or esthetic wipe avoiding the eye area. <p>Hydration Mist</p> <ul style="list-style-type: none"> For thirsty skin in need of added hydration. With eyes covered, spray directly over the face or use a cotton round or esthetic wipe to apply.
SERUM 1	Prosana Cleanse + Exfoliate	<ul style="list-style-type: none"> Turn on your AquaBoost equipment. Using the AquaBoost Serum 1, cleanse and exfoliate the skin. <p><i>*Please refer to the Owners Manual for usage instructions.</i></p>
SERUM 2	Prosana Deep Cleanse	<ul style="list-style-type: none"> Serum 2 is optional Perform another pass on the skin with Serum 2 to address excess sebum, or congestion. Commonly used in the T-Zone, around the mouth, chin and jaw, and around the nose. <p><i>*Please refer to the Owners Manual for usage instructions.</i></p>
SERUM 3	Prosana Soothe & Hydrate	<p>Turning the suction back down, and the serum flow up, hydrate the skin using Serum 3</p> <p><i>*Please refer to the Owners Manual for usage instructions.</i></p>

Details Protocol

AquaBoost Signature Facial



STEPS	PRODUCT NAME	INSTRUCTIONS
MASK	Select Prosana Sheet Mask of your Choice	<p>Apply a Prosana mask that best suits your guests needs and leave on for 8 minutes.</p> <p>Recommended masks include:</p> <ul style="list-style-type: none"> • Hyaluronic Acid Sheet Mask • 4-in-1 Full Face Crystal Collagen Gel Mask • Vitamin C Crystal Collagen Gel Mask • Neck Lift Crystal Collagen Gel Mask
HAND MASSAGE	Prosana Nail & Cuticle Oil	While your client is masking, perform a hand and nail massage.
REMOVE MASK		Gently remove the sheet mask.
CRYO WAND	Apply additional serum from within the Prosana Mask pouch .	<ul style="list-style-type: none"> • Feel the skin to determine if enough product remains on the skin to provide a slip for the Cryo Wand. If the skin is tacky, apply remaining serum from the mask package to the skin with hands for facial brush. • Using upward, circular motions, glide the Cryo-Wand over the skin moving from inside to the outside of the face. Spend extra time on any area of the skin that required additional extractions. • Do not make direct contact with the eyes.
MOISTURIZE	Prosana Calming Balm	If skin is in need of additional moisture, apply Calming Balm to the face, neck, and décolleté.