

Treatment Preparation

AquaBoost Facial



PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME
PSN130	Serum 1: Cleanse & Exfoliate
PSN131	Serum 2: Deep Cleanse
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves
PSN126	Prosana Refining Cleanser
PSN127 OR PSN123	Prosana Refining Mist Prosana Hydration Mist
MASK OF YOUR CHOICE	Suggestions: <ul style="list-style-type: none"> • Hyaluronic Acid Sheet Mask (HM4606) • 4-in-1 Full Face Crystal Collagen Gel Mask (HM4551) • Vitamin C Crystal Collagen Gel Mask (HM4569) • Neck Lift Crystal Collagen Gel Mask (HM4564)
PSN116	Prosana Nail & Cuticle Oil
PSN129	Calming Balm

CONTRAINDICATIONS
<ul style="list-style-type: none"> • Open lesions on the skin. • Active Rosacea flare-up. • Pregnant & nursing women should consult with a physician. • Accutane or other skin-thinning medications or topical. • Injections within the last 7 days. <p>**Always follow your company guidelines on who should and should not receive a treatment**</p>

SERVICE TIPS
<ul style="list-style-type: none"> • It is strongly recommended to wear gloves during the equipment use portion of this protocol. • Adjust your suction and flow settings according to your guests needs.

POST TREATMENT INSTRUCTIONS
<ul style="list-style-type: none"> • Wear Sunscreen Daily • Resume your normal skin care next day • Avoid exfoliation for the next 7 days.

HEMOCARE RECOMMENDATIONS
<p>It is recommended guests go home with or replenish the following to maintain results</p> <ul style="list-style-type: none"> • Cleanser • Purifying and/or exfoliating product • Moisturizer

Quick Reference

AquaBoost Signature Facial



STEPS	PRODUCT NAME	INSTRUCTIONS
CLEANSE	Prosana Refining Cleanser	
TONE	Prosana Refining Mist OR Hydrating Mist	Select the Prosana Refining Mist or Hydrating Mist based on your guests skin analysis.
SERUM 1	Prosana Cleanse + Exfoliate	<i>*Please refer to the Owners Manual for usage instructions.</i>
SERUM 2	Prosana Deep Cleanse	<ul style="list-style-type: none"> • Serum 2 is optional and commonly used in the T-Zone, around the mouth, chin and jaw, and around the nose. • Can also be used as a second pass for deeper exfoliation. <i>*Please refer to the Owners Manual for usage instructions.</i>
SERUM 3	Prosana Soothe & Hydrate	<i>*Please refer to the Owners Manual for usage instructions.</i>
MASK	Prosana Sheet Mask	Select Prosana Sheet mask of your choice.
HAND MASSAGE	Prosana Nail & Cuticle Oil	While your client is masking, perform a hand and nail massage.
REMOVE MASK		Gently remove the sheet mask.
CRYO WAND	Apply additional serum from within the Prosana Mask pouch .	Ensure skin has enough slip while using the CryoWand
MOISTURIZE	Prosana Calming Balm	Only needed if skin is in need of additional moisture.