

KLOwen™ Rebonding Guide

Intelligent Custom Orthodontics



Scan for KLOwen
Bracket Guide

Step 1: Identify the Correct Bracket matches between Bracket Rx and Bracket Kit.

7M =  **7M**


Example: UR3
= 7M Bracket

Clear
Twin

Metal
Twin

Molar
Tubes

Mini
Tubes

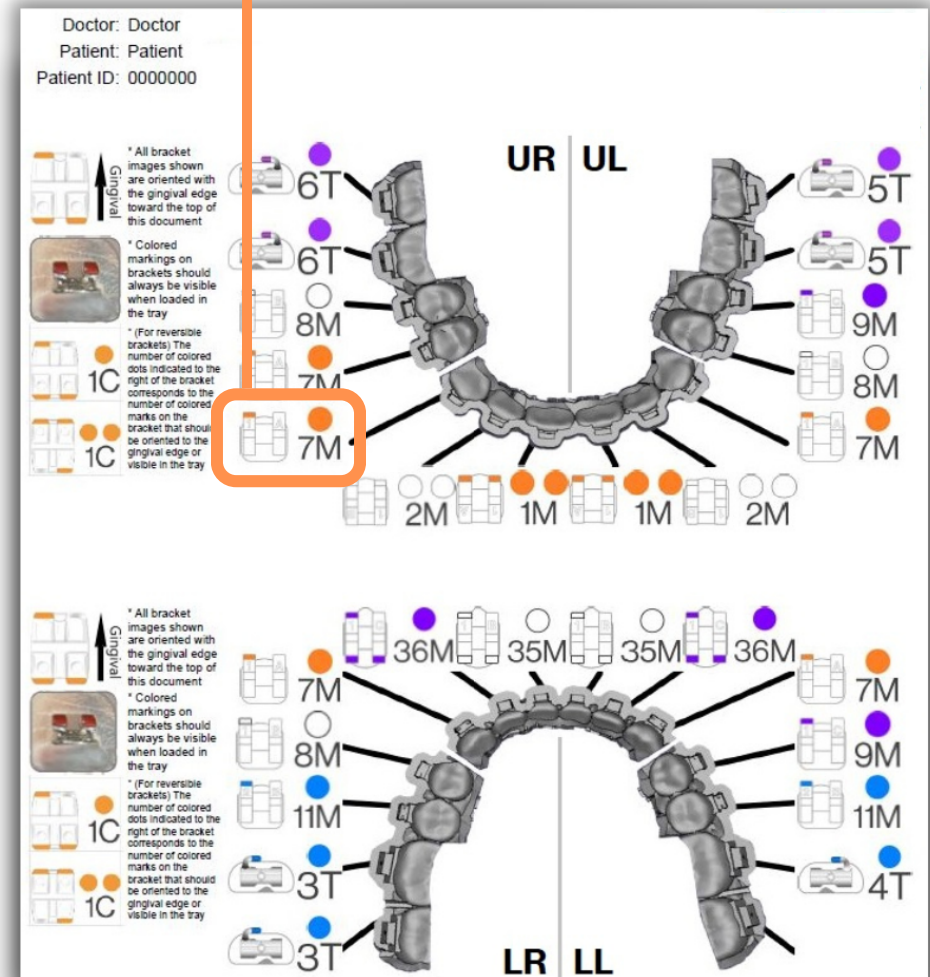

Sample Bracket Rx

Upper Incisor Brackets

- Metal: 2 colored wings face gingival out of the tray.
- Clear: Reversible; refer to Bracket Rx for orientation.

Lower Incisor Brackets

- Metal: Reversible; refer to Bracket Rx for orientation.
- Clear: Reversible; refer to Bracket Rx for orientation.

Cuspid-Bicuspid Brackets

- 1 colored wing faces gingival out of the tray.
- Metal Hook Options: no hook, left hook, or right hook.
- Clear Hook Options: no hook or universal hook.
 - Hooks currently only available for canines.
 - Refer to Bracket Rx for identifying the correct hooked bracket.

Molar Tubes

- Colored hook sits on mesial end of the tube.
- Please note, the same prescription molar tube can be used for 2 molar teeth (UR/LL, and UL/LR).
- Refer to Bracket Rx for identifying the proper molar tube.


IMPORTANT: Bracket Images on Rx always show gingival pointing UP as if looking at the IDB trays flat side down.

Check Bracket Rx if 1 or 2 painted wings face up.




1 Color Dot

= 1 painted wing points gingival; signifying negative torque.

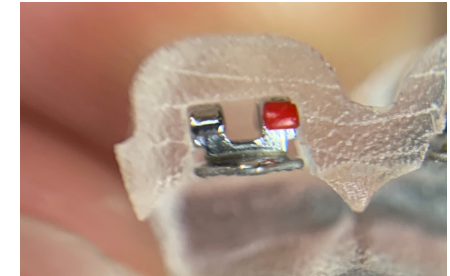


2 Color Dots

= 2 painted wings point gingival. signifying positive torque.

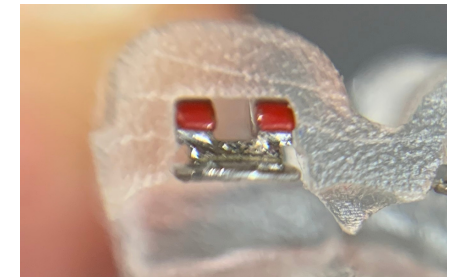

Step 2: Cut the IDB Tray

- Use an X-ACTO knife or ligature cutters to cut out a single-tooth jig, starting between the bracket boxes.
- Use a sharpie to label the correct tooth number on the jig.

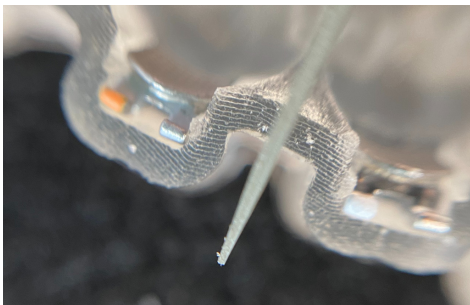

Step 3: Load Brackets into Tray, then Bond

- Grip the gingival end of the bracket pad (the same end as the colored wings).
- Seat the bracket all the way back and down into the jig.
- Proceed to rebond, following KLOWen Bonding Protocol.

