Household Oxygen Generator User Manual

Manufacturer: Shenzhen dafangya Technology Co., Ltd
Address: 401, building 4, South 1, DaFa Industrial Zone, Longxi community,
Longgang street, Longgang District, Shenzhen

Please read the instructions carefully before use.
*All languages are subject to English, and the manufacturer reserves the
right of final interpretation.

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Preface

Dear users,

This manual introduces the functional features, operation steps, precautions and maintenance of the product in detail.

To ensure that you can use this product more effectively, please read the manual carefully before use and keep it well for future reference.

Users shall not disassemble the product without permission, otherwise you might lose your right to enforce the warranty.

Please note that our company reserves the right to update our products and technologies. Some illustrations in the manual are for reference only and may differ from the actual product. Please refer to the actual product.

Safety Precautions

- ▶ This product is not intended for life support or life prolongation. Oxygen users in urgent need should use alternative backup oxygen supply devices, such as an oxygen cylinder. Not suitable for newborns or infants.
- ▶ Oxygen can support combustion. Do not smoke while using the equipment!
- ▶ Avoid using the equipment near high temperatures, sparks, or open flames.
- ▶ For safety, children should not move the equipment alone.
- ▶ When using oxygen, avoid wrapping the hose around your neck.
- ▶ Elderly users who have difficulty moving should be accompanied by a guardian.
- ▶ Do not move the equipment by pulling the power cord or hose.
- ▶ During operation, oxygen is extracted directly from the air. Do not use it in environments with smoke or poisonous gases.
- ▶ This product cannot be connected with equipment from another manufacturer.
- ▶ Contact the supplier first if any accessories are damaged or lost.

Product Introduction

1. Working Principle

Equipped with a pure copper oil-free compressor and a high-quality French molecular sieve, it uses air as raw material to produce high-purity oxygen through a new Pressure Swing Adsorption method (PSA) at room temperature. This is a totally physical method which is stable, safe and reliable.

2. Product Applications

This product is designed for health care and cosmetic oxygen supplementation. Oxygen inhalation can effectively improve the oxygen supply of the human body, thus achieving the purposes of oxygen supplementation and health care. The product is suitable for middle-aged and elderly people, pregnant women, people with weak constitutions, students, office workers and other people with different degrees of physiological hypoxia. It is also suitable for those who need to eliminate fatigue after heavy physical or mental exertion and quickly restore body functions.

3. Product Structure

- Full plastic shell for safety and reliability;
- Compact and lightweight design for easy portability;
- Features a usage time accumulation function to keep track of the machine's operation time.

Technical Parameters

- 1. Model:** DF-010
- 2. Oxygen Flow:** 1-6 L/min
 - The oxygen concentration is >90% at 1 L/min oxygen flow
 - The oxygen concentration is 65-30% at 2-6 L/min oxygen flow
- 3. Overall Noise:** <45dB(A)
- 4. Power Supply:** AC110V/60Hz (Please use the power cord provided with this machine)
- 5. Input Power:** ≤110W
- 6. Net Weight:** Approx. 4.2kg
- 7. Dimensions:** 240x155x255 mm
- 8. Altitude:** The oxygen concentration will not decrease from sea level to 1828 meters, but the efficiency will be less than 90% from 1828 meters to 4000 meters
- 9. Safety System:**
 - Shutdown for current overload protection
 - Alarm for loose power cord connection
 - Shutdown for compressor overheat protection
- 10. Normal Operating Environment:**
 - Temperature Range: +5°C~+40°C
 - Relative Humidity Range: <80%RH
 - Atmospheric Pressure Range: 860hPa~1060hPa
- 11. Storage and Transportation Environment:**
 - Temperature Range: -10°C~+55°C
 - Relative Humidity Range: 10%~93%, without condensation
 - Atmospheric Pressure Range: 700hPa~1060hPa

Note: Store the oxygen generator in a well-ventilated room without strong sunlight, or corrosive gas inside. Avoid severe shock or turning the product upside-down during transportation. If the storage and transportation temperature is below 5°C, store the machine in a normal operating environment for more than 4 hours before use.
- 12. Electrical Classification:** Class I
- 13. Operating Mode:** Continuous operation
- 14. Waterproof Rating:** IPX0
- 15. Oxygen Outlet Temperature:** <46°C

Note: The oxygen tube should not be longer than 5 meters and should not be folded.

Installation and Use

1. Package Inspection

Before installing and operating the product, please check whether the appearance of the product is intact, and whether the numbers of accessories are consistent with the packing list in the manual. In case of any defects, please contact us promptly.

2. Product Appearance

Figure 1-Front side view

Figure 2-Rear side view

Installation and Use

3. Operation Panel Introduction

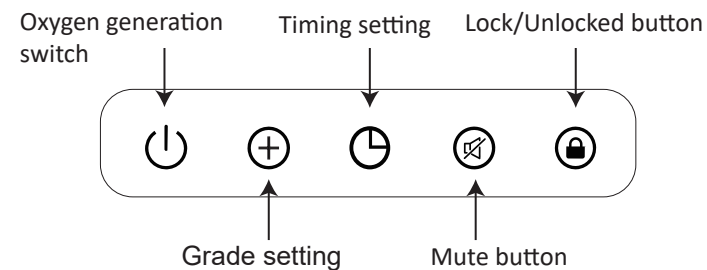
Figure 3 - Operation panel diagram

A.Buttons and functions

- **Oxygen generation switch:** To start/stop oxygen generation.
- **Grade setting:** To adjust the output oxygen flow. There are 6 adjustable levels in total.
- **Timing setting:** To set the duration of each use. Each press increases the time by 30 minutes and the duration is adjustable from 0~240 minutes.
- **Mute button:** To turn on/off the voice.
- **Locked/Unlocked button:** To lock/unlock the buttons on the operation panel to prevent accidental activation of buttons.

4. Display Introduction

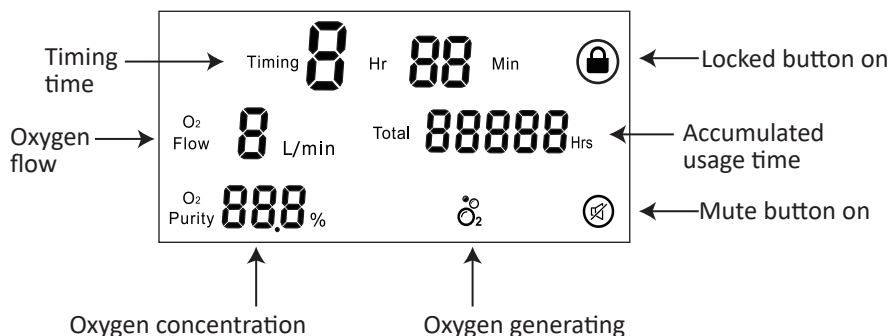
Figure 4 - Display diagram

5. Use Guidance

A. Connect the power supply

First, make sure the power switch of the oxygen generator is turned off ("O"). Connect one end of the power cord to the power connector on the oxygen generator, and the other end to the AC110V output socket which is safely grounded. Please use the supplied power cord and do not use an power extension cord.

Note: If the machine is to be used in a vehicle, you need to purchase a vehicle inverter separately.

Figure 5 - Power connector diagram

B. Connect the disposable oxygen inhaler

1. Press the power switch to "I". Then press the oxygen generation switch on the operation panel to start normal operation.

Note: When the oxygen generator is operating, it will make a "Chi~" sound every few seconds, which is the normal ventilation and exhaust sound.

2. Press the grade setting button to set the required oxygen flow.

Note:

The oxygen concentration reaches a stable value after 30 minutes of operation.

Please follow the doctor's advice regarding oxygen inhalation time and oxygen flow.

3. Press the timing setting button to set the oxygen inhalation time. Each press increases the time by 30 minutes, up to a maximum of 4 hours. When the timing time is displayed as ":-", the machine is in continuous operation.

4. Securely connect one end of the oxygen inhaler tube to the oxygen outlet of the oxygen generator and wear the other end as shown below (Figure 6) to start oxygen inhalation.

- Wear the oxygen inhaler as shown in the picture, adjust the nasal cannula to the appropriate position, and push the fixing ring up to fix it after wearing.
- Note that the tube of the inhaler should not be kinked or blocked, so as not to affect normal use of the device.
- If you need to purchase another oxygen inhaler, please contact the equipment provider or professional medical staff.

Figure 6- Disposable oxygen inhaler connection

C. Stop the operation

If you need to stop supplying oxygen during use, please press and hold the oxygen generation switch until the countdown on the operation panel ends, and the machine will stop working.

After use, unplug the oxygen inhaler from the oxygen outlet of the machine, then unplug the power cord from the machine, and finally cut off the power supply.

Maintenance

1.Clean the Shell

The outside of the machine should be wiped clean at least once a month. Cut off the power supply and then wipe it with a slightly damp cotton cloth or sponge.

Be careful not to seep liquid into the cracks of the chassis.

After cleaning, place the machine in a ventilated place for about 2 hours, and wait until the surface is completely dried before using.

2.Replace the Air Intake Filter Cotton Regularly

To ensure the cleanliness of the compressor air intake pipe, please check the air intake filter cotton frequently. If the filter cotton absorbs a lot of dust, please replace it. Generally, replace it every 2 months. The original filter cotton can be soaked and cleaned with edible detergent and then dried. (Do not expose it to the sunlight!)

Filter cotton replacement: Use a screwdriver to unscrew the fixing screw of the filter cover at the bottom of the machine to remove the filter cover. Then remove the filter cotton inside and replace it with a new one. Finally, fix the filter cover with the screw again.

The following figure is a schematic diagram of filter cotton replacement (Figure 7):

Figure 7 - Filter cotton replacement diagram

Problem and Solutions

Malfunction	Problems	Solutions
The machine cannot be started	Poor connection of the power cord	Check power connection
	The socket voltage does not match	Select output voltage as 110V, 60Hz
	The machine broke down	Contact the seller
No oxygen output	Tube of the oxygen inhaler is knotted or blocked	Replace it with a new oxygen inhaler
	The oxygen inhaler is not tightly connected to the equipment, causing air leakage	Reinstall the inhaler correctly
	The machine broke down	Contact the seller
There is an odor in the oxygen	The oxygen inhaler has not been cleaned for a long time, resulting in dirt in the tube	Clean or replace in time
	There will be a slight smell of disinfection in the initial use of the product, which is anormal phenomenon	The smell will disappear automatically after a few days of use
Regular and slight sound inside the machine	The nitrogen emission sound produced by the machine during oxygen production	This is the normal sound of the machine as it works to produce oxygen and expel nitrogen
After the machine has been turned on for a while, its temperature is high or it stops working	Air intake and exhaust vents are blocked	Place the machine in a well-ventilated location indoors, 10 cm away from walls, tapestries, furniture or other objects
	The air intake filter cotton is dirty	Check whether the air intake filter cotton on the rear shell of the machine is blocked or dirty. If so, clean it in time
	The machine temperature is too high	The machine has high temperature protection. If the temperature is too high and the machine stops automatically, turn off the power switch and check whether the air intake and exhaust vents are not blocked and whether the filter cotton is not dirty. Wait for the machine temperature to cool down, then restart the machine
If the above solutions still do not handle the malfunction, please contact after-sales service in time		

Packing List

No.	Item	Quantity
1	Oxygen Generator	1
2	AC Power Cable	1
3	User Manual	1
4	Disposable Oxygen Inhaler	1
5	Air Intake Filter Cotton	1

Certificate

Certificate

Product Name Household oxygen generator

Inspector _____

Productive dates See fuselage

After-Sales Service

- If this product has any quality problems not caused by human factors within one week from the date of purchase, our company will be responsible for return, replacement and repair.
- From the date of purchase, if the product of our company fails due to the quality problems of the product itself under normal use, this card allows you to enjoy one year free warranty and lifetime maintenance.
- The machine shell and external accessories are not included in the free warranty coverage of this warranty card.
- The warranty period of the oxygen concentrator produced by our company is one year from the date of sale. The subsequent maintenance service will be charged only the cost.
- The right to interpret the relevant terms of this "Service Commitment" belongs to our company.
- The following situations are not covered by the warranty:
 1. Fragile and consumable items, such as disposable oxygen inhalers, filter cotton or filters;
 2. Water, medicine, or other liquids spilled into the machine by the user, causing the machine to malfunction;
 3. The entire machine is flooded or exposed to rain;
 4. Failures caused by unauthorized disassembly, repair or modification of the product;
 5. Failures caused by accidental dropping during use or transportation;
 6. Failures caused by not following the correct instructions in the user manual to operate the machine;
 7. Damage caused by unforeseen natural disasters (such as fire, earthquake, flood, etc.).

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