
NT-5 Portable Oxygen Concentrator user's manual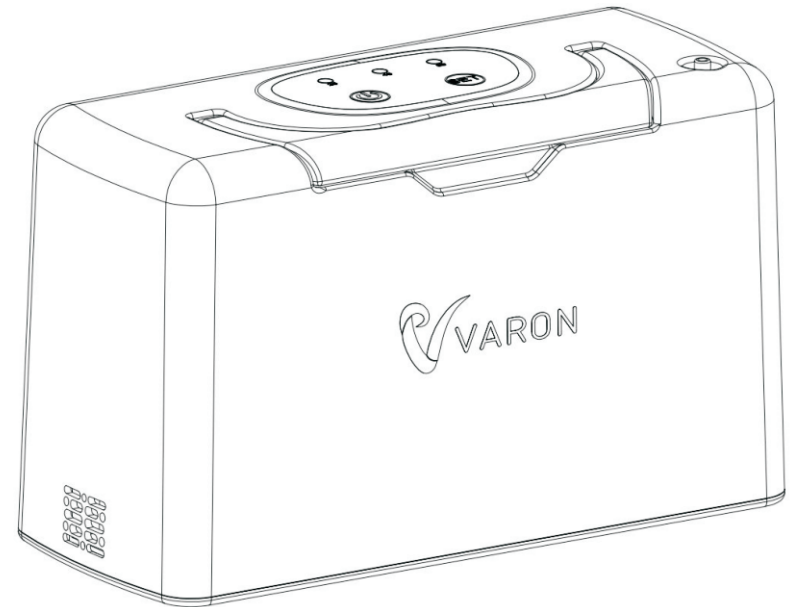

* All languages are subject to English, and the manufacturer reserves the right of final interpretation

Aerobic green life, start now

Catalogue(English) 01-06

Foreword	01
Features	01
Safety Precautions	01
Product parameters and specifications	02
Receipt Inspection	02
Getting Started with Oxygen Concentrator	03
Outline diagram	03
Use of oxygen concentrator	03-04
Precautions in use	05
Cleaning and Maintenance	05
Troubleshooting general issues	06

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Foreword

Dear customer, thank you for purchasing the NT-5 Portable Oxygen Concentrator, using it, there will be a healthy and friendly atmosphere surrounding lucky you, if you can tell us your opinion on the product and service, we will Thank you for your help.

Please note that the company reserves the right to update products and technologies. Some illustrations in the manua may be different from what you see in this product, so please refer to the actual product you have received.

Features

This equipment adopts safe and reliable PSA oxygen production technology, uses air as raw material and electric energy as the only energy source, and separates oxygen and nitrogen in the air through multiple filtration and molecular sieve adsorption to obtain oxygen; this method is a purely physical method, and the Oxygen is stable, safe and reliable.

The battery is separated from the host, which is convenient for users to replace the battery pack with a larger capacity or a smaller volume, and it is convenient to arrange and use it reasonably when going out.

Safety Precautions

1. This oxygen enrichment machine is used for health preservation and improving the quality of life, and cannot be used as a life extension or medical device.
2. Do not use the device without reading and understanding the contents of the manual, otherwise it may cause injury or damage to the product due to improper operation.
3. Oxygen is a combustion-supporting gas. Do not smoke, light candles or use other open flames when using this device.
4. When the battery is not in use, please pay attention to protect the battery interface and the battery interface on the device, and do not touch the conductor or directly with your hands.
5. The battery contains corrosive chemicals, which are flammable materials and may explode. Do not expose to heat sources such as open flames or sparks. Do not transport batteries with flammable, explosive or combustible materials. Do not put tools and other metal objects near the positive and negative terminals of the battery, if they touch the terminals, it will cause short circuit and electric shock.
6. It is forbidden to use adapters that are not equipped with the product or that do not meet the safety requirements of GB9706 and 1-2007 standards.
7. It is forbidden to use lithium batteries not equipped with the product.
8. Users are not allowed to disassemble the product without permission.

Product Parameters and Specifications

Product Name: Portable Oxygen Concentrator

Model: NT-5

Scope of application: It is only used for health preservation and improving the quality of life, not for medical treatment or life extension.

Oxygen concentration: 30%±2.5%

Oxygen production flow: 3L/MIN

Noise: 52dB

Input voltage: AC100V~240V / DC12V~16.8V

Power: 25W

Product size: length 23.5 * width 10 * height 14.1 CM

Product net weight: 1.5Kg

Standard battery pack capacity: 2500mAh 14.8V, maximum voltage 16.8V

Maximum battery usage time: 1.5h

Battery charging time: >1.5h

Receipt Check

Inspection equipment and accessories

NO.	Name	Qty	NO.	Name	Qty
1	Main machine	1unit	5	External battery	1pcs
2	AC Adaptor	1set	6	crossbody shoulder bag	1pcs
3	User manual	1pcs	7	Warranty Card/ Qualification Certificate	1pcs
4	Disposable oxygen inhaler	1pcs	8	Car line	1pcs

Please check the sufficiency according to the above list, if there is any missing, please contact your seller to make up.

Get started with an oxygen concentrator

Outline diagram

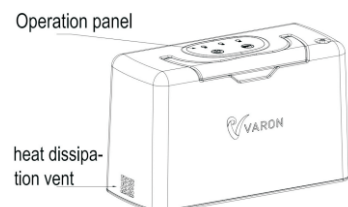
Figure 1--left side view



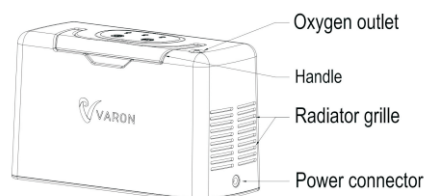
Figure 2--Right Side View

Use of oxygen concentrator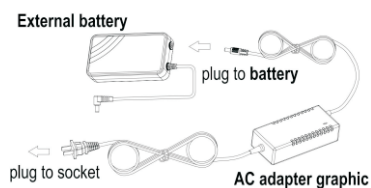


1. Charging of external battery

Connect one end of the AC adapter to AC100 ~ 240V AC, and the other end to the input end of the battery pack. The red light of the adapter indicates that it is charging, and the green light indicates that it is fully charged.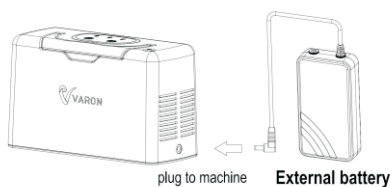


2. Connect external battery to use

Connect the output port on the right side of the battery pack to the power socket of the oxygen enrichment machine. At this time, the device will make a beeping sound, indicating that the device has been correctly connected to the power supply.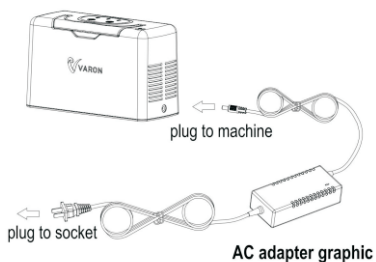


3. Connect AC adaptor to use

Plug the output port of the AC adapter into the power socket of the oxygen enrichment machine, and connect the other end to 100~240VAC. At this time, the device will make a beeping sound, indicating that the device has been correctly connected to the power supply.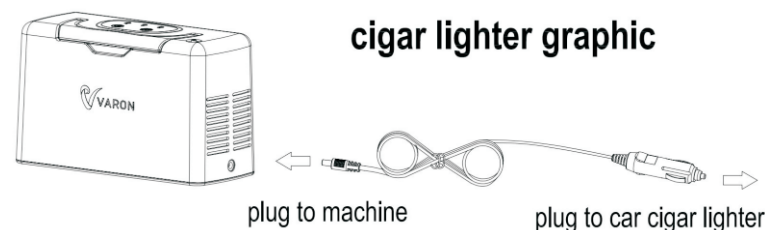

4. Connect the car power cord to the car to use


Connect the input end of the car power cord to the car cigarette lighter, and then plug the output end of the car power cord into the power socket of the oxygen enricher. At this time, the red light of the car power cord is on, and the device will have a "di". A single beep indicates that the device is properly connected to power.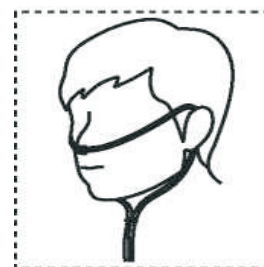

5. Connect the disposable inhaler

Connect the flare end of the disposable nasal straw firmly to the oxygen outlet of the device, then wear the oxygen inhaler as shown on the left, adjust the air outlet to a suitable position, and push the fixing ring upwards to fix it after wearing.

Be careful not to kink or block the tube of the disposable oxygen inhaler, so as not to cause equipment-related alarms and affect normal use.

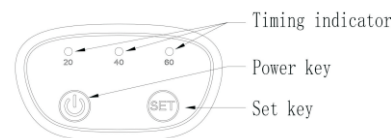6. Start the oxygen concentrator

Press the "power key" of the oxygen enrichment machine, you will hear a "beep", the device starts to work, and the default time is 20 minutes.

equipment-related alarms and affect normal use.

7. There are four time settings for the device. Press the "setting button" to select different times. According to the instructions, you can set four time levels: 20 minutes, 40 minutes, 60 minutes, and 120 minutes; Each time indicator is on, indicating the corresponding timing time; if three time indicators are on at the same time, the timing time is the sum of the three time numbers, ie = 20+40+60=120 minutes.

8. If you need to stop working during use, you can press the "power switch key", or it will automatically stop working after the timer expires.

9. After the equipment is stopped, if there is no operation command, it will enter the standby mode after five minutes; if you need to start the machine to inhale oxygen, you need to press the "power switch" once to activate the equipment, and the equipment will make a "beep" at the same time, and then press it again "Power switch key", the device can enter the working state.

Precautions in use

1. Properly store and place battery packs to prevent accidents and prolong product life;
2. Remind again that when using this product, do not smoke and keep away from open flames;
3. Do not use it as a toy for children, and place it out of reach of children to prevent accidents from children playing;
4. The equipment should be at least 10CM away from other objects or walls to ensure the air inlet and outlet are unobstructed;
5. Due to transportation control, the external battery only retains the test power when leaving the factory, or there is no power at all. Therefore, the battery needs to be fully charged before the first use.
6. When the input end (i.e. charging port) of the external battery is not in use, the rubber cap beside it should be fastened to prevent short circuit damage to the battery or accidents.

Cleaning and maintenance

1. Before cleaning the equipment, please unplug all external devices: such as AC adapter, external battery, oxygen inhaler, and ensure that the equipment is turned off.
2. Do not let the equipment get wet or let water enter the equipment. If water enters the equipment accidentally, please stop using it immediately and contact the dealer to deal with it;
3. If the equipment is not used for a long time, please wrap the main unit in a plastic bag and seal it to prevent the performance of the machine from being degraded due to damp air.
4. Please disconnect the power supply and battery in time after each use of the device.

❖ Troubleshooting general issues

NO.	Phenomenon	Problem	Possible reason	Solution
1	Not working	No power	Power failure	Replace a power supply or replace the battery pack
2			Charger broken	Replace a power supply or replace the battery pack
3			No AC	Change a power strip
4	Can't turn on	Internal machine failure	Internal damage	Contact the agent or manufacturer for processing.
5	The first light flashes slowly	The power supply is abnormal	Incorrect or damaged power supply	Change to the correct power supply
6	The first light flashes quickly	The first light flashes quickly	Incorrect or damaged power supply	Change to the correct power supply
7	The second light flashes slowly	high internal temperature	Poor heat dissipation	Smooth the air inlet and outlet of the machine and wait for the temperature of the machine to drop to normal.
8	The second light flashes quickly	Ambient temperature is too low	The working environment temperature is too low	Put the machine in an environment higher than 5 degrees Celsius for 4 hours before use.
9	breath-like rhythm	working sound	~	~

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