



Windows

Follow the steps below to set up your new GR251U microchip reader. If you have any questions, feel free to contact us.

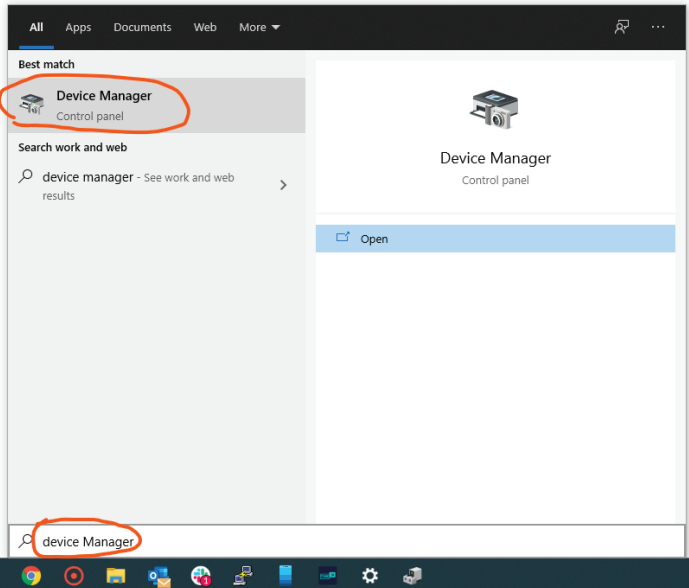

Step 1:

- Connect the communication cable to your computer's USB port and wait till the drivers are installed.
- Make sure you have administrator access on the computer you are installing the drivers.
- You can verify if the drivers are installed properly under 'Device Manager'
- To access device manager Hit Star → Type 'Device Manager'.

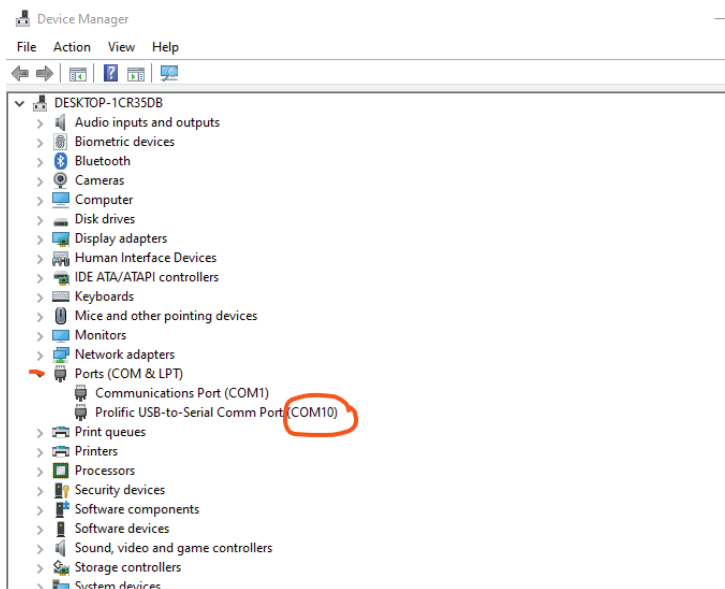

Step 2:

- Open Device manager following the previous step and select 'Ports' and make note of the COM number next to Prolific-USB-Serial Port.

Software Link

Software Manual



Step 3:

- Download the software from Microchips Australia website here <https://microchips.com.au/pages/gr250-support>
- Unzip the file and run the software installation file and follow the steps to complete the installation.




Step 4:

- Run the software installation file and follow the steps to install the program.
- Once installed open the 'GR25X Download Tool'.


Windows

Follow the steps below to set up your new GR251U microchip reader. If you have any questions, feel free to contact us.

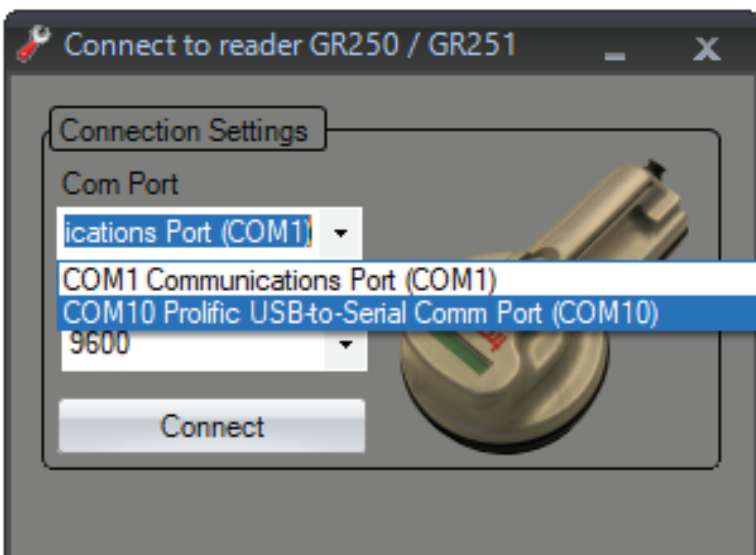


Step 5:

- From the 'COM Port' drop down list, select the COM number you made note of under device manager and click 'Connect'.
- Allow up to 30 seconds for the software to make a connection with the reader.

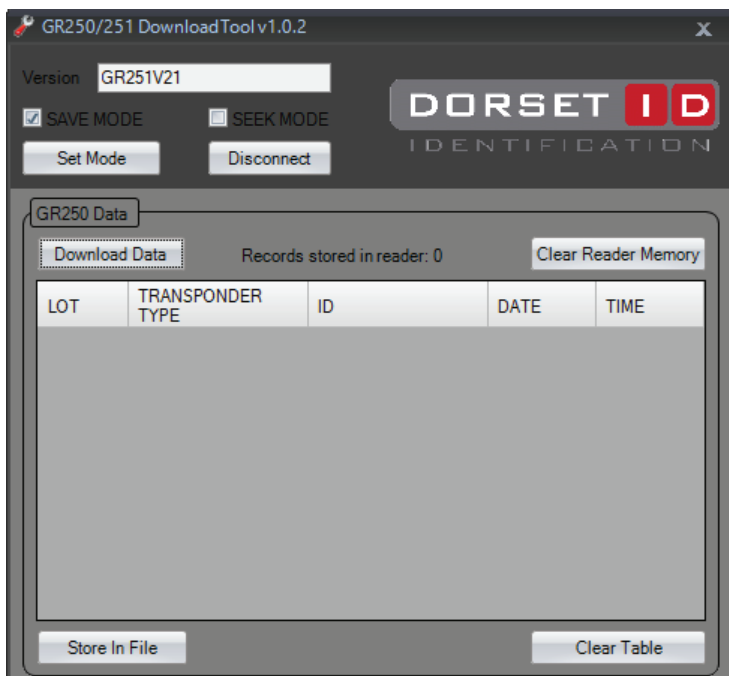

Step 6:

- After connection is established, you will see the following screen.

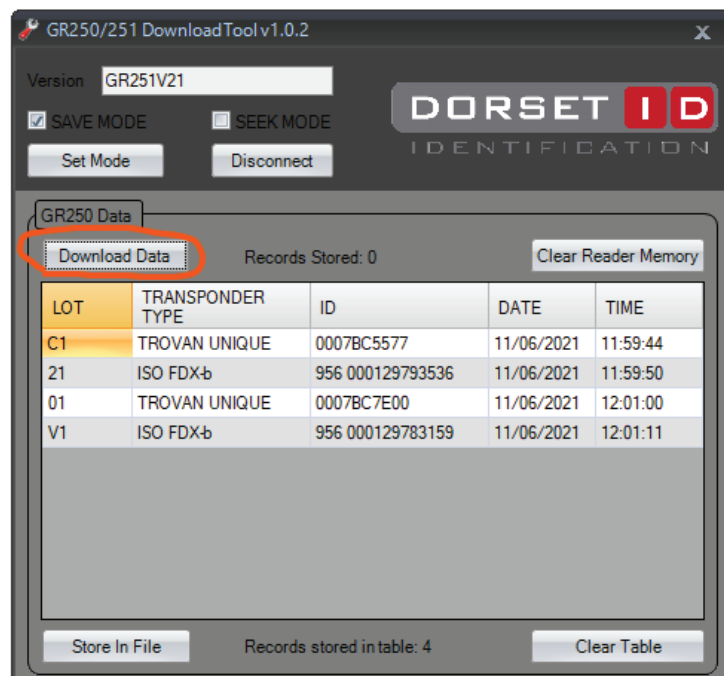


Step 7:

- To download data from the reader, Select 'Download Data' button.
- All the saved transponder codes will appear on the box below.

Windows


Follow the steps below to set up your new GR251U microchip reader. If you have any questions, feel free to contact us.

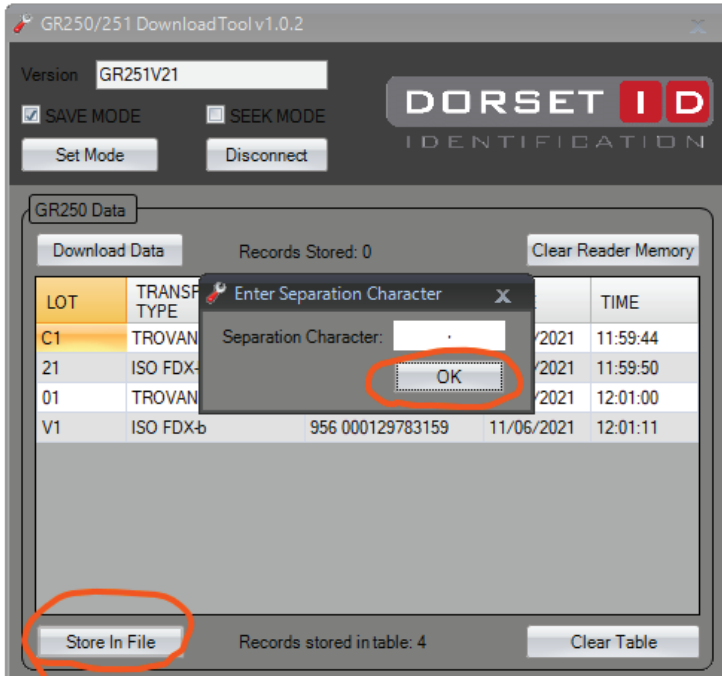

Step 8:


- To save this data to a file, select 'Store in File' button and 'OK' on the next screen.
- Choose where you want to save the file and give the file a name and save file.

You can open the saved .csv file using Microsoft excel.

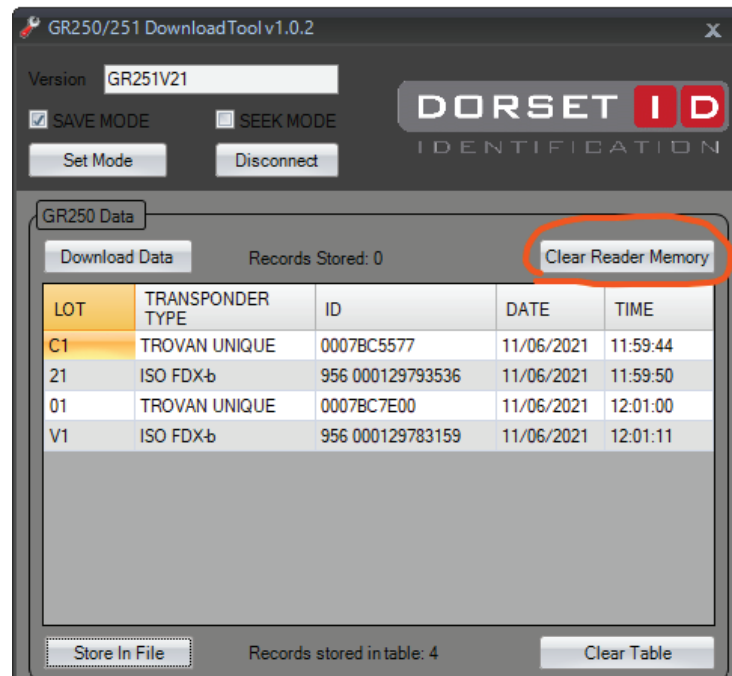

Step 9:

- After you have saved the file, you can choose to clear the memory by selecting 'Clear Reader Memory'.

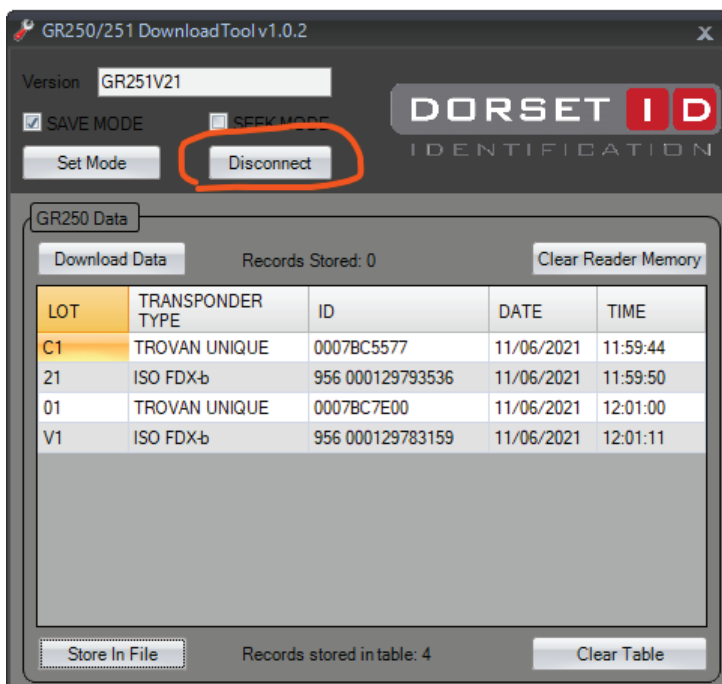

Step 10:

- Make sure you disconnect the software before removing the communication cable.