
glad skin

Inflammatory Skin Expert

ECZEMACT Cream & Gel

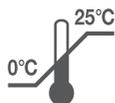STAPHEFECT™
100
Sa
S. aureus killer

Micreos Human Health B.V.
Bankplein 2
2585 EV Den Haag
The Netherlands

Gladskin BV
Bankplein 2
2585 EV Den Haag
The Netherlands

UKRP:
Advna Ltd
Pure Offices, Plato Close,
Tachbrook Park, Warwick,
CV34 6WE
United Kingdom

INS-GE-ENNLDFRSE-V8.2
REVISION DATE MAY-2023

ENGLISH
Gladskin Eczemact Cream and Gel are medical devices. Please read this information carefully before using the product, even if you have used it before. That way, you will keep up-to-date on the effects of this product and how to use it properly as we may sometimes need to change the information. Please do not hesitate to contact us if you have any questions after reading this information.

FOR WHOM ARE GLADSKIN ECZEMACT CREAM AND GEL INTENDED?

Use Gladskin Eczemact Cream and Gel if you suffer from eczema. The symptoms of eczema include dry, rough and flaky skin with redness and itching. The *Staphylococcus aureus* bacteria can cause inflammation of the skin and worsen the eczema symptoms.

TOLERANCE

This formula is fragrance-free and hypoallergenic. It is suitable for face, eyelids and body.

HOW DOES THIS PRODUCT WORK?

Gladskin Eczemact Cream and Gel contains the main ingredient Staphfect™ SA.100. Stemming from Microbiome Science, this unique ingredient has the ability to specifically target the main bacteria responsible for inflammation of the skin, while preserving the other bacteria essential for skin health.

WHAT DOES THIS PRODUCT DO?

- Reduces eczema symptoms
- Anti-itching
- Anti-red patches
- Moisturises very dry skin

HOW TO USE THIS PRODUCT

 > For external use. Use the product on intact skin that is free of creams, cleansers, powders and so on.

 > Use regularly two to three times a day.

 > Apply a thin layer of the product on and around the areas with eczema.

> Let the product dry for at least 5-10 minutes before you apply any other products on your skin.

When using the Gladskin Eczemact Gel you may notice a thin film left on your skin. This is remaining gel which can be wiped off after 10 minutes.

WHEN NOT TO USE THIS PRODUCT

Do not use this product if your skin is not intact or if you are known to be allergic to any of the ingredients.

Gladskin Eczemact Cream should be a white to off-white cream. Gladskin Eczemact Gel should be a clear gel, free from visible particles. If this is not the case, do not use the product and contact us.

WHAT CAN YOU EXPECT?

Gladskin Eczemact Cream and Gel can have a quick calming effect, but it can also take a few weeks to see results. Often eczema symptoms are related to an imbalance in the bacterial composition of the skin (microbiome). If your eczema symptoms are not due to this imbalance Gladskin Eczemact Cream will still act as an emollient. Some users may experience redness, a burning sensation, swelling or itch after using Gladskin Eczemact Cream and Gel.

If you use Gladskin Eczemact Cream and Gel for the first time your eczema symptoms may temporarily increase. This should improve within 10 days.

If your eczema does not improve or side-effects are severe or persist, please stop using the product and consult your doctor.

In case of serious side-effects, please inform us and the national competent authority of the country where you reside.

INGREDIENTS

Gladskin Eczemact Cream:
Aqua, Glycerin, Vaseline® album, Cera cetomacrogolis emulsificans, Paraffinum liquidum, Arginine HCl, Sodium chloride, Trometamol, Staphfect™ SA.100, Calcium chloride, *Gladskin Eczemact Gel:*
Aqua, Propylene glycol,

Hydroxypropyl methylcellulose, Glycerin, Arginine HCl, Sodium chloride, Trometamol, Staphfect™ SA.100, Calcium chloride.

Gladskin Eczemact Cream and Gel are hypoallergenic and dermatologically tested. It is a Medical Device to be used on intact skin. Gladskin Eczemact Cream and Gel do not contain any lanolin, acids, alcohol, perfume or parabens.

SHELF LIFE AND STORAGE

The expiration date of this product is indicated on the dispenser. For an optimal benefit of the main ingredient in Gladskin Eczemact Cream and Gel we recommend you store the product in the refrigerator. Gladskin Eczemact Cream and Gel should not be stored at temperatures below 0°C or above 25°C.

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