ROSACEAR Cream & Gel

STAPHIEFKT[™]
100
Sa
S. aureus killer

REDUCES ROSACEA SYMPTOMS

ANTI-REDNESS AND PUSTULES
ANTI-BURNING SENSATIONS
SOOTHES AND MOISTURISES

Antibiotic-free | Fragrance-free
Rosacea skin

Microeos Human Health B.V.
Bankaplein 2
2585 EV Den Haag
The Netherlands

Gladskin BV

Bankaplein 2
2585 EV Den Haag
The Netherlands

UKRP:
Advena Ltd
Pure Offices, Plato Close,
Tachbrook Park, Warwick,
CV34 6WE
United Kingdom

INS-GR-ENNLDEFRSF-V8.2
REVISION DATE MAY 2023

ENGLISH

Gladskin Rosacear Cream and Gel are Medical Devices. Please read this information carefully before using the product, even if you have used it before. That way, you will keep up-to-date on the effects of this product and how to use it properly as we may sometimes need to change the information. Please do not hesitate to contact us if you have any questions after reading this information.

FOR WHOM ARE GLADSKIN ROSACEAR CREAM AND GEL INTENDED?

Use Gladskin Rosacear Cream and Gel if you suffer from rosacea. The symptoms of rosacea include redness in the face (cheeks, nose and forehead), bumps, pimples, itching and a burning sensation. *Staphylococcus aureus* bacteria can cause inflammation of the skin and worsen the rosacea symptoms.

TOLERANCE

This formula is fragrance-free and hypoallergenic. It is suitable for face and body.

HOW DOES THIS PRODUCT WORK?

Gladskin Rosacear Cream and Gel contains the main ingredient Staphefekt[™] SA.100. Stemming from Microbiome Science, this unique ingredient has the ability to specifically target the main bacteria responsible for inflammation of the skin, while preserving the other bacteria essential for skin health.

WHAT DOES THIS PRODUCT DO?

Reduces rosacea symptoms
- Anti-redness and pustules
- Anti-burning sensations
- Soothes and moisturises

HOW TO USE THIS PRODUCT

- > For external use. Use the product on intact skin that is free of creams, cleansers, powders and so on.
- > Use regularly two to three times a day.
- > Apply a thin layer of the product on and around the rosacea prone areas.

> Let the product dry for at least 5-10 minutes before you apply any other products on your skin.

When using the Gladskin Rosacear Gel you may notice a thin film left on your skin. This is remaining gel which can be wiped off after 10 minutes.

WHEN NOT TO USE THIS PRODUCT

Do not use this product if your skin is not intact or if you are known to be allergic to any of the ingredients.

Gladskin Rosacear Cream should be a white to off-white cream. Gladskin Rosacear Gel should be a clear gel, free from visible particles. If this is not the case, do not use the product and contact us.

WHAT CAN YOU EXPECT?

Gladskin Rosacear Cream and Gel can have a quick calming effect, but it can also take a few weeks to see results. Often rosacea symptoms are related to an imbalance in the bacterial composition of the skin (microbiome). If your rosacea symptoms are not due to this imbalance you may not experience an improvement. Some users may experience redness, a burning sensation, swelling or itch after using Gladskin Rosacear Cream and Gel.

If you use Gladskin Rosacear Cream and Gel for the first time your rosacea symptoms may temporarily increase. This should improve within 10 days.

If your rosacea does not improve or side-effects are severe or persist, please stop using the product and consult your doctor.

In case of serious side-effects, please inform us and the national competent authority of the country where you reside.

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SHEL LIFE AND STORAGE

The expiration date of this product is indicated on the dispenser.

For an optimal benefit of the main ingredient in the Gladskin Rosacear Cream and Gel we recommend you store the product in the refrigerator.

Gladskin Rosacear Cream and Gel Should not be stored at temperatures below 0°C or above 25°C.

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