


Pawprint Oxygen empowers veterinarians and pet owners to administer **life saving oxygen** at home, in hospital, and in transport.

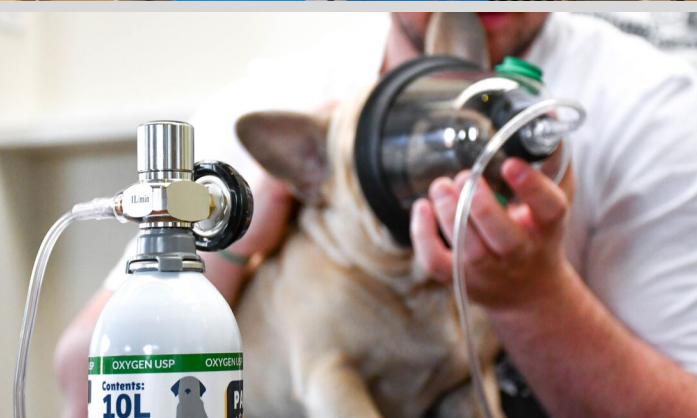

**PAWPRINT
OXYGEN**

www.VetMedSolutions.ca



**VETMED
SOLUTIONS**
**PAWPRINT
OXYGEN**

Portable Oxygen for Pets



Veterinarian Brochure



4391 Harvester Road, Unit 7
Burlington ON L7L 4X1

call: 1-888-976-8696

email: solutions@vetmedsolutions.ca

website: www.vetmedsolutions.ca

HOW IT WORKS

OFFER PAWPRINT

Rescue oxygen in the palm of your hand.

Saving lives at home, in hospital, and everywhere in between.

- Portable & Lightweight
- Cost Effective
- Shelf Life: 2 Years
- 100% Recyclable

Home Rescue

Dispense as a source of rescue oxygen to manage respiratory distress resulting from a variety of conditions including congestive heart failure, laryngeal paralysis, tracheal collapse, and brachycephalic syndrome.

Transport

Transport oxygen dependent patients between primary care, referral hospitals, and home. It's easy to use, with no more costly equipment or complicated returns.

Hospital

Use for triage and as a back-up for existing oxygen systems. Our portable canisters have a two year shelf life, making them perfect for quick and reliable oxygen relief in any situation.

Each canister provides 10 liters of Oxygen USP which will provide 5-20 minutes of oxygen therapy depending on the animal's weight and flow rate requirements.

1. INSERT Regulator into the top of canister
2. TWIST entire Regulator clockwise until it locks into place to begin oxygen flow
3. ATTACH tubing and mask to Regulator and place mask over pet's snout to begin therapy
4. STOP oxygen flow by twisting entire Regulator counter-clockwise
5. REPLACE the oxygen canister when the indicator on the Regulator reads "EMPTY"



INSERT

TWIST

BREATHE!

Distributed in cases of 12 exclusively by VetMedSolutions in Canada. Oxygen masks and regulators are available in any quantity.

Starter Kit

Contact us for pricing

- 6 Oxygen canisters
- 3 Oxygen masks
- 2 Regulators

Each mask includes 6ft of tubing.



Write a Prescription

Rx
Pawprint Oxygen
Flow Rate: 1.0 LPM

Administer for respiratory distress resulting from:

- Congestive Heart Failure
- Laryngeal Paralysis
- Feline Asthma
- Brachycephalic Syndrome
- Collapsing Tracheal Syndrome

If you don't want to keep stock, you can fill out a Prescription Approval Form on our website and send it to us. We will then call the pet owner, process the payment, and ship it.

A typical use case requires 3 canisters of oxygen, 1 Regulator, and 1 Oxygen Mask

Regulators come in 3 flow rates: 0.5, 1.0, 2.0 liters per minute

Use the chart on the canister to determine the correct Regulator based on flow rates and weight.

Oxygen masks come in 3 sizes:



Small

Medium

Large