
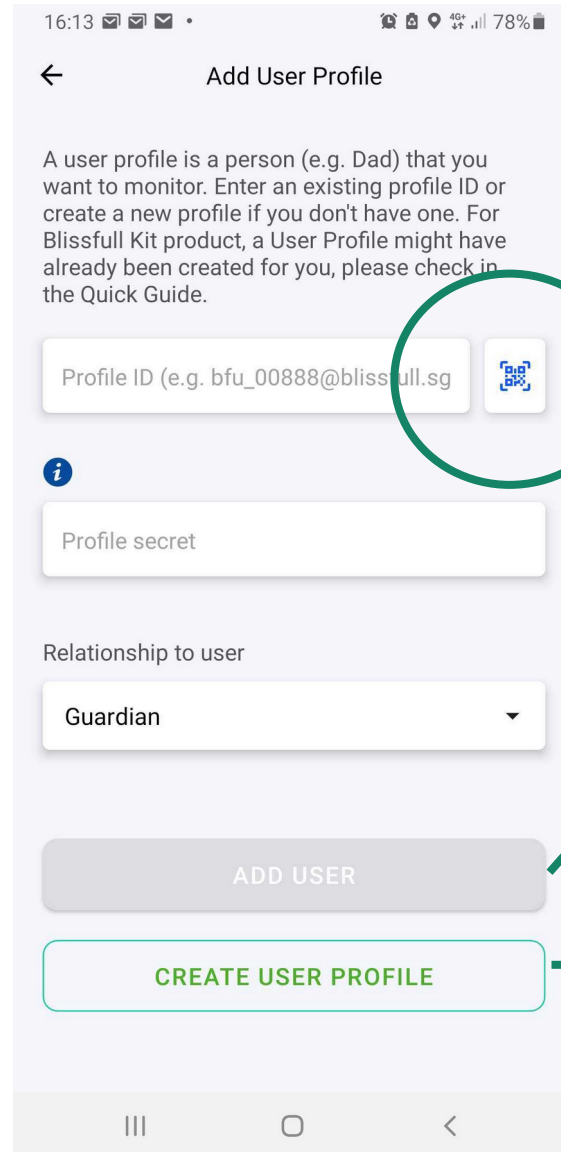


To Add New User (Elderly) Profile

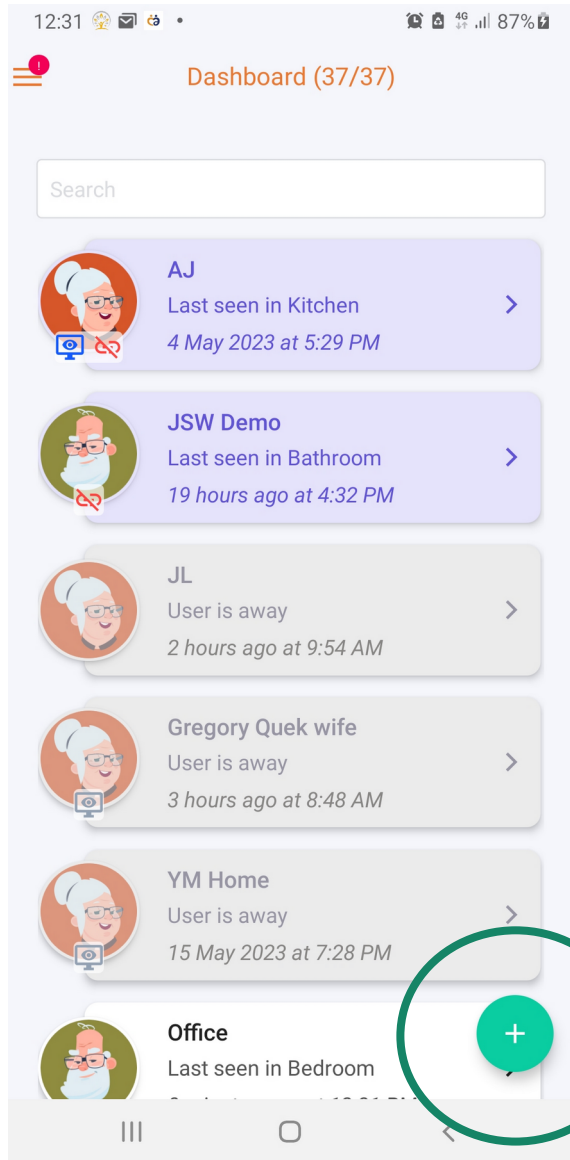


Scan the QR code as stuck on Blissfull kit box or get a family member to share the QR code. Refer to "Share User (Elderly) Profile".

If QR code is valid or right user Profile ID is entered, the button will be enabled.

Do not Create User Profile as User Profile is already created for every Blissfull Kit.

To Share User (Elderly) Profile

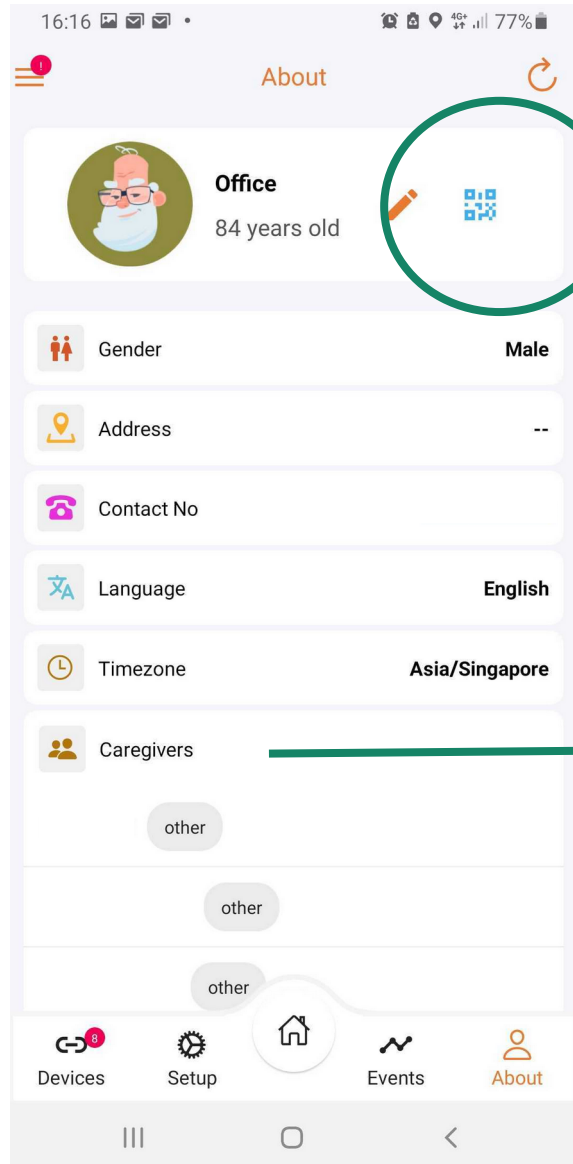

Tap on this QR icon in User Profile (About) screen and the QR code will popup as shown. For this sharing option, there is a validity period for this QR code. Also caregiver adding this user needs to input the 4-digit CODE as appeared in the QR code popup.

Caregivers monitoring this user (elderly) will appear in this list. Swipe to the left to remove the caregiver from the list.

