
ELITE ENGINEERED ELASTICTM ANKLE BRACE w/ figure-6 straps & stays



Pull the sleeve on with attention to position of ankle hones.




Position your lateral and medial ankle bones inside the c-shape of the support pad.

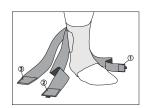

Attach strap #1 as shown. Tightly wrap the strap and attach it on the landing zone.



Attach strap #2 as shown. Tightly wrap the strap and attach it on the landing zone.

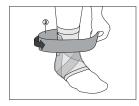

Wrap strap #3 around the lower shin, above the ankle.



Continue wrapping strap #3 with comfortable tension and attach it on the strap's landing zone.



Note: due to nature of knit and strap material, the best way to store product is to attach all attachment hook strap ends to designated areas to avoid hook attaching to and pulling on knit materials.