

Congratulations on your new DeBug Beach Walker

Safety And Instruction Manual



DeBug Mobility Products



78 E Nine Mile Rd
Pensacola, FL 32534

Corporate Address
Deming Designs, Inc.
1090 Cobblestone Dr.
Pensacola, FL 32514

www.beachwheelchair.com

850-478-5765

2

TABLE OF CONTENTS

Usage and Safety Information	3
General Maintenance Instructions	
Assembly, Using and Sitting Instructions	4
of DeBug Beach Walker	
Rear Axle Tip Assist	5
General Specifications and Warranty	6

Thanks for your purchase of the
DeBug Beach Walker

We build the DeBug Beach Walker with the highest quality materials, keeping in mind the comfort and positioning of the rider. If any repairs should be required, please do not attempt them without proper technical training. Please contact your nearest distributor for any repairs, or adjustments needed. All information in this manual is subject to change without prior notice. Please read the following few pages for Assembly and Maintenance Instructions. Thank-you The De-Bug Mobility Products Team. We love to hear and see how much you are enjoying your DeBug Beach Walker. Feel free to email us any pictures or videos, or post them to our website.

www.beachwheelchair.com

3

USAGE & SAFETY

DeBug Beach Walker should not be used during inclement weather, dangerous surf, rough ocean currents, rip tides and any other poor conditions. !

We recommended the walker be used only when a lifeguard is present. Safety comes first! Users must always be accompanied by a responsible, able-bodied adult.

Using your DeBug Beach Walker

Always maintain control of your DeBug while in use. When the DeBug is not in use, the safety brakes must be engaged to prevent the walker from moving unexpectedly. .

Do not use the components of the DeBug Beach Walker for any purpose other than what is intended. Do not have more than one person using the DeBug. Check all moving components before your DeBug Beach Walker

General Maintenance

It's so easy to maintain your DeBug Beach Walker ! Just follow these simple instructions, and you will be able to enjoy your chair year after year.

Rinse the walker off with clean water to remove any sand or debris after each use.

Check tire pressure before each use

If at any time you need to lubricate any part of the chair, DO NOT USE WD-40, as it will attract dirt and sand, causing possible damage to the wheel bearing/ bushings. We recommend P.T.F.E Dry Lube, all wearing components. This dry film is a superior lubricant and repels debris and water, marine and performance parts stores likely stock this locally.

Have any questions, or need additional support? Please don't hesitate to contact us!

4

Assembly and Use Instructions of DeBug Beach Walker



ASSEMBLING INSTRUCTIONS

- Open frame, and insert each handle (labeled "L" for left side, "R" for right side) into the frame of the rollator
- Attach backrest to frame by depressing the push buttons and inserting into hinges
- Fully depress each push button on all adjustable legs and slide up or down to the desired seat height
- User will hear an audible "click" when push buttons engage and lock.
- Recommended seat height allows user's feet to rest firmly on the ground.
- Adjust handle heights to the appropriate level using the aforementioned

HOW TO USE YOUR WALKER

- Hold the rollator upright and rest it on the rear wheels.
- Hold both handles and squeeze the handbrakes. Push down and forward on both handles until all four wheels are touching the ground. Push down on the bottom of the bag until the folding cross-bar is fully engaged.
- Hold the handles with your fingers resting around the handbrake in the squeezing position. This position will allow you to quickly stop the rollator or control your speed if necessary by squeezing the handbrake.
- Roll your rollator slowly forward so that the rear wheels are a few inches ahead of your body.
- Step forward and place one foot in line with the rear wheels
- Use Rear Wheel Axle Tilt when encountering uneven terrain



HOW TO SIT SAFELY

- Lock the brakes by pushing down on the brake handles until it locks in the downward position.
- Test the rollator before sitting down to make sure that it will not roll.
- Use the handles to stabilize your body as you turn around.
- Before sitting, touch the back of your legs to the edge of the seat to ensure proper positioning of the rollator.
- Holding the handles, slowly lower your body onto the seat
- To prevent tipping over when sitting Distribute weight evenly on the seat. DO NOT lean on one side of the rollator. Both feet should rest flat on the ground. DO NOT support your full body weight on the handles when attempting to sit down.

5



Rear Wheel Axle Tip

The wheel axle tip can be used to assist maneuvering the wheels over slightly uneven terrain.

Simply press one foot on the tip bar to elevate the front wheels slightly over the obstacle.

Please be aware of the following:

- The user may need to practice, or develop a technique to use this feature effectively.
- The user must also have enough balance and strength to use this safely, and independently.

Rear Wheel Axle Tip
Side View



Rear Wheel Axle Tip
Foot Placement

Warning

Walker must be in a stationary position, on a level surface with the brakes locked when sitting or attempting to sit. **DO NOT** scoot around or navigate the rollator when in the seated position. Risk of serious damage to the rollator and serious injury or death may result if brakes are not in the locked

6

GENERAL SPECIFICATIONS AND WARRANTY INFORMATION

30-Day 30 DAY

Money Back Guarantee

Less all shipping charges

Possible Restocking Fee

2-Year

Tires-3ply polyurethane, specifically designed to smoothly roll across sand, dirt, grass, and other uneven terrain

10-Year

Caster components #316L series stainless steel The higher the number the higher quality of stainless

The Lightweight aluminum frame, folds for transportation and storage

The DeBug Beach walker hand brakes have a locking mechanism, and adjustable and removable push handles.

It has a folding seat, back pad to sit, and includes a storage pouch
Stainless steel casters and axles are salt and rust resistant,, and include low maintenance delrin bushings,

The soft Pneumatic wheels roll across sand snow and gravel, with ease. Do Not Over Inflate. They should be squishy soft

The DeBug Beach walker fits through a 36inch ADA doorway

“Disclaimer: The ability to use this beach walker is dependent on the user’s health and strength. It is not intended to be used as a medical device or to replace professional medical advice. The user assumes all risks associated with the use of this product. The manufacturer is not responsible for any injuries or damages that may occur as a result of using this product. Please consult your doctor or physical therapist before using this product to ensure it is appropriate for your needs.”



78 E Nine Mile Rd
Pensacola, FL 32534

Corporate Address
Deming Designs, Inc.
1090 Cobblestone. Dr.
Pensacola, FL 32514

www.beachwheelchair.com

850-478-5765