



**Sedona Spa Treatment Table**



#### Legal Notice

The information provided in this document is intended for informational purposes only and is subject to change without notice. The information provided in this document should not be construed as a commitment by Earthlite®.

Earthlite® assumes no responsibility for the accuracy or suitability of the information contained in this document. This document may contain technical inaccuracies or errors. Earthlite® makes no expressed or implied warranties of any kind concerning the information contained in this document, including, but not limited to, the implied warranties of merchantability and fitness for purpose.

Earthlite® shall not be liable for incidental or consequential damages in connection with or arising out of the product, performance, or use of this document and the information and material it describes.

© Copyright 2019  
Earthlite®

All rights reserved. This document or any portion thereof may not be reproduced in any way, including photocopying or translation to another language without the prior written permission from Earthlite.

Earthlite® is a registered trademark of Earthlite, LLC.

# Introduction

Congratulations on your purchase of the Sedona spa treatment table. A favorite of day spas, health clubs, and private practitioners, the durable and versatile Sedona is a cost-effective solution for any treatment table need.

Earthlite tables are designed for licensed therapists and are intended to be used in a professional spa, massage studio, or similar commercial environment. While no special training is needed to operate the table, for safety reasons it is important that the instructions contained in this manual are reviewed in detail before the first use. This manual should be read in its entirety by each professional who will be operating the table to ensure the safety of both operator and client. This manual should be saved for future reference.

The table should not be used in any other manner than described in this manual. Please read, understand, and follow all safety warnings, labels, and instructions. Misuse, use of unnecessary force, or using the product for unintended or unauthorized procedures can result in injury to operator, client, and/or equipment.

Detailed instructions for use and maintenance can be found in this user manual. If you have further questions, please contact us at (800) 872-0560 for US customers, or (760) 599-1112 for international customers. We can also be reached via e-mail at [info@earthlite.com](mailto:info@earthlite.com).

## Table of Contents

### Section 1: Safety Information

Safety Symbols .....	2
Warnings .....	2-3

### Section 2: Product Overview

Product Photos .....	4
Product Specifications .....	5

### Section 3: Installation

Moving Your Table .....	6
Leveling Instructions .....	6
Weight/Lift Capacity .....	6

### Section 4: Operating Instructions

Height Adjustment .....	7
Pneumatic Tilt/Salon Adjustment .....	7

### Section 5: Care & Maintenance

Cleaning & Ongoing Maintenance .....	8
Troubleshooting .....	8
Warranty Information .....	9
Contact Us .....	back cover

# SECTION 1: SAFETY INFORMATION

## Safety Symbols

Familiarize yourself with the following Safety & Warning symbols. They are designed to prevent damage and injury to you, your clients, and your new Earthlite product.

**IMPORTANT:** The important safety instructions and warnings in this manual cannot cover all possible problems and conditions that can occur. Use common sense and caution when installing, operating, or maintaining this appliance.



Indicates an imminently hazardous situation which could result in serious or fatal injury.



Indicates a potentially hazardous situation which could result in serious injury.



Indicates a potentially hazardous situation which could result in equipment damage and/or injury.

## IMPORTANT SAFETY INSTRUCTIONS - FOR COMMERCIAL USE ONLY

Read all instructions before using this product.

### WARNING

#### To reduce the risk of injury to persons:

- 1) Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
- 2) Do not operate this equipment unless properly trained.
- 3) Install this furnishing in treatment area not accessible to the public.
- 4) Close supervision is necessary when this furnishing is used by, or near children, invalids, or disabled persons.
- 5) This product does not have "pinch point" protection. Only operate when no possibility of any portion of the body could be caught and injured while moving the table top upward/downward, or activating the back/foot tilt.
- 6) Do not use this furnishing to support video equipment such as televisions or computer monitors.
- 7) Never attempt to use the Head or Leg lift section to support concentrated weight such as sitting or standing

### CAUTION

1) When lowering any section of the table, make sure that no supplies or any part of yours or your client's body is between the top of your table and its base. This could result in injury, or could damage your table as well as the item. It is recommended that you always return your table top to the lowest flat position to avoid leaving the base exposed.


- 2) When lowering the table height, be sure there are no foreign obstacles underneath your table that could obstruct it from lowering completely. This could damage your table.
- 3) Never set anything under the table frame to avoid collision if lowered.
- 4) Never operate your table on an uneven surface. Adjust the leveling feet so that your table is stable.
- 5) Never raise the head or leg section with a client in the prone position (face down.)
- 6) Never use or store your table in a wet or very humid environment. Water and excessive moisture can damage the wood cabinet. Evidence of exposure to water or moisture will void your warranty.
- 7) If you have purchased the Flexible Armrest Option, never use the Armrests as handholds or body support mechanisms, especially when setting clients on or off the table.
- 8) Never drop or insert any object into any opening.

READ AND SAVE THESE INSTRUCTIONS


# SECTION 1: SAFETY INFORMATION

## WARNING

Always use extreme caution when making any position adjustments to your table. Ensure all body parts and objects are clear from all moving parts. The described adjustments can be done with a client on the table, but use caution to ensure that the client's weight is supported while positioning the table section(s).

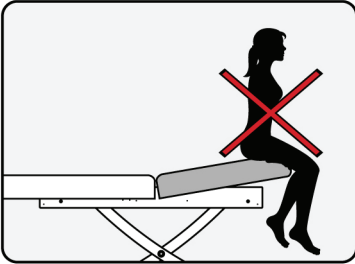

Do not sit or put unnecessary amounts of pressure on the back section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.

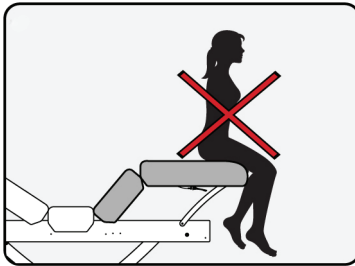


Do not sit or put unnecessary amounts of pressure on the leg section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.

## CAUTION

DO NOT store or place anything under the backrest - this can damage the top and/or tilting mechanism. This area is not designed for storage or as an area to hold accessories and/or massage supplies while performing any type of service. Placing anything in this area can cause damage to the table and/or object and will void your warranty.

# SECTION 2: PRODUCT OVERVIEW

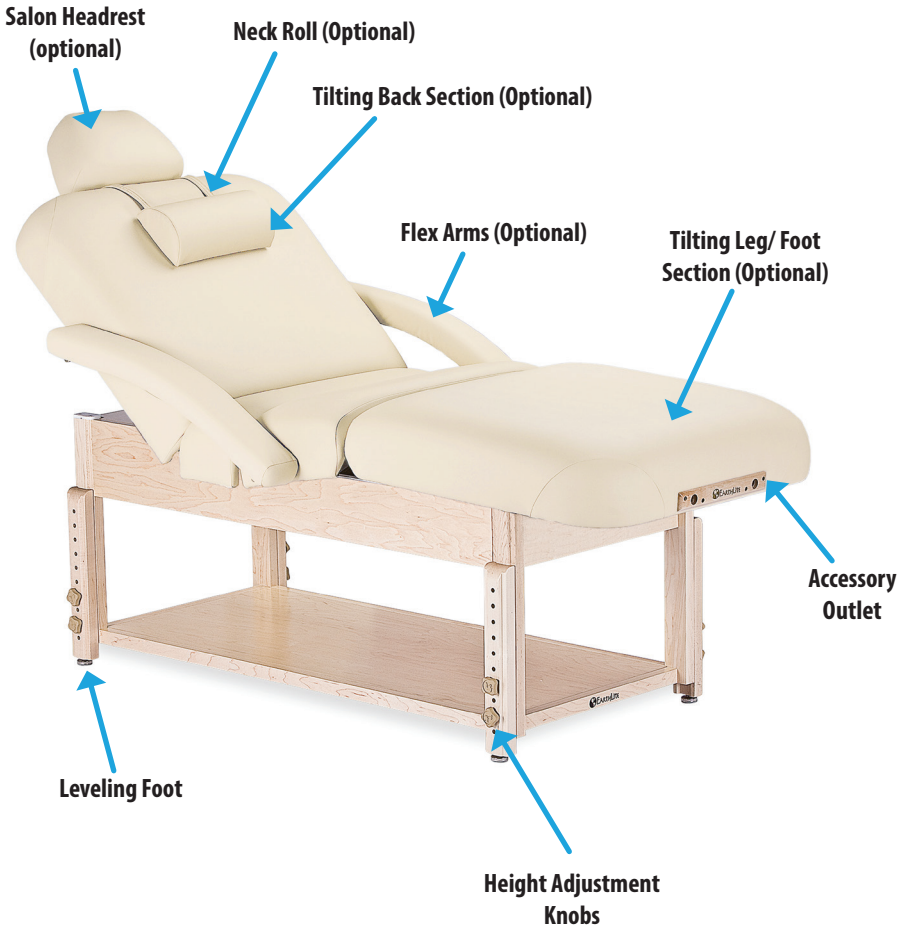

## Base Options

		
<b>Shelf Base</b>	<b>Trestle Base</b>	<b>Cabinet Base</b>

*\*table shown with optional accessories and/or finishes. Measurements may vary due to natural material variations.*

## SECTION 2: PRODUCT OVERVIEW

### Product Specifications

<b>Top Length</b>	73" (185 cm)
<b>Top Width</b>	28", 30", 32" (71 cm, 76 cm, 81 cm)
<b>Top Options</b>	Flat, Manual Tilt, Pneumatic Tilt, Pneumatic Salon
<b>Base Options</b>	Shelf, Trestle, Cabinet
<b>Height Range</b>	24" - 33" (61 cm - 84 cm)
<b>Weight Capacity</b>	800 lbs (363 kg)

### Included Features

<b>Feature</b>	<b>Standard</b>	<b>Optional</b>
<b>Hardwood Frame</b>	X	
<b>Natursoft™ Upholstery</b>	X	
<b>3.5" Deluxe Spa Cushioning</b>	X	
<b>Massage Flat Top</b>	X	
<b>Shelf or Trestle Base</b>	X	
<b>Cabinet Base</b>		X
<b>Manual Tilt, Pneumatic Tilt, Pneumatic Salon Top</b>		X
<b>Flex Arms</b> <i>(not available on manual tilt or flat)</i>		X
<b>Strata Cushioning</b>		X
<b>Caress or Flex-Rest platform</b>		X
<b>Aromatherapy Platform and Arm Shelf</b>		X

## SECTION 3: INSTALLATION

Your Sedona table comes completely assembled and ready to use. Once the table is unpacked, follow the “Moving Your Table” instructions below.

### Moving Your Table



Always use caution when moving your table - always use proper safety procedures/equipment.




Lift your table by grasping firmly from the bottom of the frame underneath the upholstered top. Do not try to lift from the upholstered top.

### Leveling Your Table

Leveling, or balancing, your table is very important during initial setup and routine maintenance. Your table is equipped with leveling feet that can adjust to uneven floors.


To adjust the leveler feet:

1. Place the table in the position you plan to use it.
2. To determine which foot needs adjusting, attempt to “rock” your table. If there is any movement, the leg that is lifting is the one that will need adjusting.
3. Raise table height approximately 6 inches.
4. Using your hand, or a wrench if necessary, turn the leveling foot in the needed direction. (Counterclockwise will lower the overall height, clockwise will raise the overall height)

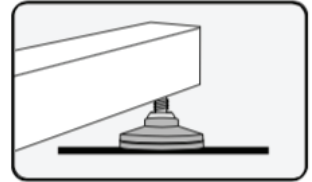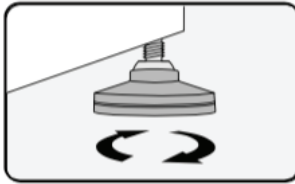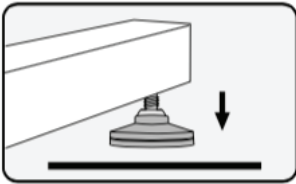

### Table Accessories

All Sedona tables come with outlets at both ends, specifically designed for providing a fixed location for FacePillow platforms, salon pillows, and other accessories.

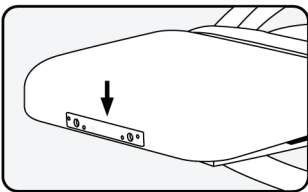

See the Earthlite website for all available accessories that will fit into the table outlets.

### Weight & Lift Capacities

**800 lbs.**

The Sedona maximum weight capacity is 800 lbs.

**50 lbs.**

The back section and leg section has a maximum weight capacity of 50 lbs. These sections are not meant to hold full body weight.



# SECTION 4: OPERATING INSTRUCTIONS

## Height Adjustment

**Note:** You will need to either turn your table on its side or have a second person to hold the table in a tilted position while you adjust the legs.

**Step 1:** Locate the adjustment knobs and turn counterclockwise to loosen.

**Step 2:** Remove knobs to expose the screws.

**Step 3:** Remove the outer part of the leg.

**Step 4:** Move outer part of the leg up or down to adjust table height. When leg is at the desired position, replace knobs and tighten securely by turning clockwise.



Step 1



Step 2



Step 3

## Pneumatic Tilt/Salon Adjustment (Optional)

To adjust, compress the lever to release, allowing you to reposition the angle of the table section. Using your other hand, support the table section until the desired angle is reached, and release the lever to lock into place.

**Note:** Adjustments can be done with a client on the table, however, use caution to ensure that the client's weight is supported with your hand that is guiding the table section.



One-Touch Lever

Lever location for head tilt section (underneath table top)



Lever location for leg/foot tilt section (underneath table top)



## SECTION 5: CARE & MAINTENANCE

### Cleaning & Ongoing Maintenance



DO NOT USE acetone, "Armor All" type cleaners, citrus-based cleaners, or industrial cleaners. These can seriously damage the upholstery. Such damage would not be covered under warranty.

With proper maintenance, your table is designed to give you years of trouble-free operation. To ensure your table will continue to function properly and keep its appearance, we recommend the following.

Category	Care Recommendation
Upholstery	Remove day-to-day dirt and smudges with a mild soap and water solution. For tough stains and to disinfect, use a (100 to 1) water to bleach solution. Spray a damp cloth with the solution to clean. After each cleaning, gently wipe the upholstery with a damp cloth and clean water, then allow upholstery to air dry before using or storing. For more difficult stains, please contact your dealer or please call us directly for guidance. For best results, use only the products suggested above.
Wood	To maintain the wood finish and retouch scratches, use a high quality furniture wax.
Leveling Feet	Regularly check to ensure the table is level and stable. See page 6 for leveling instructions.

### Troubleshooting

This helpful troubleshooting guide will provide you with information about how to recognize a problem with your table, determine possible causes, and find a solution.

In the event your specific situation is not covered in this guide, please contact Earthlite so our technical support department can assist you. Beyond the troubleshooting guide below, do not attempt to repair the table yourself. This can void your warranty.

Unauthorized service will not be covered. Please be sure to contact Earthlite Customer Service first at (800) 872-0560, (760) 599-1112 Int'l.

Issue	Solution
One-Touch Lever is difficult to move	Check the cable for any kinks or sharp bends from the lever to the gas springs. The cable should have a soft radius in order to allow the inner cable to travel freely in the cable sheath. If the cable is kinked, contact Living Earth Crafts.

## SECTION 5: CARE & MAINTENANCE

---

### **Absolute Satisfaction Return Policy**

If you are not completely satisfied with your Earthlite portable table, chair, or accessory, you may return it within thirty (30) days for a full refund (excluding shipping & handling charges on the original order). Standard stationary and lift tables may be returned within thirty (30) days for a refund (excluding shipping & handling charges on the original order) less a 20% restocking fee. Custom or upgraded products are non-returnable and non-refundable. Proof of purchase is required to obtain RMA (Return Material Authorization) and all returns must be in like-new condition and in the original packaging. Customers will be responsible for all shipping charges.

### **Warranty Information**

Earthlite offers an industry best warranty against defects in materials, design, and workmanship, to the original owner, as follows:

- Frame - lifetime warranty
- Foam and upholstery covering - warranted for three (3) years
- Lift motors, controls, electronics - warranted for two (2) years from the original date of purchase\*
- Accessories and options - warranted for one (1) year from the original date of purchase

This warranty does not cover damage caused by normal wear and tear, misuse, accidents, or neglect, which includes tears in the upholstery and damage caused by cleaners. All warranties are limited to repair or replacement at Earthlite's discretion, shipping not included. Proof of purchase is required to obtain RMA (Return Material Authorization) and customers may be required to return defective items. International customers are responsible for all associated return costs (inbound and outbound), as well as any applicable duties and taxes.

\*If within these two (2) years, there is a defect, malfunction, or failure of the lift motor or controls, Earthlite will replace or repair the defective component at our discretion; shipping not included. Parts are warranted for 90 days after replacement or 2 years from the original date of table purchase, whichever comes later.



# EarthLite®

WORLD'S #1 BRAND IN MASSAGE™

## Contact Us

Earthlite  
990 Joshua Way  
Vista, CA. 92081  
(800) 872-0560  
(760) 599-1112 Int'l  
(866) 599-7374 fax  
info@earthlite.com

Hours of operation:

Monday - Friday  
8:00 a.m. to 5 p.m. PST

Saturday - Sunday  
Closed

Visit us online at: [earthlite.com](http://earthlite.com)



Join us on  
Facebook



Follow us on  
Twitter



Connect with us  
on Linked In

