



Terra Spa Treatment Table



© Copyright 2019  
Earthlite®

All rights reserved. This document or any portion thereof may not be reproduced in any way, including photocopying or translation to another language without the prior written permission from Earthlite.

Earthlite® is a registered trademark of Earthlite, LLC.

#### Legal Notice

The information provided in this document is intended for informational purposes only and is subject to change without notice. The information provided in this document should not be construed as a commitment by Earthlite®.

Earthlite® assumes no responsibility for the accuracy or suitability of the information contained in this document. This document may contain technical inaccuracies or errors. Earthlite® makes no expressed or implied warranties of any kind concerning the information contained in this document, including, but not limited to, the implied warranties of merchantability and fitness for purpose.

Earthlite® shall not be liable for incidental or consequential damages in connection with or arising out of the product, performance, or use of this document and the information and material it describes.

# Introduction

Congratulations on your purchase of the Terra spa treatment table. A favorite of day spas, health clubs, and private practitioners, the durable and versatile Terra is a cost-effective solution for any treatment table need.

Terra tables are designed for licensed therapists and are intended to be used in a professional spa, massage studio, or similar commercial environment. While no special training is needed to operate the table, for safety reasons it is important that the instructions contained in this manual are reviewed in detail before the first use. This manual should be read in its entirety by each professional who will be operating the table to ensure the safety of both operator and client. This manual should be saved for future reference.

The Terra table should not be used in any other manner than described in this manual. Please read, understand, and follow all safety warnings, labels, and instructions. Misuse, use of unnecessary force, or using the product for unintended or unauthorized procedures can result in injury to operator, client, and/or equipment.

Detailed instructions for use and maintenance can be found in this user manual. If you have further questions, please contact us at (800) 872-0560 for US customers, or (760) 599-1112 for international customers. We can also be reached via e-mail at [info@earthlite.com](mailto:info@earthlite.com).

## Table of Contents

### Section 1: Safety Information

Safety Symbols .....	2
Warnings .....	2

### Section 2: Product Overview

Product Photos .....	3
Product Specifications .....	3

### Section 3: Installation

Assembling Your Table .....	4-5
-----------------------------	-----

### Section 4: Operating Instructions

Height Adjustment .....	6
Weight/Lift Capacity .....	6

### Section 5: Care & Maintenance

Cleaning & Ongoing Maintenance .....	7
Troubleshooting .....	7
Warranty Information .....	8
Contact Us .....	back cover

# SECTION 1: SAFETY INFORMATION

## Safety Symbols

Familiarize yourself with the following Safety & Warning symbols. They are designed to prevent damage and injury to you, your clients, and your new Earthlite product.

**IMPORTANT:** The important safety instructions and warnings in this manual cannot cover all possible problems and conditions that can occur. Use common sense and caution when installing, operating, or maintaining this appliance.



Indicates an imminently hazardous situation which could result in serious or fatal injury.



Indicates a potentially hazardous situation which could result in serious injury.



Indicates a potentially hazardous situation which could result in equipment damage and/or injury.

## IMPORTANT SAFETY INSTRUCTIONS - FOR COMMERCIAL USE ONLY

Read all instructions before using this product.

### **WARNING**

**To reduce the risk of injury to persons:**

- 1) Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
- 2) Do not operate this equipment unless properly trained.
- 3) Install this furnishing in treatment area not accessible to the public.
- 4) Close supervision is necessary when this furnishing is used by, or near children, invalids, or disabled persons.
- 5) Do not use this furnishing to support video equipment such as televisions or computer monitors.
- 6) Do not adjust table height with client on the table.

### **CAUTION**

1) When lowering the table height, be sure there are no foreign obstacles are underneath your table that could obstruct it from lowering completely. This could damage your table.

- 2) Never set anything under the table frame to avoid collision if lowered.
- 3) Never operate your table on an uneven surface. Adjust the leveling feet so that your table is stable.
- 4) Never use or store your table in a wet or very humid environment. Water and excessive moisture can damage the wood cabinet. Evidence of exposure to water or moisture will void your warranty.
- 5) Never drop or insert any object into any opening.

# SECTION 2: PRODUCT OVERVIEW

---



## Product Specifications

<b>Top Length</b>	73" (185 cm)
<b>Top Width</b>	30" (76 cm)
<b>Height Range</b>	24" - 33" (61 cm - 84 cm)
<b>Weight Limit</b>	800 lbs (363 kg)
<b>Standard Features &amp; Accessories</b>	<ul style="list-style-type: none"><li>• Hardwood Frame</li><li>• Shelf Base</li><li>• Massage Flat Top</li><li>• 3" Deluxe Spa Cushioning</li><li>• Nature's Touch Upholstery</li><li>• Deluxe Adjustable Headrest</li></ul>

*\*Measurements may vary due to natural material variations.*

# SECTION 3: INSTALLATION

## Assembling Your Table



Always use caution when moving your table - always use proper safety procedures/equipment.

### Included Hardware



Long Bolt (x4)



Short Bolt (x8)



Allen Wrench

### Instructions:



Step 1 - Lay the base/legs flat on the floor as shown



Step 2 - Place base on its side and swing legs out




Step 3 - Using 2 people, lift by the shelf base and place so legs sit flush against the top as shown above

## SECTION 3: INSTALLATION



**Step 4** - Using allen wrench, secure the long bolts at the head and foot end of the table (1 bolt per leg, 4 total) NOTE: Do not overtighten bolts

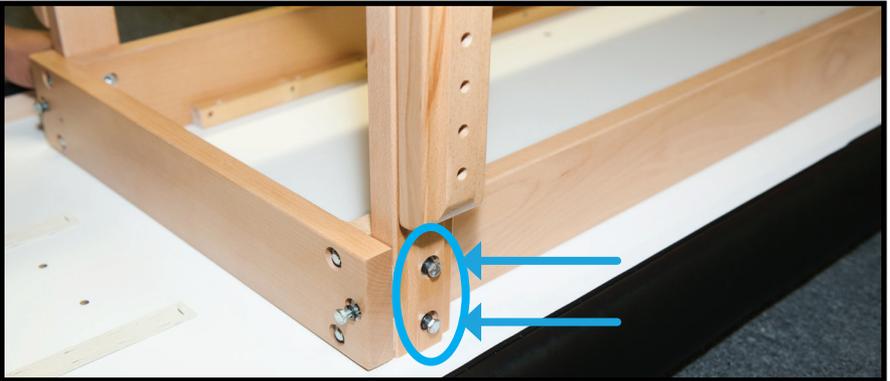

**Step 5** - Using allen wrench, secure the short bolts on the sides of the table (2 bolts per leg, 8 total) NOTE: Do not overtighten bolts



**Step 6** - Tighten the bolt shown above on each leg



**Step 7** - Tighten the height adjustment knobs on each leg

**Step 8** - Tighten the bolts shown in steps 4 and 5

## SECTION 4: OPERATING INSTRUCTIONS

---

### Height Adjustment

**Note:** You will need to either turn your table on its side or have a second person to hold the table in a tilted position while you adjust the legs.



**Step 1:** Locate the adjustment knobs and turn counterclockwise to loosen.



**Step 2:** Remove knobs to expose the screws.



**Step 3:** Remove the outer part of the leg and move up or down to adjust table height. by turning clockwise.



**Step 4:** When leg is at the desired position, replace knobs and tighten securely

### Weight & Lift Capacities

**800 lbs.**

The Terra maximum weight capacity is 800 lbs.

# SECTION 5: CARE & MAINTENANCE

## Cleaning & Ongoing Maintenance



DO NOT USE acetone, "Armor All" type cleaners, citrus-based cleaners, or industrial cleaners. These can seriously damage the upholstery. Such damage would not be covered under warranty.

With proper maintenance, your table is designed to give you years of trouble-free operation. To ensure your table will continue to function properly and keep its appearance, we recommend the following.

<b>Category</b>	<b>Care Recommendation</b>
Upholstery	Remove day-to-day dirt and smudges with a mild soap and water solution. For tough stains and to disinfect, use a (100 to 1) water to bleach solution. Spray a damp cloth with the solution to clean. After each cleaning, gently wipe the upholstery with a damp cloth and clean water, then allow upholstery to air dry before using or storing. For more difficult stains, please contact your dealer or please call us directly for guidance. For best results, use only the products suggested above.
Wood	To maintain the wood finish and retouch scratches, use a high quality furniture wax.

## Troubleshooting

If you have any issues with your table, please contact Earthlite so our technical support department can assist you. Do not attempt to repair the table yourself. This can void your warranty.

Unauthorized service will not be covered. Please be sure to contact Earthlite Customer Service first at (800) 872-0560, (760) 599-1112 Int'l.

## SECTION 5: CARE & MAINTENANCE

---

### Absolute Satisfaction Return Policy

If you are not completely satisfied with your Earthlite portable table, chair, or accessory, you may return it within thirty (30) days for a full refund (excluding shipping & handling charges on the original order). Standard stationary and lift tables may be returned within thirty (30) days for a refund (excluding shipping & handling charges on the original order) less a 20% restocking fee. Custom or upgraded products are non-returnable and non-refundable. Proof of purchase is required to obtain RMA (Return Material Authorization) and all returns must be in like-new condition and in the original packaging. Customers will be responsible for all shipping charges.

### Warranty Information

Earthlite offers an industry best warranty against defects in materials, design, and workmanship, to the original owner, as follows:

- Frame - lifetime warranty
- Foam and upholstery covering - warranted for three (3) years
- Lift motors, controls, electronics - warranted for two (2) years from the original date of purchase\*
- Accessories and options - warranted for one (1) year from the original date of purchase

This warranty does not cover damage caused by normal wear and tear, misuse, accidents, or neglect, which includes tears in the upholstery and damage caused by cleaners. All warranties are limited to repair or replacement at Earthlite's discretion, shipping not included. Proof of purchase is required to obtain RMA (Return Material Authorization) and customers may be required to return defective items. International customers are responsible for all associated return costs (inbound and outbound), as well as any applicable duties and taxes.

\*If within these two (2) years, there is a defect, malfunction, or failure of the lift motor or controls, Earthlite will replace or repair the defective component at our discretion; shipping not included. Parts are warranted for 90 days after replacement or 2 years from the original date of table purchase, whichever comes later.





# EarthLite®

WORLD'S #1 BRAND IN MASSAGE™

## Contact Us

Earthlite  
990 Joshua Way  
Vista, CA. 92081  
(800) 872-0560  
(760) 599-1112 Int'l  
(760) 477-6071 fax  
info@earthlite.com

Hours of operation:

Monday - Friday  
8:00 a.m. to 5 p.m. PST

Saturday - Sunday  
Closed

Visit us online at: [earthlite.com](http://earthlite.com)



Join us on  
Facebook



Follow us on  
Twitter



Connect with us  
on Linked In

